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1. Clinical and Laboratory Standards Institute. 2005. Approved Guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, Pa.
2. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53-80.
3. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C.
4. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021-0045.

Manufacturer / Výrobce / Producent / Fabrikant / Tootja / Valmistaja / Fabricant / Hersteller / Κατασκευαστής / Gyártó / Ditta produttrice / Gamintojas / Producent / Fabricante / Výrobca / Tillverkare / Производител / Producător / Üretici / Proizvođač / Производитель

Use by / Spotřebuje do / Anvendes før / Houdbaar tot / Kasutada enne / Viimeinkäyttöpäivä / A utiliser avant / Verwendbar bis / Ημερομηνία λήξης / Felhasználhatóság dátuma / Usare entro / Naudokite iki / Brukes før / Stosować do / Utilizar em / Použít do / Usar antes de / Använd före / Используйте до / A se utiliza până la / Son kulanma tarihi / Upotrebiti do / Использовать до /
 YYYY-MM-DD / YYYY-MM (MM = end of month) /
 RRRR-MM-DD / RRRR-MM (MM = konec měsíce) /
 ÅÅÅÅ-MM-DD / ÅÅÅÅ-MM (MM = slutning af måned) /
 JJJJ-MM-DD / JJJJ-MM (MM = einde maand) /
 AAAA-KK-PP / AAAA-KK (KK = kuu lõpp) /
 VVVV-KK-PP / VVVV-KK (kuukauden loppuun mennessä) /
 AAAA-MM-JJ / AAAA-MM (MM = fin du mois) /
 JJJJ-MM-TT / JJJJ-MM (MM = Monatsende) /
 EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα) /
 ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja) /
 AAAA-MM-GG / AAAA-MM (MM = fine mese) /
 MMMM-MM-DD / MMMM-MM (MM = mēnesio pabaiga) /
 ÅÅÅÅ-MM-DD / ÅÅÅÅ-MM (MM = slutten av måneden) /
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) /
 AAAA-MM-DD / AAAA-MM (MM = fim do mês) /
 RRRR-MM-DD / RRRR-MM (MM = koniec mesiac) /
 aaaa-mm-dd / aaaa-mm (mm = fin del mes) /
 ÅÅÅÅ-MM-DD / ÅÅÅÅ-MM (MM = slutet på månaden) /
 ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца) /
 AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii) /
 YYYY-AA-GG / YYYY-AA (AA = ayin sonu) /
 GGGG-MM-DD / GGGG-MM (MM = kraj meseca) /
 ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)

Catalog number / Katalogové číslo / Katalognummer / Catalogusnummer / Kataloogi number / Tuotenumero / Numéro catalogue / Bestellnummer / Αριθμός καταλόγου / Katalógusszám / Numero di catalogo / Katalogo numeris / Numer katalogowy / Número do catálogo / Katalogové číslo / Número de catálogo / Καταложен номер / Număr de catalog / Katalog numarası / Kataloški broj / Номер по каталогу

Authorized Representative in the European Community / Autorizovaný zástupce pro Evropskou unii / Autoriseret repræsentant i EU / Erkend vertegenwoordiger in de Europese Unie / Volitatus esindaja Euroopa Nõukogus / Valtuutettu edustaja Euroopan yhteisössä / Représentant agréé pour la C.E.E. / Autorisierte EG-Vertretung / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Hivatalos képviselő az Európai Unióban / Rappresentante autorizzato nella Comunità europea / Įgaliotasis atstovas Europos Bendrijoje / Autoriseret repræsentant i EU / Autoryzowane przedstawicielstwo w Unii Europejskiej / Representante autorizado na União Europeia / Autorizovaný zástupca v Európskom spoločenstve / Representante autorizado en la Comunidad Europea / Auktoriserad representant i EU / Оторизован представитель в EU / Reprezentant autorizat în Uniunea Europeană / Автура Топлулугу Yetkili Temsilcisi / Ovlašteni predstavnik u Evropskoj zajednici / Уполномоченный представитель в Европейском сообществе

In Vitro Diagnostic Medical Device / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medisch hulpmiddel voor in vitro diagnose / In vitro diagnostika meditsiiniaparatuur / Lääkinnällinen in vitro -diagnostiikkalaitte / Dispositif médical de diagnostic in vitro / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / In vitro diagnosztikai orvosi eszköz / Dispositivo medico diagnostico in vitro. / In vitro diagnostikos prietais / In vitro diagnostik mediskins utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Medicinska pomôcka na diagnostiku in vitro / Dispositivo médico de diagnóstico in vitro / Medicinsk anordning för in vitro-diagnostik / Медицинский уред за диагностика ин витро / Aparatură medicală de diagnosticare in vitro / In Vitro Diagnostik Tibbi Cihaz / Medicinski uređaj za in vitro dijagnostiku / Медицинский прибор для диагностики in vitro

Temperature limitation / Teplotní omezení / Temperaturbegrænsning / Temperatuurlimiet / Temperatuuri piirang / Lämpötilarajoitus / Température limite / Zulässiger Temperaturenbereich / Όριο θερμοκρασίας / Hörmérsékleti határ / Temperatura limite / Laikymo temperatūra / Temperaturbegrænsning / Ograniczenie temperatury / Limitação da temperatura / Ohraničenie teploty / Limitación de temperatura / Temperaturbegrænsning / Температурни ограничения / Limitare de temperatură / Sıcaklık sınırlaması / Ograničenje temperature / Ограничение температуры

Batch Code (Lot) / Kód (číslo) šarže / Batch kode (Lot) / Chargennummer (lot) / Partii kood / Eräkoodi (LOT) / Code de lot (Lot) / Chargencode (Chargenbezeichnung) / Κωδικός παρτίδας (Παρτίδα) / Tétel száma (Lot) / Codice del lotto (partita) / Partijos numeris (Lot) / Batch-kode (Serie) / Kod partii (seria) / Código do lote (Lote) / Kód série (šarža) / Código de lote (Lote) / Satskod (parti) / Код (Партида) / Număr lot (Lotul) / Parti Kodu (Lot) / Kod serije / Код партии (лот)

Contains sufficient for <n> tests / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> test / Voldoende voor <n> tests / Küllaldane <n> testide jaoks / Sisältöön riittävä <n> testejä varten / Contenu suffisant pour <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα <n> εξετάσεις / <n> teszthez elegendő / Contenuto sufficiente per <n> test / Pakankamas kiekis atlikti <n> testų / Innholder tilstrekkelig for <n> tester / Zawiera ilość wystarczającą do <n> testów / Contém suficiente para <n> testes / Obsah vystačí na <n> testov / Contenido suficiente para <n> pruebas / Räckertill <n> antal tester / Съдържанието е достатъчно за <n> теста / Conține suficient pentru <n> teste / <n> testleri için yeterli miktarda içerir / Sadržaj dovoljan za <n> testova / Достаточное для <n> тестов(а)

Becton, Dickinson and Company
 7 Loveton Circle
 Sparks, MD 21152 USA

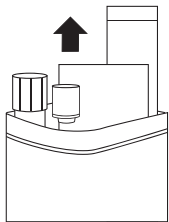
Benex Limited
 Pottery Road, Dun Laoghaire
 Co. Dublin, Ireland

RoboGrip is a trademark of Allied Concepts, Inc.

BD, BD Logo and Affirm are trademarks of Becton, Dickinson and Company. © 2014 BD.

Affirm VPIII Ambient Temperature Transport System

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