

BD SafetyGlide™ Syringe Usage Guidelines


1



Aspiration

- Rotate the safety arm as needed to read the scale.
- Puncture the **center** of the vial to minimize potential bending of the needle.
- Aspirate the medication following standard aseptic procedure.
- **Helpful Hint:** Support the vial during aspiration to avoid bending the needle.

2

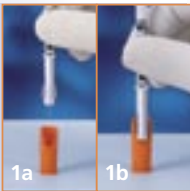

Re-Capping

- If there is a need to re-cap a clean needle for medication transport — re-cap using one of the two following techniques:

1. Cap Stand: (1a) Stand the cap on a level surface, aim the needle into the center of the cap with the safety arm aligned in the slot, and **(1b)** insert until it snaps securely.

OR

2. Scoop Technique: place the flat part of the cap on a level surface, aim the needle into the center of the cap, and push the base of the cap onto the syringe until it snaps securely.



3



Injection

- To facilitate a low angle injection, rotate the safety arm so that it is oriented to the needle bevel.
- Give the injection following standard aseptic procedure.

4



Activation

- Immediately after the injection, use your thumb or forefinger to activate the safety mechanism by pushing the safety arm completely forward.
- The safety arm is locked and fully extended when you hear the click and the needle tip is covered.

5



Disposal

- Dispose of the BD SafetyGlide™ Syringe in an approved sharps container.