

BD Twinpak™

Dual Cannula Device



The BD Twinpak™ is a truly universal IV access device, offering twice the performance of traditional methods. It features two cannulas in one shield; one red hub cannula for syringe filling and one clear, blunt plastic cannula for safe split septum access. The Twinpak helps to streamline IV workflow. It reduces the number of individual pieces, thereby saving time and reducing costs.

**Twice the
Performance,
Twice the
Convenience**



Fill...

Fill syringe from single-dose vials or multidose vials.



Flip...

Flip Twinpak over to attach the BD™ Blunt Plastic Cannula to syringe.



Note:
When expelling air/fluid from a syringe, contents exit sideways from the cannula in two separate paths. Expel air or fluid carefully, directing flow paths away from face or mucous membranes.

This is a "photo reminder". Please consult the detailed Twinpak inservice poster for more specific directions for use.



Access...

Access pre-slit injection site safely using the BD Blunt Plastic Cannula.