


BD SafetyGlide™ Syringe for Insulin, TB, and Allergy

QUICK REFERENCE CARD

Rotate the Safety Arm


1

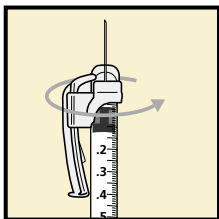

Aspiration

- Aspirate medication into syringe as per your usual technique
- Safety arm can be rotated for scale readability

2

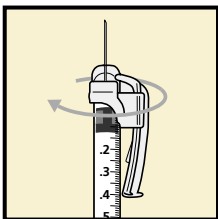

Injection



- Administer injection following established technique
- To facilitate a low angle injection, safety arm can be rotated so it is oriented to the needle bevel

BD SafetyGlide™ Syringe for Insulin, TB, and Allergy

QUICK REFERENCE CARD

Lock the Safety Arm

3

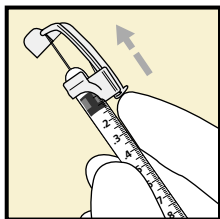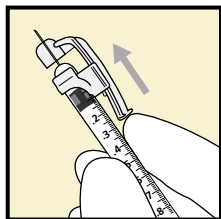

Activation

- After injection, apply a single finger stroke to activate the safety arm by moving it completely forward
- Safety arm is locked and fully extended when you hear the click and the needle tip is covered

Things to Remember:

- Aseptic technique and proper skin preparation of the site are essential
- Discard SafetyGlide syringe in the nearest sharps collector. Follow your institution's policy for safe disposal of all medical waste