

Safety made simple.

BD Eclipse™ Needles. Important Usage Guidelines.

Step 1.



Firmly attach needle onto a Luer Lock* syringe with a push and clockwise twist.

Step 2.



Pull back on the safety cover toward the syringe and away from the needle.

Remove needle shield.

Step 3.



Draw up medication as per your usual technique.

NOTE: If changing needles, activate safety cover prior to removing the needle from the syringe.



Inject according to protocol.

For user convenience, the needle “bevel-up” position is oriented to the safety cover.

Step 4.



After injection, immediately activate safety cover as follows:

- Using the same hand holding the syringe, center thumb or forefinger on the finger pad area of the safety cover.
- Push the cover forward toward the needle until you hear and/or feel it lock.

Visually confirm that the needle tip is covered.



After activation, discard immediately into an approved sharps container.

*For use with Luer Lock syringe only.

NEW ISO NEEDLE HUB/PACKAGING CHART

27G	25G	23G	22G	21G	18G
-----	-----	-----	-----	-----	-----

BD Medical
2771 Bristol Circle
Oakville, ON L6H 6R5
1.800.268.5430
www.bd.com