

BD Eclipse™ Needle


QUICK REFERENCE CARD

1




- Firmly attach needle onto a Luer Lock syringe with a push and clockwise twist.

2a

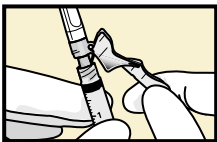


- Pull back on the safety cover toward the syringe and away from the needle.

2b

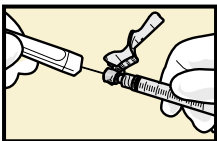

- Grasp the syringe with one hand and with the other pull the clear needle shield straight off.


3a

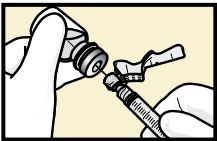

- Draw up medication as per your usual technique.

NOTE: If changing needles, activate safety cover prior to removing the needle from the syringe.

3b


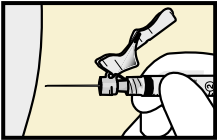


- Inject according to protocol.
- For user convenience, the needle “bevel-up” position is oriented to the safety cover.

BD Eclipse™ Needle

QUICK REFERENCE CARD

4

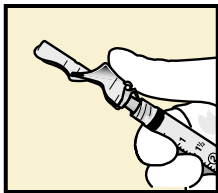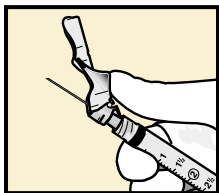

After injection, immediately activate safety cover as follows:

- Using the same hand holding the syringe, center thumb or forefinger on the finger pad area of the safety cover.
- Push the cover forward over the needle until you hear and/or feel it lock.

Visually confirm that the needle tip is covered.

After activation, discard immediately into an approved sharps container.

Things to Remember:

- Aseptic technique and proper skin preparation of the site are essential.
- Follow your institution's policy for safe disposal of all medical waste.
- Not for use on luer slip syringes.