Revisions

SO 0191-5

Rev from	Rev to	ECO #
1103	0105	3276-05

Notes:

1. BD Cat. Number 271034
2. Blank (Sheet) Size : Length: 22" Width: 13"
 Number of Pages: 6 Number of Sheets: 1
 Page Size: Length 22" Width 4.25" Final Folded Size: 2.75" x 4.5"
3. Style (see illustrations below): #7

4. See Specification Control Number N/A for Material Information
5. Ink Colors: Printed two sides Yes No
 No. of Colors: 1 PMS # Black
6. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level

VS Controlled by BD Caribe, Ltd.

Label Design	Date	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION	Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA	
Proofer	Date			
Checked By	Date			
Part Number: 8012591		Category and Description Package Insert BBL CampyPak Microaerophilic System Envelopes	Sheet: 1 of 7 <hr/> Scale: 1:1	A

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INTENDED USE

BBL™ CampyPak™ Microaerophilic System Envelopes are disposable hydrogen and carbon dioxide generating envelopes that produce a microaerophilic atmosphere in **BBL GasPak** jars suitable to support the primary growth and isolation and cultivation of microaerophilic bacteria, such as *Campylobacter*.

SUMMARY AND EXPLANATION

First recognized as a significant agent of diarrhea and acute gastroenteritis in the 1970s, *C. jejuni* is one of the leading causes of bacterial diarrhea in the United States.¹ A reduced oxygen environment enriched with carbon dioxide is essential for the isolation and optimal growth of *Campylobacter* organisms. **CampyPak** envelopes, when used with **GasPak** jars, provide such an atmosphere.

PRINCIPLES OF THE PROCEDURE

The **CampyPak** envelope is activated by the addition of water, which passes through a series of channels to a filter paper wick. The wick regulates the flow of water to the gas-generating tablets in the tablet chamber, providing a controlled release of gases. The wick also delays the introduction of water into the tablet chamber, allowing the lid to be placed on the jar before a significant amount of the gases are released. After the water is added, hydrogen is generated from the sodium borohydride tablet in the tablet chamber and combines, in the presence of a palladium catalyst, with a portion of the oxygen in the jar to form water. The amount of hydrogen generated is insufficient to combine with the total volume of oxygen in the jar resulting in a reduced oxygen atmosphere. Carbon dioxide is generated from the sodium bicarbonate plus citric acid tablet also present in the tablet chamber.

REAGENTS

CampyPak Microaerophilic System Envelope

Each envelope contains: 1 filter paper strip, 1 sodium borohydride tablet, 1 sodium bicarbonate and citric acid tablet.

Warnings and Precautions:

For *in vitro* Diagnostic Use.

All usual precautions attendant to handling hydrogen gas should be observed when employing **CampyPak** hydrogen + carbon dioxide generator envelopes.

Hydrogen is a flammable gas. A mixture of hydrogen gas with oxygen or air in a confined area will explode if ignited by a spark, flame or other source of ignition.

CampyPak envelopes are intended for use only in properly maintained and operated **GasPak 100** and **GasPak 150** systems. Their use in other systems may yield misleading results and may even be hazardous to laboratory personnel. For satisfactory and safe use, carefully follow the instructions provided with each system. One envelope is to be used in each **GasPak 100** system and three envelopes are to be used in each **GasPak 150** system. Never mix **CampyPak** and **CampyPak Plus** envelopes in the same jar (i.e., envelopes with or without integral catalysts).

ANY UNSATISFACTORY ENVELOPE MUST BE CUT OPEN AND EXAMINED FOR INTACT OR PARTIALLY INTACT TABLETS. ANY REMAINING TABLETS SHOULD BE PLACED IN A CONVENIENT SINK AND DISSOLVED IN FLOWING WATER. DO NOT DISCARD INTACT ENVELOPES.

Storage Instructions: On receipt, store envelopes in a dry environment at 2-30°C. Avoid storage in close proximity to laboratory chemicals.

CampyPak envelopes are ready to use and should be activated according to instructions below and on the envelope. The expiration date is for unopened and intact envelopes.

Do not open until ready to use.

Product Deterioration: Do not use if envelopes show evidence of having been punctured, crushed, distorted, exposed to moisture, or exhibit other signs of deterioration.

PROCEDURE

Material Provided: **CampyPak** Microaerophilic System Envelopes.

Materials Required But Not Provided: Culture media in Petri dishes, tubes, bottles or other suitable containers, Incubator (35°C), **GasPak 100** system or **GasPak 150** system, Pipette, 10 mL, or syringe.

Test Procedure:

GasPak 100 and GasPak 150 Anaerobic Systems

- When using **CampyPak** envelopes, place a fresh or rejuvenated palladium catalyst in the reaction chamber of the lid. Palladium catalyst may be rejuvenated by heating at 160 to 170°C for 2 h. Use one catalyst charge (2.5 g ± 0.5) in the reaction chamber of the **GasPak 100** system and one catalyst charge in each of the three reaction chambers of the **GasPak 150** system.
- Place inoculated plates or tubes (in **GasPak** tube holder) in the **GasPak 100** or **GasPak 150** rack. Cut off the corner of the envelope along the dotted line and place the envelope behind the shielded clip of the rack with the printed side toward the outside. One envelope is to be used in each **GasPak 100** system and three envelopes are to be used in each **GasPak 150** system. Do not crease, fold or crush the envelope.
- Add 10 mL of tap, distilled or deionized water through the open/cut corner of the envelope. Use a pipette or syringe inserting only the tip into the open corner. Never push any object down into the envelope.
- Close the **GasPak Jar** promptly after envelope is activated and finger tighten the lid clamp in the manner described in the insert directions for the system being used. Incubate at conditions appropriate for the organisms being cultured, but not above 42°C.
- After use, open the jar and allow it to aerate for approximately 15 s prior to removing the rack with its contents. Do not handle the **CampyPak** envelope until after the rack has been removed from the jar.

User Quality Control:

- Each system should be tested periodically for its ability to provide adequate conditions for growth of *Campylobacter jejuni*.
- If visible condensation does not occur within 60 min after system is activated, the reaction should be terminated by opening jar. Check condition of lid and O-ring gasket; if gasket has lost its resiliency or has developed cracks, bumps, etc., replace with a new gasket. Activate again with one fresh **CampyPak** generator envelope (**GasPak 100** system) or three fresh **CampyPak** envelopes (**GasPak 150** system).
- The following quality control check list will help assure proper performance:
 - System must contain fresh or rejuvenated palladium catalyst, Cat No. 270303, in the reaction chamber(s), Cat. No. 260412.
 - System should be upright at all times while in operation.
 - Envelope:** Keep upright and cut open along the dotted line indicated at the upper right-hand corner. The gas-generating tablets must be present and not displaced or crushed, and the foil channels and reaction chambers must not be crushed or distorted and must be able to accept 10 mL of water.

Improper positioning of envelope or use of a damaged envelope may lead to imbalances in reactivity. Damage may result in spattering and "boiling over" of contents of envelope and hence inadequate gas output.
 - Plastic Lid and Clamp:** Inspect for cracks, chips, split O-ring grooves and other irregularities. Most cracks (craze or stress lines) develop on prolonged use or excessive tightening of clamps on lids. Early cracks may appear as fine black lines near center of lid or along upper edges of the **GasPak 100** clamp near the screw. A cracked lid and clamps exhibiting these fine black lines should not be used.
 - O-Ring Gasket:** Examine for cracks, bumps, dirt and improper seating in O-ring groove. Examine O-ring groove wall for splitting or cracking. O-ring must be resilient, clean and snugly retained within O-ring groove for satisfactory operation of system. Replace as necessary.

Quality control requirements must be performed in accordance with applicable local, state and/or federal regulations or accreditation requirements and your laboratory's standard Quality Control procedures. It is recommended that the user refer to pertinent NCCLS guidance and CLIA regulations for appropriate Quality Control practices.

RESULTS

Conditions suitable for cultivation of microaerophilic organisms are achieved within 60 min in a properly operating **GasPak** jar.

LIMITATIONS OF THE PROCEDURE

The amount of oxygen remaining after 24 to 48 h of incubation is influenced by the number of organisms growing in the jar. When growth of organisms is heavy, colony size of *Campylobacter* isolates may be reduced. This effect may be exaggerated at high altitudes.

PERFORMANCE CHARACTERISTICS

Prior to release, all lots of **BBL CampyPak** Microaerophilic System Envelopes are tested for specific product characteristics. Samples are placed in **GasPak** jars along with opened wet anaerobic indicators. The envelopes are activated according to label directions. The **GasPak** system is carefully observed and the time required for condensate to appear in the jars is noted. Condensate must appear within 2 to 45 min. The jars are then placed in a 42 ± 2°C incubator for 48 h. The anaerobic indicator strips are observed for color change (reduction) after 3 to 8 h and again after 24 h and 48 h. All indicators remain non-reduced (blue) during the 48 h incubation time.

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BENEX Limited