


Revisions

SO 0191-5

Rev from	Rev to	ECO #
2011/05	2011/08	5881-11

Notes:

1. BD Cat. Number: 441748
2. Blank (Sheet) Size: Length: 11" Width: 17"
 Number of Pages: 8 Number of Sheets: 1
 Page Size: Length: 11" Width: 4.25" Final Folded Size: 5.5" x 4.25"
3. Style (see illustrations below): # 7

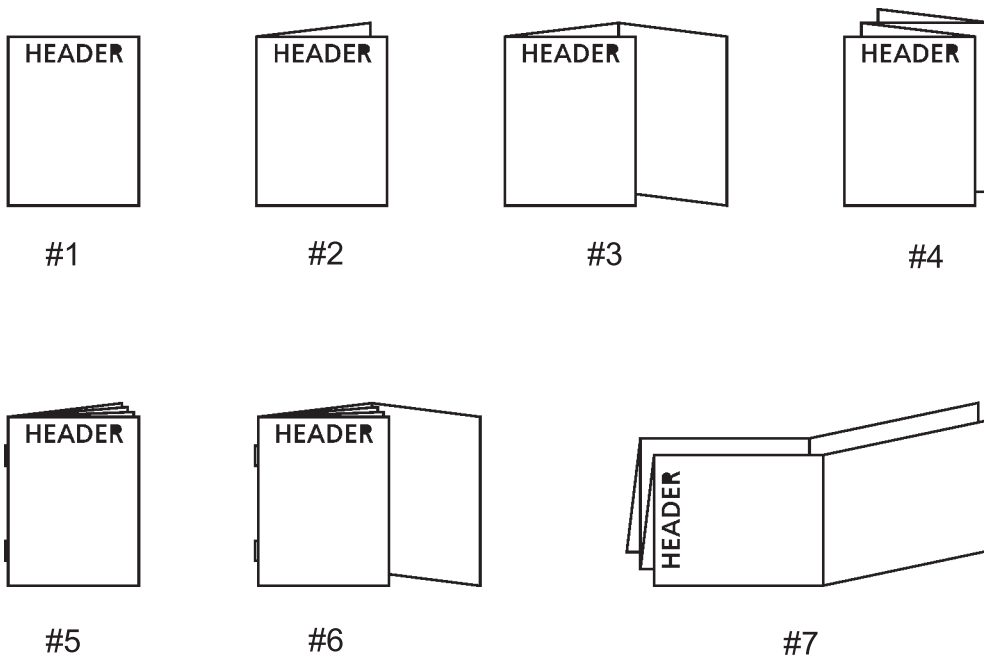

4. See Specification Control Number BALT 8088319 for Material Information.
5. Ink Colors: Printed two sides Yes No
 No. of Colors: 1 PMS # Standard Black
6. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	Date	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	BD Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA	
Proofer	Date			
Checked By	Date			
Part Number: 8088319		Category and Description Package Insert, BD ProbeTec HSV Q* Control Set	Sheet: 1 of 3	A
			Scale: N/A	



Manufacturer / Výrobce / Producent / Fabrikant / Tootja / Valmistaja / Fabricant / Hersteller / Κατασκευαστής / Gyártó / Ditta produttrice / Gamintojas / Producent / Fabricante / Výrobca / Tillverkare / Производител / Producător / Üretici / Proizvođač / Производитель / Атқарушы



Use by / Spotřebujte do / Anvendes før / Houdbaar tot / Kasutada enne / Viimeinkäyttöpäivä / A utiliser avant / Verwendbar bis / Ημερομηνία λήξης / Felhasználhatóság dátuma / Usare entro / Naudokite iki / Brukes før / Stosować do / Utilizar em / Použít do / Usar antes de / Använd före / Используйте до / A se utiliza până la / Son kullanna tarihi / Uпотřebiti do / Используйте до / дейн пайдалануға / Uпотrijebiti do / YYYY-MM-DD / YYYY-MM (MM = end of month) / RRRR-MM-DD / RRRR-MM (MM = konec měsíce) / ÁÁÁÁ-MM-DD / ÁÁÁÁ-MM (MM = slutning af måned) / JJJJ-MM-DD / JJJJ-MM (MM = einde maand) / ÁÁÁÁ-KK-PP / ÁÁÁÁ-KK (KK = kuu lõpp) / VVVV-KK-PP / VVVV-KK (kuukauden loppuun mennessä) / ÁÁÁÁ-MM-JJ / ÁÁÁÁ-MM (MM = fin du mois) / JJJJ-MM-TT / JJJJ-MM (MM = Monatsende) / EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα) / ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja) / ÁÁÁÁ-MM-GG / ÁÁÁÁ-MM (MM = fine mese) / MMMM-MM-DD / MMMM-MM (MM = mensesio pabaiga) / ÁÁÁÁ-MM-DD / ÁÁÁÁ-MM (MM = slutten av måneden) / RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) / ÁÁÁÁ-MM-DD / ÁÁÁÁ-MM (MM = fim do mês) / RRRR-MM-DD / RRRR-MM (MM = koniec mesiacu) / áaaa-mm-dd / áaaa-mm (mm = fin del mes) / ÁÁÁÁ-MM-DD / ÁÁÁÁ-MM (MM = slutet på månaden) / ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца) / ÁÁÁÁ-LL-ZZ / ÁÁÁÁ-LL (LL = sfârșitul lunii) / YYYY-AA-GG / YYYY-AA (AA = ayın sonu) / GGGG-MM-DD / GGGG-MM (MM = kraj meseca) / ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца) / ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА (АА = айдың соңы) / GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)



Catalog number / Katalogové číslo / Katalognummer / Catalogusnummer / Kataloogi number / Tuotenumero / Numéro catalogue / Bestellnummer / Αριθμός καταλόγου / Katalógusszám / Numero di catalogo / Katalogo numeris / Numer katalogowy / Número do catálogo / Katalogové číslo / Número de catálogo / Καταλογος номер / Număr de catalog / Katalog numarası / Kataloški broj / Номер по каталогу / Каталог нөмірі



Authorized Representative in the European Community / Autorizovaný zástupce pro Evropskou unii / Autoriseret repræsentant i EU / Erkend vertegenwoordiger in de Europese Unie / Volitatud esindaja Euroopa Nõukogus / Valtuutettu edustaja Euroopan yhteisössä / Représentant agréé pour la C.E.E. / Autorisierte EG-Vertretung / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Hivatalos képviselő az Európai Unióban / Rappresentante autorizzato nella Comunità europea / Igallotias atstovas Europos Bendrijoje / Autorisert representant i EU / Autoryzowane przedstawicielstwo w Unii Europejskiej / Representante autorizado na União Europeia / Autorizovaný zástupca v Európskom spoločenstve / Representante autorizado en la Comunidad Europea / Auktoriserad representant i EU / Оторизиран представител в EU / Représentant autorizat în Uniunea Europeană / Автура Төрлүлүгү Yetkili Temsilcisi / Ovlašćeni predstavnik u Evropskoj zajednici / Уполномоченный представитель в Европейском сообществе / Европа қауымдастығындағы уәкілетті өкіл / Autorizirani predstavnik u EU



In Vitro Diagnostic Medical Device / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medisch hulpmiddel voor in vitro diagnose / In vitro diagnostika meditsiiniaparatuur / Lääkinnällinen in vitro -diagnostiikkalaitte / Dispositif médical de diagnostic in vitro / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / In vitro diagnostikai orvosi eszköz / Dispositivo médico diagnóstico in vitro. / In vitro diagnostikos prietaisais / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Medicínska pomôcka na diagnostiku in vitro / Dispositivo médico de diagnóstico in vitro / Medicinsk anordning för in vitro-diagnostik / Медицински уред за диагностика ин витро / Aparatură medicală de diagnosticare in vitro / In Vitro Diagnostik Tibbi Cihaz / Medicinski uređaj za in vitro dijagnostiku / Медицинский прибор для диагностики in vitro / Жасанды жағдайда жүргізілетін медициналық диагностика аспабы / Medicínska pomagala za In Vitro Dijagnostiku



Temperature limitation / Teplotní omezení / Temperaturbegrænsning / Temperatuurlimiet / Temperatuuri piirang / Lämpötilarajoitus / Température limite / Zulässiger Temperaturenbereich / Όριο θερμοκρασίας / Hörmérsékleti határ / Temperatura límite / Laikymo temperatūra / Temperaturbegrænsning / Ograniczenie temperatury / Limitação da temperatura / Ohraničenie teploty / Limitación de temperatura / Temperaturbegrænsning / Температури ограничения / Limitare de temperatură / Sıcaklık sınırlaması / Ograničenje temperature / Ограничение температуры / Температураны шектеу / Dozvoljena temperatura



Batch Code (Lot) / Kód (číslo) šarže / Batch kode (Lot) / Chargennummer (lot) / Partii kood / Eräkoodi (LOT) / Code de lot (Lot) / Chargencode (Chargenbezeichnung) / Κωδικός παρτίδας (Παρτίδα) / Tétel száma (Lot) / Codice del lotto (partita) / Partijos numeris (Lot) / Batch-kode (Serie) / Kod partii (seria) / Código do lote (Lote) / Kód série (šarža) / Código de lote (Lote) / Satskod (parti) / Код (Партида) / Număr lot (Lotul) / Parti Kodu (Lot) / Kod serije / Код партии (лот) / Топтама коды / Lot (kod)



Consult Instructions for Use / Prostudujte pokyny k použití / Læs brugsanvisningen / Raadpleeg gebruiksaanwijzing / Lugeda kasutusjuhendit / Tarkista käyttöohjeista / Consulter la notice d'emploi / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Skaitykite naudojimo instrukcijas / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consulte as instruções de utilização / Pozri Pokyny na používanie / Consultar las instrucciones de uso / Se bruksanvisningen / Направете справка в инструкциите за употреба / Consultați instrucțiunile de utilizare / Kullanım Talimatları'na başvurun / Pogledajte uputstvo za upotrebu / См. руководство по эксплуатации / Пайдалану нұсқаулығымен танысып алыңыз / Koristi upute za upotrebu





Negative control / Negativní kontrola / Negativ kontrol / Negative controle / Negatiivne kontroll / Negatiivkontrolli / Contrôle négatif / Negative Kontrolle / Αρνητικός έλεγχος / Negativ kontroll / Controllo negativo / Neigiama kontrolė / Negativ kontroll / Kontrola ujemna / Controllo negativo / Negativna kontrola / Control negativo / Отрицателен контрол / Etalon negatív / Negatív kontrol / Negativna kontrola / Отрицательный контроль / Негативтік бақылау



Positive control / Pozitivní kontrola / Positiv kontrol / Positieve controle / Positiivne kontroll / Positiivkontrolli / Contrôle positif / Positive Kontrolle / Βετικός έλεγχος / Pozitiv kontroll / Controllo positivo / Teigiamo kontrolė / Positiv kontroll / Kontrola dodatná / Controllo positivo / Pozitivna kontrola / Control positivo / Положительен контрол / Etalon pozitiv / Pozitif kontrol / Pozitivna kontrola / Положительный контроль / Жатымды бақылау



 Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA
800-638-8663
www.bd.com/ds

 Benex Limited
Rineanna House
Shannon Free Zone
Shannon, County Clare, Ireland