Revisions

Rev from	Rev to	ECO #
0704	2010/01	5261-10

Notes:

- BD Cat. Number 245031 & 245032
- Blank (Sheet) Size: Length: 22" Width: 8.5"
 Number of Pages: 8 Number of Sheets: 1
 Page Size: Length 5.5" Width 8.5" Final Folded Size: 5.5" x 8.5"
- Style (see illustrations below): # 4

- See Specification Control Number N/A for Material Information
- Ink Colors: Printed two sides Yes No
 No. of Colors: 1 PMS# Black
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level

Label Design	Date	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION	 Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA	
Proofer	Date			
Checked By	Date			
Part Number: 81013301JAA		Category and Description BBL Crystal Panel Viewer Operator Instructions	Sheet: 1 of 9 Scale: N/A	A

BD BBL Crystal™ Panel Viewer **Operator's Instructions**

English: pages 1 – 2 Italiano: pagine 4 – 5
Français: pages 2 – 3 Español: páginas 6 – 7
Deutsch: Seiten 3 – 4

 81013301JAA
2010/01

Pokyny vám poskytne miestni zástupce spoločnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / A használati utasítást kérje a BD helyi képviselőjétől. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielstwem BD. / Contacte o seu representante local da BD para obter instruções. / Instrukcie získate u miestneho zástupcu spoločnosti BD. / Kontakta lokal Becton Dickinson-representant för anvisningar. / Свържете се с местния представител на BD за инструкции. / Contactați reprezentantul dumneavoastră local BD pentru instrucțiuni. / Talimatlar için yerel BD temsilcilerinize danışın. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Для получения инструкций свяжитесь с местным представителем компании BD. / Өзiңiздiң жергiлiктi БД өкiлiне жүгiнiп нұсқау алыңыз. / Kontaktiraj lokalnog predstavnika BD za upute.

INTENDED USE

The **BBL Crystal™** Panel Viewer is for use with the **BBL Crystal ID** Test Panels.

SUMMARY AND EXPLANATION

The **BBL Crystal** Panel Viewer is designed for use with **BBL Crystal ID** Systems. The Panel Viewer illuminates the **BBL Crystal ID** System color reactions with either white light or longwave (365 nm) ultraviolet (UV) light depending upon the position of the panel viewer.

PROCEDURES FOR USE

Set the Panel Viewer on a flat dry surface. Insert the plug of the power cord into a three-wire grounded receptacle providing: Domestic (Model No. 245031) 115 volts, 60 Hz AC only; European (Model No. 245032) 230 volts, 50/60 Hz AC only; and Japanese (Model No. 245033) 100 volts, 50/60 Hz. If a two-wire receptacle is encountered, it must be replaced with a properly grounded three-wire receptacle in accordance with the U.S. National Electric Code. Do not remove the grounding plug from the power plug of this unit. When the unit is not in use, the electrical cord may be removed from the unit for storage purposes.

To turn the Panel Viewer on, press the switch located on the right side of the unit to the "1" position. The internal white light will illuminate the lower panel and the ultraviolet light will illuminate the upper chamber. With the slider in the down position, place an inoculated panel on the white light viewing ledge to allow recording of the data. For proper orientation, the lettering across the top of the panel should read A to J and the lettering along the left side should read 4, 2, 1, from top to bottom. Report forms and instructions on scoring the **BBL Crystal ID** System color reactions are included in each **BBL Crystal ID** Systems kit.

To utilize the ultraviolet illumination, slide the panel up into the UV chamber. Viewing through the preformed viewport will allow recording of the UV-induced fluorescence. The **BBL Crystal** Panel Viewer is equipped with a special contrast control filter which blocks out UV light to prevent direct exposure from the longwave UV.

The angle adjustment bracket can be adjusted to raise or lower the angle of the viewer for comfortable viewing of panels. The bracket, located on the back of the unit, can be adjusted by sliding the bracket to the appropriate angle and tightening the knob.

Indoor use, altitude up to 2000 m, temperature 5 to 40°C, maximum relative humidity of 80% to temperatures up to 31°C linearly to 50% relative humidity at 40°C, mains supply voltage not to exceed ± 10 of the nominal voltage. Installation Category II, Pollution degree 2.

Symbol Definition:

On-Off

Alternating Current

Fuse

PREVENTIVE MAINTENANCE

Do not operate the unit when wet. Store unit in a clean, dry area at room temperature (23 to 29°C). The viewer is painted with a scratch-resistant paint and can be cleaned with commercial cleaners and a damp cloth, **WITH THE UNIT UNPLUGGED**. To avoid direct UV exposure do not operate the unit when the viewport and filter have been removed for cleaning. Handle the filter glass carefully when cleaning. Power cord is the mains disconnect device.

Warnings and Precautions:

For *in vitro* Diagnostic Use.

Read instructions before attempting to operate this device.

INSTRUCTIONS FOR REPLACEMENT OF TUBES

Turn the Panel Viewer off, disconnect the power cord from the wall receptacle and the unit as instructed in the warning label. Remove back cover of the unit by removing the 11 attachment screws. Remove the defective UV or white light tube from the retaining sockets and replace with a new tube. **Caution: defective tube may be warm to touch**. Reattach the back cover using **ALL** of the screws.

Replacement Tube Part Numbers: White Tube Light – Cat. No. 245036, Longwave (UV) Tube – Cat. No. 245034

SERVICE INFORMATION

The operator should not perform any servicing except replacement of light tubes as described above. For other types of services, contact the following companies for response to questions, or to obtain a Return Goods Authorization (RGA) number to return the unit for repair.

UVP, Inc.
2066 W. 11th Street, Upland, CA 91786, U.S.A.
Toll Free: (800) 452-6788
Outside U.S.A.: (909) 946-3197 • Fax: (909) 946-3597

Ultra Violet Products Limited
Uniti, Trinity Hall Estate, Nuffield Road
Cambridge LB4 1TE U.K.
Telephone: 1223 420022 • Fax: 1223 420561

SPECIFICATIONS:

Domestic Model No. 245031

Voltage	Frequency	Cord	AMPS	Unit Size	Weight	Color	Warranty
115	60 Hz	3-wire grounded Type SV	0.32	12½ x 10½ x 9½ in.	9 lbs.	Black Outer Case	1 yr. from date of purchase

European Model No. 245032

Voltage	Frequency	Cord	AMPS	Unit Size	Weight	Color	Warranty
230	50/60 Hz	3-wire grounded Type SV	0.32	31.75 x 26.67 x 24.13 cm	4.1 kg	Black	1 yr. from date of purchase

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Benex Limited