


## Revisions

SO 0191-5

Rev from	Rev to	ECO #
0704	2010/07	5393-10

**Notes:**

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1. BD Cat. Number 295982, 299455, 297809
2. Blank (Sheet) Size : Length: 11"      Width: 8.5"  
 Number of Pages: 2      Number of Sheets: 1  
 Page Size: Length 11"      Width 8.5"      Final Folded Size: N/A
3. Style (see illustrations below): # 1



4. See Specification Control Number N/A for Material Information
5. Ink Colors: Printed two sides  Yes     No  
 No. of Colors: 1      Standard Black
6. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level

Label Design	Date	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION	Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA	
Proofer	Date			
Checked By	Date			
Part Number: 8806291JAA		Category and Description Package Insert BBL Prepared Tubed Media for Cultivation of Anaerobic Microorganisms	Sheet: 1 of 3 <hr/> Scale: 1 : 1	A

# BD BBL™ Prepared Tubed Media for Cultivation of Anaerobic Microorganisms



8806291JAA  
2010/07

## Cooked Meat Medium with Glucose, Hemin and Vitamin K<sub>1</sub>

Pokyny vám poskytne miestní zástupce společnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / A használati utasítást kérje a BD helyi képviselőjétől. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielstwem BD. / Contacte o seu representante local da BD para obter instruções. / Instrukcie získate u miestneho zástupcu spoločnosti BD. / Kontakta lokal Becton Dickinson-representant för anvisningar. / Свържете се с местния представител на BD за инструкции. / Contactați reprezentantul dumneavoastră local BD pentru instrucțiuni. / Talimatlar için yerel BD temsilcilerinize danışın. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Для получения инструкций свяжитесь с местным представителем компании BD. / Өзіндің жергілікті БД өкіліне жүгініп нұсқау алыңыз. / Kontaktiraj lokalnog predstavnika BD za upute.

### INTENDED USE

Cooked Meat Medium with Glucose, Hemin and Vitamin K<sub>1</sub> is a general-purpose medium used for the cultivation of obligate anaerobes, especially *Clostridium* spp.

### SUMMARY AND EXPLANATION

Cooked Meat Medium with Glucose, Hemin and Vitamin K<sub>1</sub> is based on the formulation of Robertson.<sup>1</sup> It supports the growth of most sporeforming and nonsporeforming obligate anaerobes and may be used for a variety of purposes.<sup>2</sup> This medium is also useful as an enrichment broth for cultivating anaerobes that may be present in small numbers and as a subculture medium for determination of proteolysis (meat digestion) and spore formation by clostridia. Cooked Meat Medium with Glucose, Hemin and Vitamin K<sub>1</sub> is also recommended as a subculture medium for anaerobic isolates to be examined by gas-liquid chromatography.<sup>3</sup>

### PRINCIPLES OF THE PROCEDURE

Muscle tissue and animal tissue peptone provide organic nitrogen and other nutrients to support the growth of microorganisms. The muscle tissue also provides reducing substances, particularly glutathione, which permits the growth of strict anaerobes.<sup>4</sup>

Cooked Meat Medium, enriched, is supplemented with added glucose, yeast extract, hemin and vitamin K<sub>1</sub> to enhance the growth of anaerobic microorganisms. Growth is indicated by turbidity and, with some organisms, by the presence of gas bubbles in the medium. Disintegration and blackening of the meat particles indicates proteolysis.

### REAGENTS

#### Cooked Meat Medium Base

Approximate Formula\* Per Liter Purified Water

Heart Tissue Granules	98.0 g
Peptic Digest of Animal Tissue	20.0 g
Dextrose	2.0 g
Sodium Chloride	5.0 g

\* Adjusted and/or supplemented as required to meet performance criteria.

Cooked Meat Medium with Glucose, Hemin and Vitamin K<sub>1</sub> consists of the above ingredients with an additional 3.0 g dextrose, 5.0 g yeast extract, 5.0 mg hemin and 1.0 mg vitamin K<sub>1</sub> added per liter.

#### Warnings and Precautions:

For *in vitro* Diagnostic Use.

Tubes with tight caps should be opened carefully to avoid injury due to breakage of glass.

Observe aseptic techniques and established precautions against microbiological hazards throughout all procedures. After use, prepared tubes, specimen containers and other contaminated materials must be sterilized by autoclaving before discarding.

**Storage Instructions:** On receipt, store tubes in the dark at 2 to 25°C. Avoid freezing and overheating. Tubed media stored as labelled until just prior to use may be inoculated up to the expiration date and incubated for recommended incubation times. Allow the medium to warm to room temperature before inoculation. Do not open until ready to use.

**Product Deterioration:** Do not use tubes if they show evidence of microbial contamination, discoloration, drying or other signs of deterioration.

#### SPECIMEN COLLECTION AND HANDLING

This medium is not suitable for use directly with clinical specimens or other sources containing mixed microbial flora, unless used as an enrichment broth in addition to the primary plating media. Consult appropriate texts for more information.<sup>3,5-9</sup>

#### PROCEDURE

**Material Provided:** Cooked Meat Medium with Glucose, Hemin and Vitamin K<sub>1</sub>

**Materials Required But Not Provided:** Ancillary culture media, reagents, quality control organisms and laboratory equipment as required for this procedure.

**Test Procedure:** Observe aseptic techniques. Liquid media for anaerobic incubation should be reduced by placing the tubes, with caps loosened, under anaerobic conditions for 18 to 24 h prior to use. An efficient and easy way to obtain suitable anaerobic conditions is through the use of a GasPak™ EZ anaerobic system. Alternatively, liquid media may be reduced immediately prior to use by boiling, with caps loosened, and cooling, with tightened caps, to room temperature before inoculation.

Using a sterile inoculating loop or needle, transfer growth from the primary plating medium, inoculating heavily in the area of meat particles. Incubate the tubes at 35°C

under anaerobic conditions for up to 7 days. It is recommended that an indicator of anaerobiosis be used.

If the medium is being used as a back-up enrichment medium, in addition to the primary plating medium, tubes should be held at least 1 week before discarding as negative.

#### User Quality Control:

1. Examine the tubes for signs of deterioration as described under "Product Deterioration."
2. Check performance by inoculating a representative sample of tubes with pure cultures of stable control organisms that produce known, desired reactions.

The following cultures are recommended:

TEST STRAIN	EXPECTED RESULT
<i>Clostridium sporogenes</i> ATCC™ 11437	Growth. Gas produced.
<i>Clostridium perfringens</i> ATCC 13124	Growth. Gas produced.

Quality control requirements must be performed in accordance with applicable local, state and/or federal regulations or accreditation requirements and your laboratory's standard Quality Control procedures. It is recommended that the user refer to pertinent NCCLS guidance and CLIA regulations for appropriate Quality Control practices.

#### RESULTS

Following incubation, growth is indicated by turbidity and, in some cases, by the production of gas. Proteolysis is characteristic of some species and is evidenced by blackening of the meat particles with eventual digestion or dissolution of the meat. A Gram or spore stain should be performed as the location and size of spores is characteristic of *Clostridium* spp.<sup>7</sup>

#### LIMITATIONS OF THE PROCEDURE

Enrichment broths should not be used as the sole isolation medium. They are to be used in conjunction with selective and nonselective plating media to increase the probability of isolating pathogens, especially when they may be present in small numbers in a specimen.

For identification, the organism must be in a pure culture. Biochemical and other identification tests may be performed for complete identification. Appropriate texts should be consulted for further information.<sup>3,5-11</sup>

#### PERFORMANCE CHARACTERISTICS

Prior to release, all lots of Cooked Meat Medium with Glucose, Hemin and Vitamin K<sub>1</sub> are tested for performance characteristics. Using a 0.01 mL calibrated loop, representative samples of the lot are inoculated with cultures of *Clostridium perfringens* (ATCC 13124) and *C. sporogenes* (ATCC 11437). The inocula are taken from either Cooked Meat Medium or from colonies grown on CDC Anaerobe 5% Sheep Blood Agar plates and subsequently diluted to a 1.0 McFarland Standard in Fluid Thioglycollate Medium. The inoculated tubes are incubated in a GasPak™, GasPak™ Plus or GasPak™ EZ anaerobic system at 35 ± 2°C and read after 1, 3 and 7 days incubation. Growth and gas production are evident with both *C. perfringens* and *C. sporogenes*.


#### AVAILABILITY


Cat. No.	Description
295982	BBL™ Cooked Meat Medium with Glucose, Hemin and Vitamin K <sub>1</sub> , Pkg of 10 size K tubes, 9 mL
299455	BBL™ Cooked Meat Medium with Glucose, Hemin and Vitamin K <sub>1</sub> , Ctn. of 100 size K tubes, 9 mL
297809	BBL™ Cooked Meat Medium with Glucose, Hemin and Vitamin K <sub>1</sub> , Ctn. of 100 size C tubes, 10 mL

#### REFERENCES

1. Robertson, M.J. 1916. Notes upon certain anaerobes isolated from wounds. J. Pathol. Bacteriol. 20:327.
2. Dowell, V.R., Jr., G.L. Lombard, F.S. Thompson, and A.Y. Armfield. 1978. Media for isolation, characterization, and identification of obligately anaerobic bacteria. DHEW Publication. Center for Disease Control, Atlanta.
3. Holdeman, L.V., E.P. Cato and W.E.C. Moore (ed.). 1977. Anaerobe laboratory manual, 4th ed. Virginia Polytechnical Institute and State University, Blacksburg.
4. Willis, A.T. 1977. Anaerobic bacteriology: clinical and laboratory practice, 3rd ed. Butterworths, London.
5. Dowell, V.R., Jr., and T.M. Hawkins. 1987. Laboratory methods in anaerobic bacteriology. CDC Laboratory manual. HHS Publication No. (CDC) 87-8272. Centers for Disease Control, Atlanta.
6. Rodloff, A.C., P.C. Appelbaum, and R.J. Zabrusky. 1991. Cumitech 5A, Practical anaerobic bacteriology. Coordinating ed., A.C. Rodloff. American Society for Microbiology, Washington, D.C.
7. Allen, S.D., and E.J. Baron. 1991. *Clostridium*, p. 505-521. In A. Balows, W.J. Hausler, Jr., K.L. Herrmann, H.D. Isenberg, and H.J. Shadomy (ed.), Manual of clinical microbiology, 5th ed. American Society for Microbiology, Washington, D.C.
8. Summanen, P., E.J. Baron, D.M. Citron, C.A. Strong, H.M. Wexler, and S.M. Finegold. 1993. Wadsworth anaerobic bacteriology manual, 5th ed. Star Publishing Company, Belmont, Calif.

9. Murray, P.R., E.J. Baron, M.A. Pfaller, F.C. Tenover, and R.H. Tenover (ed.). 1999. Manual of clinical microbiology, 7th ed. American Society for Microbiology, Washington, D.C.
10. Seip, W.F., and G.L. Evans. 1980. Atmospheric analysis and redox potentials of culture media in the GasPak System. 11:226-233.
11. Engelkirk, P.G., J. Duben-Engelkirk, and V.R. Dowell, Jr. 1992. Principles and practice of clinical anaerobic bacteriology. Star Publishing Company, Belmont, Calif.
12. Marshall (ed.). 1993. Standard methods for the examination of dairy products, 16th ed. APHA, Washington, D.C.


 Manufacturer / Výrobce / Producent / Fabrikant / Tootja / Valmistaja / Fabricant / Hersteller / Κατασκευαστής / Gyártó / Ditta produttrice / Gamintojas / Producent / Fabricante / Výrobca / Tillverkare / Производител / Producător / Üretici / Proizvođač / Производител / Атқарушы

 Use by / Spotřebujte do / Anvendes for / Houdbaar tot / Kasutada enne / Viimeinkäyttöpäivä / A utiliser avant / Verwendbar bis / Ημερομηνία λήξης / Felhasználhatóság dátuma / Usare entro / Naudokite iki / Brukes for / Stosować do / Utilizar em / Usare antes de / Använd före / Исползуйте до / A se utiliza până la / Son kullanna tarihi / Upotrebiti do / Исползовать до / дейін пайдалануға / Upotrijebiti do /  
 YYYY-MM-DD / YYYY-MM (MM = end of month) /  
 RRRR-MM-DD / RRRR-MM (MM = konec měsíce) /  
 ÅÅÅÅ-MM-DD / ÅÅÅÅ-MM (MM = slutning af måned) /  
 JJJJ-MM-DD / JJJJ-MM (MM = einde maand) /  
 AAAA-KK-PP / AAAA-KK (KK = kuu lõpp) /  
 VVVV-KK-PP / VVVV-KK (kuukauden loppuun mennessä) /  
 AAAA-MM-JJ / AAAA-MM (MM = fin du mois) /  
 JJJJ-MM-TT / JJJJ-MM (MM = Monatsende) /  
 EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα) /  
 ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja) /  
 AAAA-MM-GG / AAAA-MM (MM = fine mese) /  
 MMMM-MM-DD / MMMM-MM (MM = mēnesio pabaiga) /  
 ÁÁÁÁ-MM-DD / ÁÁÁÁ-MM (MM = slutten av månaden) /  
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) /  
 AAAA-MM-DD / AAAA-MM (MM = fim do mês) /  
 RRRR-MM-DD / RRRR-MM (MM = koniec mesiac) /  
 aaaa-mm-dd / aaaa-mm (mm = fin del mes) /  
 ÁÁÁÁ-MM-DD / ÁÁÁÁ-MM (MM = slutet på månaden) /  
 ГГГГ-MM-ДД / ГГГГ-MM (MM = края на месеца) /  
 AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii) /  
 YYYY-AA-GG / YYYY-AA (AA = ayın sonu) /  
 GGGG-MM-DD / GGGG-MM (MM = kraj meseca) /  
 ГГГГ-MM-ДД / ГГГГ-MM (MM = конец месяца) /  
 ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА (АА = айдың соңы) /  
 GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)


**REF** Catalog number / Katalogové číslo / Katalognummer / Catalognummer / Kataloogi number / Tuotenumero / Numéro catalogue / Bestellnummer / Αριθμός καταλόγου / Katalógusszám / Numero di catalogo / Katalogo numeris / Numer katalogowy / Número do catálogo / Katalogové číslo / Número de catálogo / Каталоген номер / Număr de catalog / Katalog numarası / Kataloški broj / Номер по каталогу / Каталог нөмірі


**EC REP** Authorized Representative in the European Community / Autorizovaný zástupce pro Evropskou unii / Autoriseret repræsentant i EU / Erkend vertegenwoordiger in de Europese Unie / Volitatud esindaja Euroopa Nõukogus / Valtuutettu edustaja Euroopan yhteisössä / Représentant agréé pour la C.E.E. / Autorisierte EG-Vertretung / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Hivatalos képviselő az Európai Unióban / Rappresentante autorizzato nella Comunità europea / Įgaliotasis atstovas Europos Bendrijoje / Autorisert representant i EU / Autoryzowane przedstawicielstwo w Unii Europejskiej / Representante autorizado na União Europeia / Autorizovaný zástupca v Európskom spoločenstve / Representante autorizado en la Comunidad Europea / Auktoriserad representant i EU / Оторизиран представител в EU / Reprezentant autorizat în Uniunea Europeană / Avrupa Topluluğu Yetkili Temsilcisi / Ovlašćeni predstavnik u Evropskoj zajednici / Уполномоченный представитель в Европейском сообществе / Европа қауымдастығындағы уәкілетті өкіл / Autorizuirani predstavnik u EU

**IVD** In Vitro Diagnostic Medical Device / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medisch hulpmiddel voor in vitro diagnose / In vitro diagnostika meditsiiniaparatuur / Lääkinnällinen in vitro -diagnostiikkalaitte / Dispositif médical de diagnostic in vitro / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / In vitro diagnostikai orvosi eszköz / Dispositivo medico diagnostico in vitro. / In vitro diagnostikos prietais / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Medicínska pomôcka na diagnostiku in vitro / Dispositivo médico de diagnóstico in vitro / Medicinsk anordning för in vitro-diagnostik / Медицински уред за диагностика ин витро / Aparatură medicală de diagnosticare in vitro / In Vitro Diagnostik Tibbi Cihaz / Medicinski uređaj za in vitro dijagnostiku / Медицинский прибор для диагностики ин витро / Жасанды жағдайда жұризетін медициналық диагностика аспабы / Medicínska pomagala za In Vitro Dijagnostiku

 Temperature limitation / Teplotní omezení / Temperaturbegrænsning / Temperatuurlimiet / Temperaturi piirang / Lämpötilarajoitus / Température limite / Zulässiger Temperaturbereich / Όριο θερμοκρασίας / Hölmérsékleti határ / Temperatura limite / Laikymo temperatūra / Temperaturbegrænsning / Ograniczenie temperatury / Limitação da temperatura / Ohraničenie teploty / Limitación de temperatura / Temperaturbegrænsning / Температури ограничения / Limitare de temperatură / Sıcaklık sınırlaması / Ograničenje temperature / Ограничение температуры / Температураны шектеу / Dozvoljena temperatura

**LOT** Batch Code (Lot) / Kód (číslo) šarže / Batch code (Lot) / Chargennummer (lot) / Partii kood / Eräkoodi (LOT) / Code de lot (Lot) / Chargencode (Chargenbezeichnung) / Κωδικός παρτίδας (Παρτίδα) / Tétel száma (Lot) / Codice del lotto (partita) / Partijos numeris (Lot) / Batch-kode (Serie) / Kod partii (seria) / Código do lote (Lote) / Kód série (šarža) / Código de lote (Lote) / Satskod (parti) / Код (Партида) / Număr lot (Lotul) / Parti Kodu (Lot) / Kod serije / Код партии (лот) / Топтама коды / Lot (Kod)

 Consult Instructions for Use / Prostudujte pokyny k použití / Læs brugsanvisningen / Raadpleeg gebruiksaanwijzing / Lugeđa kasutusjuhendit / Tarkista käyttöohjeista / Consulter la notice d'emploi / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Skaitykite naudojimo instrukcijas / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consulte as instruções de utilização / Pozri Pokyny na používání / Consultar las instrucciones de uso / Se bruksanvisningen / Направете справка в инструкциите за употреба / Consultați instrucțiunile de utilizare / Kullanım Talimatları'na başvurun / Pogledajte uputstvo za upotrebu / См. руководство по эксплуатации / Пайдалану нұсқаулығымен танысып алыңыз / Koristi upute za upotrebu

 Becton, Dickinson and Company  
 7 Loveton Circle  
 Sparks, MD 21152 USA  
 800-638-8663  
 www.bd.com/ds

**EC REP** Benex Limited  
 Rineanna House  
 Shannon Free Zone  
 Shannon, County Clare, Ireland