## Revisions

SO 0191-5

Rev from	Rev to	ECO #
0604	2010/06	5370-10

**Notes:**

1. BD Cat. Number 231747
2. Blank (Sheet) Size : Length: 12"      Width: 19.5"  
 Number of Pages: 12    Number of Sheets: 1  
 Page Size: Length 12"    Width 3.25"      Final Folded Size: NA
3. Style (see illustrations below): #4

4. See Specification Control Number N/A for Material Information
5. Ink Colors: Printed two sides  Yes     No  
 No. of Colors: 1                      PMS # Black
6. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level

VS Controlled by BDDS Caribe, Ltd.

Label Design	Date	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION	<b>BD</b> Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA	
Proofer	Date			
Checked By	Date			
Part Number: 8820381		Category and Description Package Insert BD BBL DrySlide PYR Kit	Sheet: 1 of 13 <hr/> Scale: 1 : 1	A

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## INTENDED USE

**BBL™ DrySlide™ PYR Kit**, containing **BBL™ DrySlide™ PYR** and **BBL™ DrySlide™ PYR Color Developer**, is used for detecting hydrolysis of PYR in the rapid presumptive identification of group A streptococci and group D enterococci from pure culture.

**BBL DrySlide PYR**, provided in an easy-to-handle slide format, and **BBL DrySlide PYR Color Developer**, provided in a reagent dispenser, accommodate up to four tests per slide/dispenser.

## SUMMARY AND EXPLANATION

The PYR test is a qualitative procedure for determining the ability of streptococci to enzymatically hydrolyze L-pyrrolidonyl- $\beta$ -naphthylamide (PYR). Facklam, Thacker, Fox and Eriquez<sup>1</sup> reported that 98% of group A streptococci and 96% of group D enterococci hydrolyze PYR. Although *Aerococcus* species are rarely isolated in the clinical laboratory,<sup>2</sup> these organisms are also expected to hydrolyze PYR.<sup>3</sup>

Facklam et al. further reported that 98% of group B streptococci, 100% of non-group A, B and D streptococci, 100% of group D non-enterococci and 82% of viridans streptococci yield negative PYR test results.<sup>1</sup>

## PRINCIPLES OF THE PROCEDURE

L-pyrrolidonyl- $\beta$ -naphthylamide (PYR) is hydrolyzed by bacteria that possess the enzyme pyrrolidonyl peptidase.

Demonstrating PYR hydrolysis involves two reactions. Pyrrolidonyl peptidase, if present, hydrolyzes PYR (**BBL DrySlide PYR**) to liberate L-pyrrolidone carboxylic acid and  $\beta$ -naphthylamine.  $\beta$ -naphthylamine, in turn, reacts with *p*-dimethyl-aminocinnamaldehyde (**BBL DrySlide PYR Color Developer**) to form a pink/fuchsia precipitate.

## REAGENTS

**BBL DrySlide PYR** is a disposable 2" by 2" slide having four filter paper reaction areas containing L-pyrrolidonyl- $\beta$ -naphthylamide.

**BBL DrySlide PYR Color Developer** is 0.015% *p*-dimethyl-aminocinnamaldehyde prepared according to a conventional formulation.

## Warnings and Precautions

For *in vitro* Diagnostic Use.

**Warning:** **BBL DrySlide PYR Color Developer** causes eye, skin or respiratory irritation. Possible cancer hazard (contains material which may cause cancer based on animal data).

Avoid contact with eyes, skin and clothing. Avoid breathing vapor or mist. Wash thoroughly after handling. Use with proper ventilation.

**First Aid:** In case of contact, immediately flush eyes with plenty of water for at least 15 min. Flush skin with plenty of water. In case of contact with eyes or if skin irritation persists, call a physician. Wash clothing before reuse.

If inhaled, remove to fresh air. If not breathing, give artificial respiration. If breathing is difficult, give oxygen. Call a physician.

Handle **BBL DrySlide PYR** only by the slide mount; do not touch the reaction area.

Use aseptic technique and established laboratory procedures in handling and disposing of infectious materials.

**Storage Instructions:** Store **BBL DrySlide PYR Kit** at 15 – 30°C. Protect **BBL DrySlide PYR Color Developer** from light.

The expiration date applies to the products in their intact containers when stored as directed.

**Product Deterioration:** Do not use if the product fails to yield expected results when tested with known positive and negative control organisms.

## SPECIMEN COLLECTION AND PREPARATION

Obtain the test specimen from a primary isolation culture streaked on an appropriate solid medium (i.e., chocolate agar or tryptic soy blood agar).

Observe the culture for purity and sufficient growth.

Mixed cultures (i.e., confirmed by Gram stain) require subculturing for purification. Insufficient growth indicates the need for subculturing or further incubation.

Pick isolated colonies or a sweep of confluent growth from the pure culture.

## PROCEDURE

**Materials Provided:** **BBL DrySlide PYR** and **BBL DrySlide PYR Color Developer**.

**Materials Required But Not Provided:** Inoculating loop (platinum or plastic, approximately 10  $\mu$ L), wooden applicator stick, pipettor and distilled or deionized water.

### Test Procedure:

1. Open the **BBL DrySlide PYR** pouch and remove the slide.

Do not place a used or partially used **BBL DrySlide** PYR (with organisms) back into the pouch.

- Deposit a loopful (approximately 10 µL) of distilled or deionized water onto a reaction area of **BBL DrySlide** PYR. Avoid adding excess water.
- Using an appropriate inoculation device, pick isolated colonies or a sweep of confluent growth from the culture to be tested.
- Generously smear the specimen onto the moistened reaction area of **BBL DrySlide** PYR.
- Incubate the slide at room temperature for 2 min.
- Holding the reagent dispenser upright and with the tip pointing away from the user, squeeze gently to crush the glass ampule inside the dispenser. Invert and squeeze slightly to dispense 1 drop of **BBL DrySlide** PYR Color Developer onto the inoculated area of the slide.
- Examine for appearance of a pink/fuchsia color within 1 min.

#### User Quality Control:

- Examine **BBL DrySlide** PYR.
- Before use, the dry reaction area should be white in color. After wetting, the reaction area should remain white. After dispensing **BBL DrySlide** PYR Color Developer, the reaction area becomes pale yellow.
- Check product performance by testing with positive and negative control cultures, such as those listed below.

Organism	ATCC™	PYR Reaction	Color Response
<i>Enterococcus faecalis</i>	29212	+	pink/fuchsia
<i>Streptococcus agalactiae</i>	13813	-	yellow

Quality control requirements must be performed in accordance with applicable local, state and/or federal regulations or accreditation requirements and your laboratory's standard Quality Control procedures. It is recommended that the user refer to pertinent NCCLS guidance and CLIA regulations for appropriate Quality Control practices.

#### RESULTS

PYR-positive organisms change the color of the reaction area from pale yellow to pink/fuchsia within 1 min of adding **BBL DrySlide** PYR Color Developer.

PYR-negative organisms produce no color change from yellow within 1 min.

#### LIMITATIONS OF THE PROCEDURE

If excess liquid is applied to the **BBL DrySlide** reaction area, the liquid will seep into adjacent windows.

The PYR test may be used as a presumptive diagnostic aid for identifying organisms that hydrolyze L-pyrrolidonyl-β-naphthylamide (PYR); i.e., group A streptococci and group D enterococci. However, additional biochemical and serological tests should be performed for complete identification.

Some streptococcal strains produce only a small amount of the enzyme pyrrolidonyl peptidase, so the **BBL DrySlide** PYR color change from yellow to pink is weak. A weak positive reaction can be enhanced by allowing the test organism to incubate 5 min (rather than 2 min) at room temperature before adding **BBL DrySlide** PYR Color Developer.

#### PERFORMANCE CHARACTERISTICS

One hundred seventy-four (174) gram-positive cocci (streptococci, enterococci and aerococci) were tested using **BBL DrySlide** PYR; 91 organisms were PYR-positive, 83 were PYR-negative. **BBL DrySlide** PYR demonstrated 99.4% accuracy.

Reactions obtained with **BBL DrySlide** PYR in a comparison study:

Test Species	Positive Reaction	Negative Reaction
<i>Aerococcus</i> sp.	11	
<i>Enterococcus avium</i>	3	
<i>Enterococcus casseliflavus</i>	2	
<i>Enterococcus durans</i>	7	
<i>Enterococcus faecalis</i>	30	
<i>Enterococcus faecium</i>	12	1
<i>Enterococcus gallinarum</i>	1	
<i>Enterococcus hirae</i>	2	
<i>Enterococcus malodoratus</i>	1	
<i>Enterococcus mundtii</i>	2	
<i>Streptococcus agalactiae</i>		7
<i>Streptococcus aginosus</i>		10
<i>Streptococcus bovis</i>		2
<i>Streptococcus bovis</i> I		13
<i>Streptococcus bovis</i> II		6
<i>Streptococcus</i> group A	7	
<i>Streptococcus</i> group C		4
<i>Streptococcus</i> group F		3
<i>Streptococcus</i> group G		4
<i>Streptococcus intermedius</i>		2
<i>Streptococcus mitis</i>		4
<i>Streptococcus mutans</i>		1
<i>Streptococcus pneumoniae</i>		6
<i>Streptococcus pyogenes</i>	13	1
<i>Streptococcus salivarius</i>		2
<i>Streptococcus sanguis</i>		5
<i>Streptococcus viridans</i> group		10
<i>Streptococcus zooepidermicus</i>		2
Total	91	83

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Benex Limited