

For Accurate, Rapid, Cost-Efficient Results...



Indispensable to human health

Go  with the
**BD BBL™ Staphyloslide™
Latex Test**

4 5 6

For Accurate, Rapid, Cost-Efficient Results.

The BD BBL™ Staphyloslide™ Latex Test brings you rapid, sensitive and accurate testing for the identification of *Staphylococcus aureus* colonies, with the ease-of-use and dependability you expect from BD!

BD BBL Staphyloslide Latex Test delivers:

- Complete, ready-to-go kit including prepared reagents, with disposable test cards.
- Convenient work station that holds all test components.
- 100% sensitivity and 99% specificity.¹
- Specificity to clumping factor and/or Protein A for a wide range of *S. aureus* detection.

Contact your BD representative for further information or to place a standing order for the BD BBL Staphyloslide Latex Test Kit. And look for the blue!

Easy 60-Second Procedure:

1 Dispense one drop of Test Latex onto a test circle.²

2 Dispense one drop of Control Latex onto another test circle.

3 Using a loop, smear and mix 5 colonies into the Test Reagent, covering the circle.

4 Proceed the same way for Control Latex.

5 Rock card and read for agglutination.

Product Description

Catalog No.

BBL™ Staphyloslide™ Latex Test Kit

Includes all needed reagents (Test Latex and Control Latex), disposable test cards and work station

100 Test Kit

240952

500 Test Kit

240953

BBL™ QualiSwab™ Quality Control Swabs

Staphylococcus aureus ATCC™ 25923

237052

Staphylococcus aureus ATCC™ 29213

237053

Staphylococcus aureus ATCC™ 49476

237150

Staphylococcus epidermidis ATCC™ 12228

237055

Staphylococcus epidermidis ATCC™ 49134

237151

BBL™ Gram Stain Kit

One 250 mL bottle each of Crystal Violet Solution, Iodine Solution (stabilized), Decolorizer Solution, Safranin Solution

212539

BBL™ Gram Stain Kit

One 250 mL bottle each of Crystal Violet Solution, Iodine Diluent with Iodine Concentrate, Isopropyl Decolorizer Solution, Safranin Solution

212524

Additional Gram Stain Kits and individual components are available.

¹ See package insert for complete data.

² Please consult package insert for complete procedural instructions.



BD Diagnostic Systems

7 Loveton Circle
Sparks, MD 21152-0999 USA
Tel: 800.638.8663
www.bd.com/clinical

ATCC is a trademark of the American Type Culture Collection. BD, BD Logo and all other trademarks are the property of Becton, Dickinson and Company. © 2002 BD.

1-2333 March 2002 Printed in USA