

# BD GasPak™ EZ Anaerobe Sachets

With Indicator for the GasPak EZ Anaerobe Pouch System



**AVAILABLE ONLY FROM BD!**

Environmental Generation QC  
Made Easier with On-board  
Anaerobic Indicator



Helping all people  
live healthy lives

# On-board anaerobic indicators

The latest innovation from the inventors of disposable gas generating systems.

## Easy as 1, 2, 3



**1** Place inoculated plates inside resealable pouch.



**2** Add one GasPak EZ Anaerobe Sachet with Indicator.



**3** Seal pouch by pressing zipper together and incubate for culture.

For your convenience, BD has attached a dry, anaerobic indicator directly to its BD GasPak™ EZ Anaerobe Sachet for use with the BD GasPak EZ Anaerobe Pouch System.

### Benefits

- Assured compliance with quality control procedures
- One less step increases lab efficiency and saves time
- 100% assurance that indicator is present, eliminating the need to re-culture or re-sample
- New, on-board methylene blue indicator tablet with distinct color reaction accurately monitors anaerobic conditions as recommended in the ASM *Clinical Microbiology Procedures Handbook*, 2<sup>nd</sup> ed. Section 4.5.2.
- No new catalog number—continue to order the BD GasPak EZ Anaerobe Pouch System as you always have

BD Diagnostics has been the market leader in disposable environmental generation products since it developed the first disposable environmental generation product, BBL™ GasPak™, in the 1960s. Since that time, there have been many additions and enhancements made to the BD GasPak family. As the product line has grown and improved it has become necessary to make changes to the BBL GasPak offering that will provide the best solutions in environmental generation to our customers.

Please be aware that as of October 1, 2005, BD Diagnostics will be phasing out the manufacture of the traditional BBL GasPak product line. The chart below identifies the BBL GasPak Pouch products that we will be discontinuing, as well as the replacement product numbers.

### Order Now

Contact your local BD sales representative or preferred distributor or call BD Customer Service at 1-800-675-0908.

Product to be discontinued effective 10/01/05	Product Replacement
260651 BBL™ GasPak™ Pouch, 25 pk	260683 BD GasPak™ EZ Anaerobe Pouch System, 20 pk
260652 BBL™ GasPak™ Pouch, Sealing bars, 10 pk	Not required for BD GasPak™ EZ products
260656 BBL™ CampyPouch™, 25 pk	260685 BD GasPak™ EZ CampyPouch System, 20 pk
260662 BBL™ GasPak™ CO <sub>2</sub> Pouch, 50 pk	260684 BD GasPak™ EZ CO <sub>2</sub> Pouch System, 20 pk



**BD Diagnostics**  
7 Loveton Circle  
Sparks, MD 21152-0999  
800.638.8663  
www.bd.com/ds