

# BD Insyte™ Autoguard™


Shielded IV Catheter

## POINTS TO PRACTICE

1

### PREPARATION


- Prior to venipuncture, hold catheter hub and **rotate catheter hub 360 degrees**
- Make sure catheter is seated back in the notch

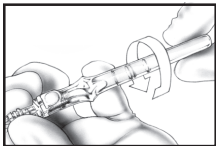

2

### VENIPUNCTURE

- Approach vein slowly at a low angle
- Observe flashback (along catheter in 20, 22, 24 gauge only)
- Upon flashback visualization, lower catheter almost parallel to the skin
- Advance ENTIRE UNIT slightly before threading catheter
- Thread catheter into vein while maintaining skin traction

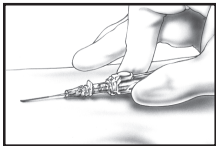


3

### NEEDLE RETRACTION

*Before Pressing The Button:*

- Release tourniquet
- Apply digital pressure beyond the catheter tip
- Stabilize catheter hub
- Press the white button

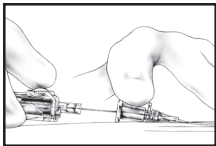

## **CAUTION REMINDERS**

- **Do Not** bend the needle prior to insertion.
- **Do Not** withdraw needle from catheter hub before pressing the white button.
- **Never Reinsert Needle** into the catheter as this could shear the catheter.
- **Do Not Use Scissors** at or near the insertion site.
- Needle should be retracted prior to disposal.
- Place all needles in a puncture-resistant, leak-proof sharps container.
- Report all needlestick injuries and follow established protocol.
- Aseptic technique and proper skin preparation of the site are essential.

Refer to package insert for complete instructions for use.

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. ©2004 BD.



**BD Medical**  
Sandy, Utah 84070  
[www.bd.com/infusion](http://www.bd.com/infusion)

D12968-2 F(3-05)