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
Revisions

BALTSO0191 Version 10.0 Template 4
Inserts

Rev from	Rev to	JOB #
02	03	8870-16

NOTES:

- BD Catalog Number: 441444
- Blank (Sheet) Size: Length: 11" Width: 8.5"
- Number of Pages: 12 Number of Sheets: 3
- Page Size: Length: 8.5" Width: 5.5" Final Folded Size: 8.5" x 5.5"
- Ink Colors: No. of Colors: 2 PMS#: 032 Red, Standard Black
- Printed two sides: Yes ☒ No ☐
- Style (see illustrations below): # 5

- Vendor Printed ☒ Online/In House Printed ☐ Web ☐
- See specification control BALTSO0191 for material information.
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	REVISED BY By Sonia Thompson at 11:34 am, Dec 06, 2016	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	 BD Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA
Proofer	PROOFING APPROVED BY By Daria Kuznetsova at 7:57 am, Dec 08, 2016		
Checked By	THIRD EYE BY By Natalie Morio at 8:35 am, Dec 09, 2016		
Part Number:	8083954	Category and Description Package Insert, BD ProbeTec™ LBC Dilution Tubes	Sheet: 1 of 13 Scale: N/A A

BD Liquid-Based Cytology Specimen (LBC) Dilution Tubes for the BD ProbeTec™ Q^x Amplified DNA Assays

English: pages 1 – 2
Français : pages 2 – 4

Deutsch: Seiten 4 – 6
Español: páginas 6 – 7

8083954(03)
2016-11

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne místní zástupce společnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyts lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőtől. / Нұсқаулар үшін жеріңіңің BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielem BD. / Contacte o reprezentante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Inštrukcie ziskate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD.

INTENDED USE

The Liquid-Based Cytology Specimen (LBC) Dilution Tubes for the **BD ProbeTec Q^x Amplified DNA Assays** are designed to allow for detection of *Chlamydia trachomatis* and *Neisseria gonorrhoeae* in gynecological specimens that are collected in **BD SurePath™** Preservative Fluid or PreservCyt® Solution and tested with the **BD ProbeTec CT Q^x** and **GC Q^x Amplified DNA Assays** on the **BD Viper™** System in Extracted Mode using an aliquot that is removed prior to processing for either the **BD SurePath** or ThinPrep® Pap test.

REAGENTS

Materials Provided: Each Liquid-Based Cytology Specimen (LBC) Dilution Tube for the **BD ProbeTec Q^x Amplified DNA Assays** contains approximately 1.7 mL of a Tris/Sodium Chloride solution and preservative.

Materials Required But Not Provided: Liquid-Based Cytology Specimen (LBC) Dilution Tube Caps for the **BD ProbeTec Q^x Amplified DNA Assays**, **BD Viper** LBC Specimen Rack, displacement pipettes, polypropylene aerosol-resistant tips capable of delivering 0.5 ± 0.05 mL, and a vortex mixer.

Storage Instructions: Store kit at 2–33 °C. Do not use past the expiration date.

Warnings and Precautions:

1. For *in vitro* Diagnostic Use.
2. For **BD SurePath** or PreservCyt specimens, use only the Liquid-Based Cytology Specimen (LBC) Dilution Tube for the **BD ProbeTec Q^x Amplified DNA Assays**.
3. LBC solutions contain flammable substances. Do not place specimens that have been transferred to the LBC Specimen Dilution Tubes in the **BD Viper** Lysing Rack or the Lysing Heater. Specimens transferred to the LBC Specimen Dilution Tubes should be placed in the **BD Viper** LBC Specimen Rack.
4. For testing with the **BD ProbeTec Q^x Amplified DNA Assays** on the **BD Viper** System in Extracted Mode, be sure to obtain aliquots of specimens collected in **BD SurePath** Preservative Fluid or PreservCyt Solution, prior to processing for either the **BD SurePath** or ThinPrep Pap test. Failure to do so may result in erroneous results.
5. Do not store specimens in LBC Specimen Dilution Tubes beyond the product expiration date.

Warning

H315 Causes skin irritation. **H319** Causes serious eye irritation. **H335** May cause respiratory irritation.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray. **P264** Wash thoroughly after handling. **P271** Use only outdoors or in a well-ventilated area. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P302+P352** IF ON SKIN: Wash with plenty of soap and water. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P312** Call a POISON CENTER or doctor/physician if you feel unwell. **P321** Specific treatment (see on this label). **P332+P313** If skin irritation occurs: Get medical advice/attention. **P337+P313** If eye irritation persists: Get medical advice/attention. **P362+P364** Take off contaminated clothing and wash it before reuse. **P403+P233** Store in a well-ventilated place. Keep container tightly closed. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

SPECIMEN COLLECTION, TRANSFER, STORAGE AND TRANSPORT

LBC Specimen Collection, Storage and Transport

BD SurePath or PreservCyt specimens must be collected using either an endocervical broom or a brush/spatula combination as described in the **BD SurePath** Preservative Fluid or PreservCyt Solution product insert. LBC specimens in their original vials can be stored and/or transported for up to 30 days at 2–30 °C from the date of collection.

Specimen Transfer to LBC Specimen Dilution Tubes

A 0.5 mL aliquot of the **BD SurePath** or PreservCyt specimen must be transferred from the original vial to the LBC Specimen Dilution Tube prior to processing for either the **BD SurePath** or ThinPrep Pap test.

Wear gloves when handling the LBC Specimen Dilution Tube and the **BD SurePath** or PreservCyt specimen vial. If gloves come in contact with the specimen, immediately change them to prevent contamination of other specimens.

BD SurePath Specimen Transfer

NOTE: Refer to the **BD PrepStain™** System product insert for instructions on removing an aliquot from the **BD SurePath** specimen vial prior to performing the **BD SurePath** liquid-based Pap test.

1. Label an LBC Specimen Dilution Tube with patient identification information.
2. Remove the cap from the LBC Specimen Dilution Tube.
3. Transfer 0.5 mL from the specimen vial to the LBC Specimen Dilution Tube. Avoid pipetting fluid from the bottom of the vial. Discard pipette tip. **NOTE:** A separate pipette tip must be used for each specimen.
4. Tighten the cap on the LBC Specimen Dilution Tube securely.
5. Invert the LBC Specimen Dilution Tube 3–4 times to ensure that the specimen and diluent are well mixed.

PreservCyt Specimen Transfer

NOTE: Refer to the ThinPrep 2000/3000 System Operator's Manual Addendum for instructions on removing an aliquot from the PreservCyt sample vial prior to performing the ThinPrep Pap test.

1. Label an LBC Specimen Dilution Tube with patient identification information.
2. Remove the cap from the LBC Specimen Dilution Tube.
3. Transfer 0.5 mL from the specimen vial to the LBC Specimen Dilution Tube. Avoid pipetting fluid from the bottom of the vial. Discard pipette tip. **NOTE:** A separate pipette tip must be used for each specimen.
4. Tighten the cap securely on the LBC Specimen Dilution Tube.
5. Invert the LBC Specimen Dilution Tube 3–4 times to ensure that the specimen and diluent are well mixed.

LBC Specimen Storage and Transport in the LBC Specimen Dilution Tube

Store the **BD SurePath** or PreservCyt specimens that have been transferred to the LBC Specimen Dilution Tubes at 2–30 °C for up to 30 days. Specimens may also be stored at -20 °C for up to 90 days. Specimen storage should not exceed the expiration date of the LBC Specimen Dilution Tube.

For domestic and international shipments, specimens should be packaged and labeled in compliance with applicable state, federal, and international regulations covering the transport of clinical specimens and etiologic agents/infectious substances. Recommended time and temperature conditions for storage must be maintained during transport.

AVAILABILITY

Cat. No. Description

441444	Liquid Based Cytology Specimen (LBC) Dilution Tubes for the BD ProbeTec™ Qx Amplified DNA Assays, 400
441443	Liquid Based Cytology Specimen (LBC) Dilution Tube Caps for the BD ProbeTec™ Qx Amplified DNA Assays, 400

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

BD Liquid-Based Cytology Specimen (LBC)

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PreservCyt and ThinPrep are registered trademarks of Hologic, Inc.
© 2016 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.
Benex Limited