 BD ProbeTec ET *Chlamydia trachomatis*/Neisseria gonorrhoeae (CT/GC) Amplified DNA Assay Endocervical Specimen Collection and DRY TRANSPORT Kit

L008315(05)

2019-12

Norsk

Europa: Medisinsk utstyr

93/42/EEC

BRUKSOMRÅDE

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REFERANSER

1. Clinical and Laboratory Standards Institute. 2005. Approved Guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, PA.
2. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53–80.
3. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C.
4. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021–0045.

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For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiiri / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrense / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Минимальна температура / 温度下限

Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Contollo / Бақылау / 컨트롤 / Kontrolé / Kontrole / Controle / Controlo / Контроль / kontroll / Контроль / 对照

Positive control / Положительный контроль / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitívna kontrola / Pozitív kontroll / Contollo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívá kontrola / Positieve controle / Kontrola dodatnia / Controllo positivo / Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивный контроль / 阳性对照试剂

Negative control / Отрицательный контроль / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negatívna kontrola / Negatív kontroll / Contollo negativo / Негативтік бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívá kontrola / Negatieve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативный контроль / 阴性对照试剂

Method of sterilization: ethylene oxide / Метод на стерилизация: этиленов оксид / Způsob sterilizace: ethylenoxid / Steriliseringmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseerimismeetod: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Стерилизация әдісі – этилен тотығы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksisds / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringmetode: etylenoksid / Metoda sterylizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Metod sterilizacijos: etilenoksid / Metoda sterilizácie: etylenoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: етиленоксидом / 灭菌方法: 环氧乙烷

Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringmetode: bestråling / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseringmetode: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metodo di sterilizzazione: irradiazione / Стерилизация әдісі – сәулелі түрде / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringmetode: bestråling / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизации: облучение / Metoda sterilizácie: ožiarovanie / Metoda sterilizacije: ozračevanje / Steriliseringsmetod: strålning / Sterilizasyon yöntemi: ırdıyasyon / Метод стерилизації: опроміненням / 灭菌方法: 辐射

Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologilised riskid / Risques biologiques / Biološki rizik / Biológiallag veszélyes / Rischio biologico / Биологиялык төуекелдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyolojik Riskler / Біологічна небезпека / 生物学风险

Caution, consult accompanying documents / Внимание, направте справка в придружаващите документи / Pozor! Prostudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitedokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeda kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Urozorenje, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dėmesio, žiūrėkite pridėdamus dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Vůstřaha, pozří spřievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацію / 小心, 请参阅附带文档。

Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiiri / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrense / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sicaklık üst sınırı / Максимальна температура / 温度上限


Keep dry / Пазете сухо / Skladujte v suchém prostředí / Opbevares tørt / Trockklagen / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suho / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laikykite sausu / Uzglabāt sausu / Droog houden / Hoides tørt / Se zachowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať v suchu / Držite na suvom mestu / Förvaras tørt / Kuru bir şekilde muhafaza edin / Бергти від вологи / 请保持干燥

Collection time / Време на събиране / Cas odběru / Orsamlingstidspunkt / Entnahmehurzeit / Ωρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жинау уакыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colheita / Ora colectării / Время сбора / Doba odberu / Vreme prikupljanja / Uppsamlingstid / Toplama zamanı / Час забору / 采集时间

Peel / Обелете / Otefete zde / Abn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Ўстиғи қабатын алып таста / 벗기 / Plésti čia / Atfimt / Schillen / Trekk av / Oderwać / Destacar / Se dezlipeste / Отклеить / Odtrhnite / Oljuštiti / Dra isår / Ayırma / Відкнеїти / 撕下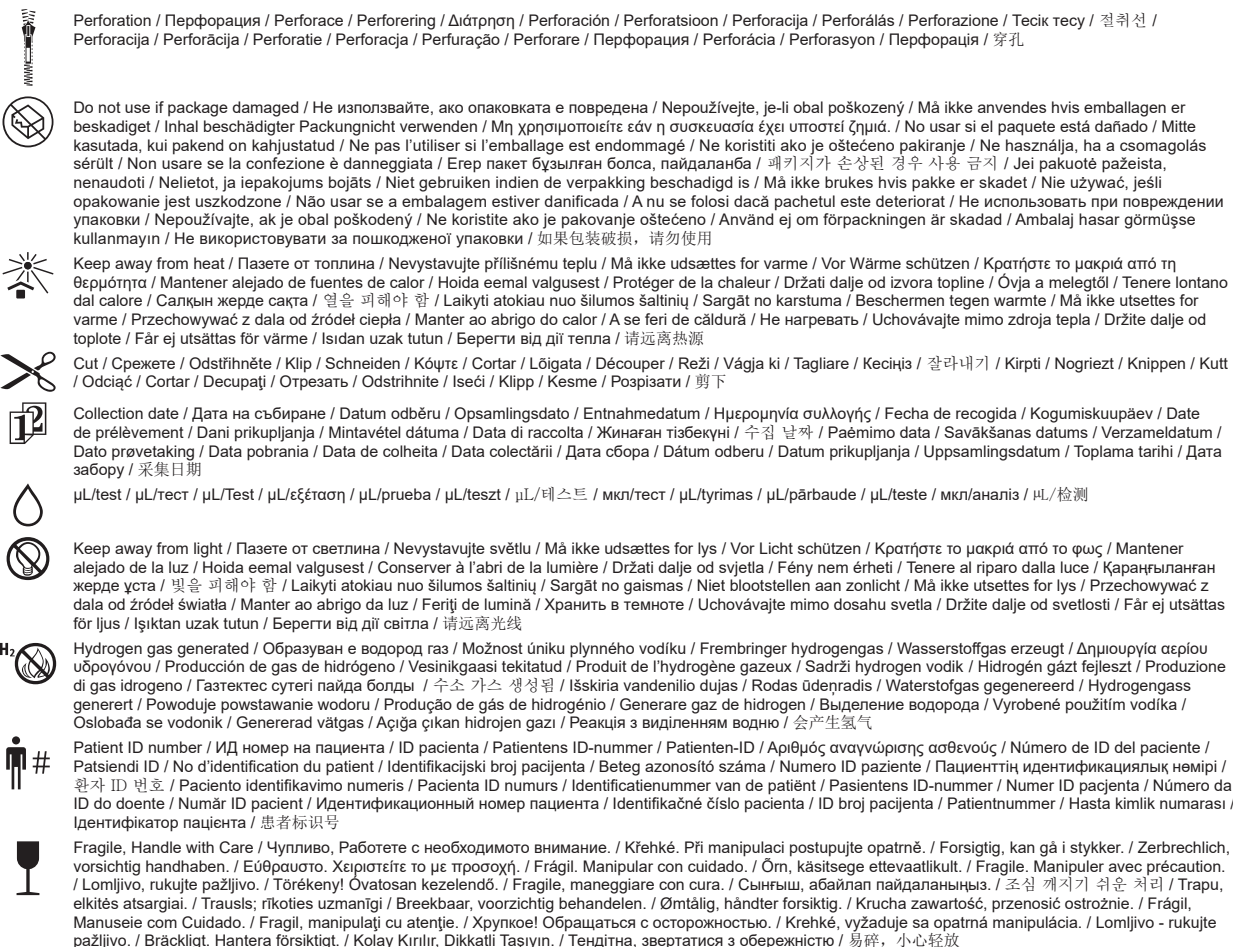

 Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Becton Dickinson France S.A.S.
11 rue Aristide Bergès
38800 Le Pont de Claix, France

Australian Sponsor:

Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

BD, the BD Logo and ProbeTec are trademarks of Becton, Dickinson and Company or its affiliates. © 2019 BD. All rights reserved.