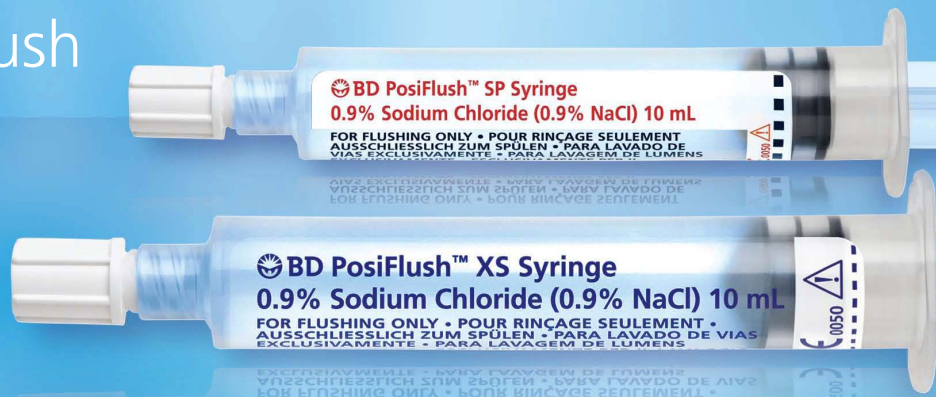


BD PosiFlush™ (NaCl 0.9%)

Pre-filled Flush Syringe

(Refer to Instructions
For Use for more
information)



- Wearing sterile gloves during procedure?
- Working within a sterile field?
- Treating a patient who is immunosuppressed?

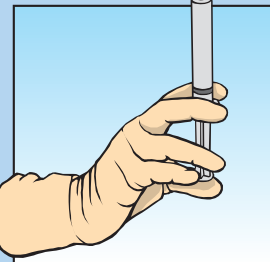
use BD PosiFlush™ XS

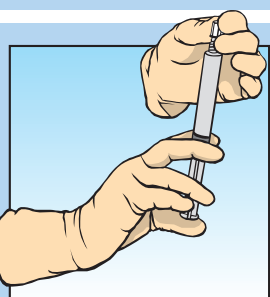
For all other IV flush procedures,
use BD PosiFlush™ SP

1. Check patient's identity

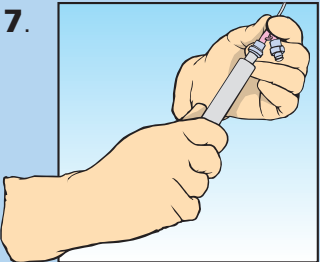
2. Check packaging is intact then open

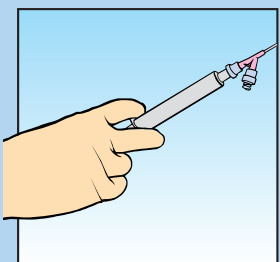
3. Check syringe contents and verify expiry date

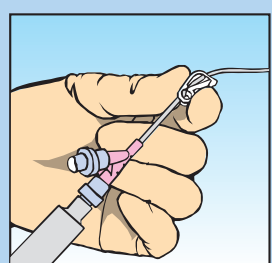
4.  Click to unstick
Gently depress the
plunger until you
feel/hear a click.
This will free the
plunger.

5.  Unscrew tip cap to
expel air

6. Clean catheter access port according to
Trust policy

7.  Remove cap and
attach syringe to the
catheter

8.  Use push/pause
technique to
administer, which
creates turbulent
flow

9.  Finish flush using
positive pressure.
If using an extension
with a clamp, clamp
line while thumb
remains on syringe
plunger

10. Dispose into clinical waste or sharps bin
according to Trust policy

**NOT FOR RECONSTITUTION OF DRUGS
OR PATIENT REHYDRATION**