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REFERANSLAR

1. Harrington, B.J., and G.J. Hageage, Jr. 1991. Calcofluor white: tips for improving its use. Clin. Microbiol. Newsl. 13:3-5.
2. Monheit, J.E., D.F. Cowan, and D.G. Moore. 1984. Rapid detection of fungi in tissues using calcofluor white and fluorescence microscopy. Arch. Pathol. Lab. Med. 108:616-618.
3. Stringer, J.R., C.B. Beard, R.F. Miller, and A.E. Wakefield. 2002. A new name (*Pneumocystis jirovecii*) for *Pneumocystis* from humans. Emerg. Infect. Dis. 8(9):891-6.
4. Baselski, V.S., M.K. Robinson, L.W. Pifer, and D.R. Woods. 1990. Rapid detection of *Pneumocystis carinii* in bronchoalveolar lavage samples by using Cellufluor staining. J. Clin. Microbiol. 28:393-394.
5. Kim, Y.K., S. Parulekar, P.K.W. Yu, R.J. Pisani, T.F. Smith, and J.P. Anhalt. 1990. Evaluation of calcofluor white stain for detection of *Pneumocystis carinii*. Diagn. Microbiol. Infect. Dis. 13:307-310.
6. Hageage, G.J., and B.J. Harrington. 1984. Use of calcofluor white in clinical mycology. Lab. Med. 15:109-112.
7. National Committee for Clinical Laboratory Standards. 2001. Approved Guideline M29-A2. Protection of laboratory workers from occupationally acquired infections, 2nd ed. NCCLS, Wayne, Pa.
8. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53-80.
9. U.S. Department of Health and Human Services. 1999. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 4th ed. U.S. Government Printing Office, Washington, D.C.
10. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EECP). Official Journal L262, 17/10/2000, p. 0021-0045.

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For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiiri / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrense / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperatura / Nedre temperaturgräns / Sıcaklık alt sınırı / Мінімальна температура

Control / Контрольно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / Kontrolé / Kontrolle / Controle / Controllo / Контроль / kontroll / Контроль

Positive control / Положителен контрол / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Pozitiv kontroll / Controllo positivo / Оң бақылау / Teigiamą kontrolę / Pozitívá kontrolé / Positieve controle / Kontrola dodatnia / Controllo positivo / Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивный контроль

Negative control / Отрицателен контрол / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negativna kontrola / Negativ kontroll / Controllo negativo / Негативна бақылау / Neigiamą kontrolę / Negatívá kontrolé / Negatieve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативный контроль

Method of sterilization: ethylene oxide / Метод на стерилизация: этиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringsmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseerimismetode: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Стерилизация әдісі – этилен тотығы / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksiids / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringsmetode: etylenoksid / Metoda sterylizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metóda sterilizácie: etylénoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetode: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом

Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringsmetode: bestrålning / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseermisete: kiirgus / Méthode de stérilisation: irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metodo di sterilizzazione: irradiazione / Стерилизация әдісі – сәулелі типі / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseer met behulp van bestraling / Steriliseringsmetode: bestrålning / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизации: облучение / Metóda sterilizácie: ožiarenie / Metoda sterilizacije: ozračavanje / Steriliseringsmetod: strålning / Sterilizasyon yöntemi: ıradayasyon / Метод стерилизації: опроміненням

Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologised risk / Risques biologiques / Biološki rizik / Biológiallag veszélyes / Rischio biologico / Биологиялык төүөкөлдөр / Biologinis pavojus / Bioloģiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologicke riziko / Biološki rizici / Biologisk risk / Biyojlojik Riskler / Біологічна небезпека

Caution, consult accompanying documents / Внимание, направте справка в придружаващите документи / Pozor! Pristudjite si priloženou dokumentacii! / Forsigtig, se ledsagende dokumenter / Achtung, Begleiddokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaantus! Lugeda kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Upozorenje, koristite prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / Dmesio, žiúrekite pridedamus dokumentus / Piesardziba, skatit pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atentie, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere bagvurun / Увага: див. супутню документацию

Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ūleminė temperatūriųrii / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurilimiet / Øvre temperaturgrense / Górná granica temperature / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sıcaklık üst sınırı / Максимальна температура

Keep dry / Пазете сухо / Skladujte v suchém prostředí / Opbevaras tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Százás helyen tartandó / Tenere all'asciutto / Құрақ күйінде ұста / Laikykite sausiai / Uzglabāt sausu / Droog houden / Hoides tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať v suchu / Držite na suvom mestu / Förvaras torrt / Kuru bir şekilde muhafaza edin / Беретти від вологоти

Collection time / Време на събиране / Čas odběru / Opsamlingstidspunkt /
Επληµήρευση / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de
prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жинау уақыты /
Paémimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania
/ Hora de colheita / Ora colectării / Время сбора / Doba odboru / Vreme prikupljanja /
Uppsamlingsstid / Toplama zamani / Час забору

Peel / Обелете / Otevretre zde / Åbn / Abziehen / Αποκολλήστε / Desprendre / Koorida /
Décoller / Otvoriti skin / Húzza le / Staccare / Ҷстиги қабатын алып таста / Plēsti čia /
Atīmēt / Schillen / Trekkv av / Oderwač / Destacar / Se dezilpeşte / Отклеить / Odrhните /
Oljuštiti / Dra isår / Ayırma / Bıdkaletti

Perforation / Перфорация / Perforace / Perforering / Διήτρηση / Perforación /
Perforatsioon / Perforacija / Perforálás / Perforazione / Tecik tescu / Perforacija /
Perforacja / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforácia /
Perforasyon / Перфорация

Do not use if package damaged / Не използвайте, ако опаковката е повредена /
 Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget /
 Inhal beschädigter Packung nicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία
 έχει υποστεί ζημιά. / No usar si el paquete está dañado / Mitte kasutada, kui pakend
 on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je
 oštećeno pakiranje / Ne használnia, ha a csomagolás sérült / Non usare se la confezione
 è danneggiata / Ереп пакет бүзылган болса, пайдаланба / Jei pakuotė pažeista,
 nenaudoti / Nelietoti, ja iepakojums bojāts / Niet gebruiken indien de verpakking
 beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest
 uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul
 este deteriorat / Не использовать при повреждении упаковки / Nepoužívať, ak je obal
 poškodený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är
 skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої
 упаковки

Keep away from heat / Пазете от топлина / Nevystavujte přílišnému teplu / Må ikke utsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida etmaal valgestu / Protégez de la chaleur / Držati dalje od izvora topline / Óvja a melegtől / Tenere lontano dal calore / Салқын жерде сақта / Laikyti atokiau nuo šilumos šaltinių / Sargat' no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / Не нагревать / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Isidan uzak tutun / Берегти від дії тепла

Cut / Срежете / Odstrihnēte / Klip / Schneiden / Κόψτε / Cortar / Lõigata / Découper /
Reži / Vágja ki / Tagliare / Keciңіз / Kirpti / Nogriezt / Knippen / Kutt / Odciać / Cortar /
Decupați / Отрезать / Odstrihnite / Iseći / Klipp / Kesme / Розрізати

Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuupäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаған тізбекүні / Paémimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colheita / Data colectării / Дата сбора / Datum odberu / Datum prikupljanja / Uppsamlingsdatum / Toploma tarihi / Дата забору

µL/test / µL/тест / µL/Test / µL/εξέταση / µL/prueba / µL/teszt / мкЛ/тест / µL/tyrimas / µL/pārbaude / µL/teste / мкл/анализ

Keep away from light / Пазете от светлина / Nevystavujite svētlū / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svetla / Fény nem érheti / Tenere al riparo dalla luce / Қазақылған жерде ұста / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródła światła / Manter ao abrigo da luz / Feriți de lumină / Хранить в темноте / Uchovávajte mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Išikta užak tutun / Беретти від дії світла

Hydrogen gas generated / Образуван е водород газ / Možnost úniku plynného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadržaj hydrogen vodik / Hidrogén gázt fejleszt / Produzione di gas idrogeno / Газтекес сутери пайда болды / Išskiria vandenilio dujas / Rodas vandenilio / Waterstofgas gegenereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené použitím vodíka / Oslobađa se vodonik / Genererad vätegas / Açığa çıkan hidrojen gazı / Реакция з виділенням водню

Patient ID number / ИД номер на пациента / ID pacienta / Patientens ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsiendi ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттің идентификациялық нөмірі / Paciento identifikavimo numeris / Pacienta ID numurs / Identificatienummer van de patiënt / Pasientens ID-nummer / Numer ID pacienta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası / Ідентифікатор пацієнта

Fragile, Handle with Care / Чупливо, Роботете с необходимото внимание. / Křehké. Při manipulaci postupujte opatrně. / Forsigtig, kan gå i stykker. / Zerbrechlich, vorsichtig handhaben. / Εύθραστο. Χειριστείτε το με προσοχή. / Frágil. Manipular con cuidado. / Örn, käsitlese ettevaatlikult. / Fragile. Manipuler avec précaution. / Lomljivo, rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. / Сынғыш, абайлап пайдаланыңыз. / 조심 깨지기 쉬운 처리 / Trapu, elkitės atsargiai. / Trausls; rūkoti uzmanīgi / Breekbaar, voorzichtig behandelen. / Ømtålig, håndter forsiktig. / Krucha zawartość, przenosić ostrożnie. / Frágil, Manuseie com Cuidado. / Frágil, manipulati cu atenție. / Хрупкое! Обращаться с осторожностью. / Krehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşıyın. / Тендітна, звертатися з обережністю / 易碎，小心轻放

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Australian Sponsor:
Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland