Part Number:	L001118	BALTSO0191 Version 13.0 Template 4 Inserts	
Category and Description: Package Insert, BD BBL Dobell & O'Connor Iodine Stain Droppers	Rev from: 02	Rev to: 03	
	Job Number:		760-18

Catalog Number: 261189

Blank (Sheet) Size: Length: 11" Width: 18"

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
Number of Pages: 8 Number of Sheets: 1

Page Size: Length: 11" Width: 4.5" Final Folded Size: 4 1/2" X 1 7/8"

Ink Colors: Number of Colors: 1 PMS #: 2755 Blue

Printed Two Sides: Yes: ☒ No: ☐

Style (see illustrations below): # 4

Vendor Printed: ☒ Online / In House Printed: ☐ Web Printed: ☐

See Specification control no. NA for material information.

BD Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Company confidential. This document is the property of Becton, Dickinson and Company and is not to be used outside the company without written permission. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Revised By: **REVISED BY**
By Maria Zacharias at 10:06 am, Sep 25, 2018

Proofing Approved By: **PROOFING APPROVED BY**
By Nichole Graham at 3:56 pm, Sep 25, 2018

Third Eye By: **THIRD EYE BY**
By Natalie Morio at 8:47 am, Sep 26, 2018

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INTENDED USE

BD BBL™ Dobell & O'Connor Iodine Stain Droppers have a variety of uses as a microbiological stain. Among them are staining the trophozoite, cyst, and egg stages of intestinal parasites.¹⁻³

SUMMARY AND EXPLANATION

Use of BD BBL Dobell & O'Connor Iodine Stain Droppers aid the visualization of the components of specimens observed microscopically. The selection of the appropriate technique depends upon the specimen.²⁻⁴

PRINCIPLES OF THE PROCEDURE

The addition of the BD BBL Dobell & O'Connor Stain to the specimen will iodinate proteins and carbohydrates.

Reagents

BD BBL Dobell & O'Connor Iodine Stain Droppers contain 0.5 mL of solution consisting of iodine 1%, potassium iodide 2%, and water.

Warnings and Precautions:

For *in vitro* Diagnostic Use.

Follow established laboratory procedures in handling and disposing of infectious materials.

Storage Instructions: Store at controlled room temperature 15–30 °C (59–86 °F). Protect from light.

Product Deterioration: Reagents are hermetically sealed in an ampule, which affords protection of the solution from chemical instability until expiration date. Stain droppers need no refrigeration.

PROCEDURE

Materials Provided: BD BBL Dobell and O'Connor Iodine Stain Droppers.

Material Required But Not Provided: Ancillary reagents and laboratory equipment as required for this procedure.

Test Procedure

1. Hold dropper upright and **POINT TIP AWAY FROM YOURSELF**. Grasp the middle with thumb and forefinger and squeeze gently to break inner ampule. **Caution: Break ampule close to its center one time only. Do not manipulate dropper any further as the plastic may puncture and injury may occur.** Tap bottom on tabletop a few times.
2. Invert for convenient drop-by-drop dispensing of solution.
3. Prepare a wet mount by commonly accepted laboratory technique as described in the references below.

User Quality Control

Direct microscopic examination of a clinical specimen will demonstrate an increase of color and contrast of intestinal parasites subjected to BD BBL Dobell & O'Connor iodine stain.

Quality control requirements must be performed in accordance with applicable local, state and/or federal regulations or accreditation requirements and your laboratory's standard Quality Control procedures. It is recommended that the user refer to pertinent CLSI guidance and CLIA regulations for appropriate Quality Control practices.

RESULTS

In a properly stained cyst, the glycogen appears reddish brown, the cytoplasm is yellow and the chromatin stains brown or black.

LIMITATIONS OF THE PROCEDURE

BD BBL Dobell & O'Connor Iodine Stain Droppers should only be used in procedures that require 3% iodine/potassium iodide.

Protect from light.

For accurate and reliable identification, specimens must be properly collected and handled before staining procedures. Ideally, fecal specimens should reach the laboratory within 1–2 h after passage; other materials (urine, duodenal material, aspirations from lesions) should be sent to the laboratory immediately after collection. If transportation is delayed, specimens should be preserved to maintain the diagnostic characteristics of organisms that might be present.

PERFORMANCE CHARACTERISTICS

Prior to release, all lots of BD BBL Dobell and O'Connor Iodine Stain Droppers are tested to verify specific product characteristics. The percentage of iodine in BD BBL Dobell and O'Connor Iodine Stain is determined by titration. The concentration of iodine is 0.95 to 1.10%. The percentage of potassium iodide in BD BBL Dobell and O'Connor Iodine Stain is also determined by titration. The concentration of potassium iodide is 1.8 to 2.2%.

Availability

Cat. No. Description

261189 BD BBL™ Dobell & O'Connor Iodine Stain Droppers, 50.

REFERENCES

1. Dobell, C., and F.W. O'Connor. 1921. Intestinal protozoa of man. William Wood, New York.

2. Melvin, D.M., and M.M. Brooke. 1974. Laboratory procedures for the diagnosis of intestinal parasites. U.S. Department of Health and Human Services publication No. (CDC) 80-8282 (Rev. Dec. 1974), p. 84-85. Center for Disease Control, Atlanta.

3. Lennette, E.H., A. Balows, W.J. Hausler, Jr., and H.J. Shadomy (ed.). 1985. Manual of clinical microbiology, 4th ed. American Society for Microbiology, Washington, D.C.

4. Finegold, S.M., and W.J. Martin. 1982. Bailey and Scott's diagnostic microbiology, 6th ed. The C.V. Mosby Co., St. Louis, Mo.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

 BD BBL Dobell & O'Connor Iodine Stain Droppers

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Benex Limited
Becton Dickinson Pty Ltd.