BD BBL AFB Slide

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REFERANSLAR

1. Sewell, D.L. 1994. Laboratory records, p. 13.2.1-13.2.35. *In* H.D. Isenberg (ed.), Clinical microbiology procedures handbook, vol. 2. American Society for Microbiology, Washington, D.C.
2. August, M.J., J.A. Hindler, T.W. Huber, and D.L. Sewell. 1990. Cumitech 3A, Quality control and quality assurance practices in clinical microbiology. Coordinating ed., A.S. Weissfeld. American Society for Microbiology, Washington, D.C.
3. Commission on Laboratory Accreditation. 1990. Inspection checklist. Diagnostic immunology and syphilis serology. College of American Pathologists, Northfield, Ill.
4. Miller, J.M. 1987. Quality control in microbiology. Centers for Disease Control, Atlanta.
5. Miller, J.M., and B.B. Wentworth (ed.). 1985. Methods for quality control in diagnostic microbiology. American Public Health Association, Washington, D.C.
6. National Committee for Clinical Laboratory Standards. 1996. Approved guideline: GP2-A3. Clinical laboratory technical procedure manuals, 3rd ed. National Committee for Clinical Laboratory Standards, Wayne, Pa.
7. Weissfeld, A.S., and R.C. Bartlett. 1987. Quality control, p. 35-65. *In* B.J. Howard, J. Klass II, S.J. Rubin, A.S. Weissfeld, and R.C. Tilton (ed.), Clinical and pathogenic microbiology. The C.V. Mosby Co., St. Louis.
8. Health Care Financing Administration. 1988. Medicare, Medicaid, and CLIA programs; revision of the clinical laboratory regulations for Medicare, Medicaid, and Clinical Laboratories Improvement Act of 1967 programs. Fed. Regist. 53: 29590-29632.
9. Chapin, K. 1995 Clinical microscopy, pp. 40-46. *In* Murray, P. R., Baron, E. J., Pfaller, M.A., Tenover, F.C. and R. H. Tenover (ed). Manual of clinical microbiology, 6th ed. American Society for Microbiology, Washington, D.C.
10. Vet. Adm. Arm. Forces Coop. Study on the Chemo. of Tuber. Nov. 1962. VA Dept. of Med. & Sur., Wash., D.C.
11. Strong, B.E., and G.P. Kubica. Isolation and identification of *Mycobacterium tuberculosis*, a guide for the Level II Laboratory, U.S. Public Health service Publication. 81-8390, Center for Disease Control, Atlanta.
12. Lab. Methods Clin, and Publ. Hlth. Mycobact., USPHS. 1968

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Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Benex Limited
Pottery Road, Dun Loaghaire
Co. Dublin, Ireland

Australian Sponsor

Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

ATCC is a trademark of the American Type Culture Collection.

BD, BD Logo and BBL are trademarks of Becton, Dickinson and Company. ©2014 BD