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1. DIN 58943-3: Diagnosis of tuberculosis – Part 3: Detection of mycobacteria by culture methods. Beuth-Verlag, Berlin 1996.
2. Garrod, L.P., F. O'Grady. 1971. Antibiotics and chemotherapy, 3rd ed. The Williams and Wilkins Company, Baltimore.
3. Korzybski, T., Z. Rowsyk-Gindifer, and W. Kurylowicz. 1978. Antibiotics – origin, nature and properties, vol. II. American Society for Microbiology, Washington, D.C.
4. Hitchings, G.H. 1974. Mechanisms of action of trimethoprim-sulfamethoxazole, p. 1–4 *In* Trimethoprim-sulfamethoxazole-I. Microbiological, pharmacological and clinical considerations. The University of Chicago Press, Chicago.
5. National Committee for Clinical Laboratory Standards. 2001. Approved Guideline M29-A2. Protection of laboratory workers from occupationally acquired infections, 2nd ed. NCCLS, Wayne, Pa.
6. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53–80.
7. U.S. Department of Health and Human Services. 1999. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 4th ed. U.S. Government Printing Office, Washington, D.C.
8. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021-0045.
9. Murray, P.R., E.J. Baron, M.A. Pfaller, F.C. Tenover, and R.H. Tenover (ed.). 1999. Manual of clinical microbiology, 7th ed. American Society for Microbiology, Washington, D.C.
10. Forbes, B.A., D.F. Sahm, and A.S. Weissfeld. 1998. Bailey & Scott's diagnostic microbiology, 10th ed. Mosby, Inc., St. Louis.
11. Kent, P.T., and G.P. Kubica. 1985. Public health mycobacteriology: a guide for the level III laboratory. USDHHS. Centers for Disease Control, Atlanta.
12. Carnoch, P.I., R.K. Enns, M.A. Soubolle, and R.J. Wallace, Jr. 1994. Cumitech 16A, Laboratory diagnosis of the mycobacterioses. Coordinating ed., A.S. Weissfeld. American Society for Microbiology, Washington, D.C.
13. Metchock, B.G., F.S. Nolte, and R.J. Wallace, Jr. 1999. *Mycobacterium*, p. 399-437. *In* P.R. Murray, E.J. Baron, M.A. Pfaller, F.C. Tenover, and R.H. Tenover (ed.), Manual of clinical microbiology, 7th ed. American Society for Microbiology, Washington, D.C.

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For US: "For Investigational Use Only"

	Lower limit of temperature / Долен лимит на температурата / Dolni hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrænse / Dolna granica temperatury / Limite mínimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sıcaklık alt sınırı / Мінімальна температура / 温度下限
CONTROL	Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / 컨트롤 / Kontrolé / Kontrolle / Controle / Controllo / Контроль / kontroll / Контроль / 对照
CONTROL +	Positive control / Положителен контрол / Pozitívni kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitívna kontrola / Pozitív kontroll / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívá kontrolle / Positieve controle / Kontrola dodatnia / Controllo positivo / Control pozitiv / Положительный контроль / Pozitíf kontrol / Позитивный контроль / 阳性对照试剂
CONTROL -	Negative control / Отрицателен контрол / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negativna kontrola / Negatív kontroll / Controllo negativo / Негативтік бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívá kontrolle / Negatieve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatív kontrol / Негативный контроль / 阴性对照试剂
STERILE EO	Method of sterilization: ethylene oxide / Метод на стерилизация: етиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringsmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseerimismeetod: etüleenoksiid / Méthode de stérilisation: oxyde d'éthylène / Metoda sterilizácie: etilen oksid / Sterilizálás módszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Стерилизация әдісі – этилен тотығы / 소독 방법: 에틸렌 옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksīds / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringsmetode: etylenoksid / Metoda sterilizacji: tenek etylu / Método de esterilização: óxido de etileno / Metódá de sterilizáre: oxid de etilenă / Μεθοδ στεριλίζασις: εθιλενοξιδο / Metódá sterilizácie: etylénoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetode: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: етиленокси́дом / 灭菌方法: 环氧乙烷
STERILE R	Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringsmetode: bestråling / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseerimismeetod: kiirgus / Méthode de stérilisation: irradiation / Metoda sterilizácie: zračenie / Sterilizálás módszere: besugárzás / Metodo di sterilizzazione: irradiazione / Стерилизация әдісі – сәуле түсіру / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringsmetode: bestråling / Metoda sterilizacji: napromienianie / Método de esterilização: irradiação / Metódá de sterilizáre: iradiere / Μεθοδ στεριλίζασις: οβлучение / Metódá sterilizácie: ožiarenie / Metoda sterilizacije: ozračavanje / Steriliseringsmetod: strålning / Sterilizasyon yöntemi: iradyasyon / Метод стерилизації: опроміненням / 灭菌方法: 辐射
	Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Bioloogilised riskid / Risques biologiques / Biološki rizik / Biológiallag veszélyes / Rischio biologico / Биологический төуекел / 생물학적 위험 / Bioginis pavojus / Biologiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyolojik Riskler / Биологічна небезпека / 生物学风险
	Caution, consult accompanying documents / Внимание, направате справка в придружаващите документи / Pozor! Prostudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeda kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Urozojenje, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dmesio, žiūrėkite priededamus dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažňjal Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацию / 小心, 请参阅附带文档。
	Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrænse / Góna granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sıcaklık üst sınırı / Максимальна температура / 温度上限
	Keep dry / Пазерте сухо / Skladujte v suchém prostředí / Opbevaras tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Százás helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laikykite sausiai / Uzglabāt sausu / Droog houden / Holdes tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávaťe v suchu / Držite na suvom mestu / Förvaras tørt / Kuru bir şekilde muhafaza edin / Бергетти від вологи / 请保持干燥
	Collection time / Време на събиране / Čas odběru / Opsamlingsstidspunkt / Entnahmehurzeit / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жынау уақыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colecta / Ora colectării / Время сбора / Doba odboru / Vreme prikupljanja / Uppsamlingsstid / Toplama zamanı / Час забору / 采集时间
	Peel / Obelene / Otevřete zde / Abn / Abziehen / Αποκολλήστε / Desprendre / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Υστίμηri қабатын алып таста / 벗기기 / Plēsti cia / Atīmēt / Schillen / Trekk av / Oderwać / Destacar / Se dezlipeste / Отклеить / Odrhňte / Oljuštiti / Dra isär / Ayırma / Відклеїти / 撕下
	Perforation / Перфорация / Perforace / Perforering / Διάτρηση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Tecik teci / 절취선 / Perforacija / Perforácia / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация / 穿孔
	Do not use if package damaged / Не използвайте, ако опаковката е повредена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packungsmittel verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά. / No usar si el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használja, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Егер пакет бұзылған болса, пайдаланба / पैकेज کا 손상된 경우 사용 금지 / Jei pakuotė pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не использовать при повреждении упаковки / Nepoužívejte, ak je obal poškodený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包装破损, 请勿使用
	Keep away from heat / Пазерте от топлина / Nevystavujte přílišnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Сапқын жерде сақта / 열을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / He награвать / Uchovávaťe mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Isidan uzak tutun / Бергетти від дії тепла / 请远离热源
	Cut / Срежете / Odstřihněte / Klip / Schneiden / Κόψτε / Cortar / Lõigata / Découper / Režti / Vágja ki / Tagliare / Keciňia / 잘라내기 / Kirpti / Nogriez / Knippen / Kutt / Odciać / Cortar / Decupați / Отрезать / Odstihněte / Iseći / Klipp / Kesme / Pozрізати / 剪下
	Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuupäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жынаған тізбекүні / 수집 날짜 / Paėmimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colecta / Data colectării / Дата сбора / Datum odboru / Datum prikupljanja / Uppsamlingsdatum / Toplama tarihi / Дата забору / 采集日期
	µL/test / µL/тест / µL/Test / µL/εξέταση / µL/prueba / µL/teszt / µL/테스트 / мкл/тест / µL/tyrimas / µL/pārbaude / µL/teste / мкл/анализ / µL/檢測
	Keep away from light / Пазерте от светлина / Nevystavujte světlu / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svjetla / Fény nem érheti / Tenere al riparo dalla luce / Қараңғыланған жерде ұста / 빛을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródeł światła / Manter ao abrigo da luz / Feriți de lumină / Хранити в темноте / Uchovávaťe mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Işıktan uzak tutun / Бергетти від дії світла / 请远离光线
	Hydrogen gas generated / Образуван е водород газ / Možnost úniku plyného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikagaasi tekitatud / Produit de l'hydrogène gazeux / Sadrži hydrogen vodik / Hidrogén gázt fejleszt / Produzione di gas idrogeno / Газтекес сутегі пайда болды / 수소 가스 생성됨 / Išiskiria vandenilio dujas / Rodas degradis / Waterstofgas gegenereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené puzitím vodíka / Oslobada se vodonik / Genererad vätgas / Açığa çıkan hidrojen gazı / Реакция з виділенням водню / 会产生氢气

Patient ID number / ИД номер на пациента / ID pacienta / Patientens ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsiendi ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттин идентификациялык нөмірі / 환자 ID 번호 / Paciento identifikavimo numeris / Pacienta ID numurs / Identificatienummer van de patiënt / Pasientens ID-nummer / Numer ID pacienta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası / Идентификатор пациента / 患者标识号

Fragile, Handle with Care / Чупливо, Роботете с необходимото внимание. / Křehké. Při manipulaci postupujte opatrně. / Forsigtig, kan gå i stykker. / Zerbrechlich, vorsichtig handhaben. / Εύθραυστο. Χειριστείτε το με προσοχή. / Frágil. Manipular con cuidado. / Őrn, käsitsege ettevaatlikult. / Fragile. Manipuler avec précaution. / Lomljivo, rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. / Сынгыш, абайлап пайдаланыңыз. / 조심 깨지기 쉬운 처리 / Trapu, elkites atsargiai. / Trausis; rikoties uzmanigi / Breekbaar, voorzichtig behandelen. / Ømtålig, håndter forsigtig. / Krucha zawartość, przenosić ostrożnie. / Frágil, Manuseie com Cuidado. / Frágil, manipulați cu atenție. / Хрупкое! Обращаться с осторожностью. / Křehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşıyın. / Тендітна, звертатися з обережністю / 易碎，小心轻放

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Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

ATCC is a trademark of the American Type Culture Collection.

© 2019 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.