

BD BACTEC™ FOS™ Culture Supplement Kit*

R_x Only  

PP083JAA(04)
2019-09

INTENDED USE

BD BACTEC™ FOS™ Culture Supplement Kit is a fastidious organism supplement and growth enhancer. FOS is provided in a lyophilized form along with a special FOS Reconstituting Fluid (FOS RF) for use with BD BACTEC culture media (radiometric and non-radiometric) to enhance the growth of fastidious organisms, such as *Haemophilus* and *Neisseria*. Principal use is in conjunction with BD BACTEC blood culture media and the BD BACTEC instruments.

SUMMARY AND EXPLANATION

The sample to be tested is inoculated into one or more BD BACTEC culture vials. Two (2) mL of the reconstituted FOS is then added to each BD BACTEC blood culture vial and the vials are then incubated. The culture vial is periodically tested on the BD BACTEC instrument by aspiration of the supernatant gas and assay of its CO₂ content. A positive reading indicates the presence of viable organisms in the vial.

PRINCIPLES OF THE PROCEDURE

The FOS Supplement contains nicotinamide adenine dinucleotide (NAD) and hemin which are essential growth requirements for some microorganisms such as *Haemophilus*. Use of the supplement as an enrichment when these organisms are suspected, particularly when blood is not present such as with body fluid specimens, enhances the opportunity for growth. The reconstituted FOS Supplement also contains components that neutralize sodium polyanetholesulfonate (SPS) thereby enhancing the growth of SPS-sensitive organisms such as *Neisseria*.¹

REAGENTS

The kit consists of four small amber vials of lyophilized FOS Culture Supplement and one larger clear vial of FOS Reconstituting Fluid. Lyophilized FOS Culture Supplement (each vial)

Bovine Serum Albumin	825	mg
Nicotinamide Adenine Dinucleotide (NAD)	11	mg
Hemin	165	µg

FOS Reconstituting Fluid

Vinylpyrrolidone Copolymer	4.0	mL
Citric Acid	202	mg
Processed Water	44.0	mL

Warnings and Precautions

The FOS Culture Supplement is for *in vitro* diagnostic use.

This Product Contains Dry Natural Rubber.

For use in blood culture medium, read and follow instructions and information supplied with these products.

Pathogenic microorganisms including hepatitis viruses and Human Immunodeficiency Virus may be present in clinical specimens. "Standard precautions"²⁻⁵ and institutional guidelines should be followed in handling all items contaminated with blood or other body fluids.

Prior to use, each lyophilized FOS Culture Supplement vial should be examined for evidence of deterioration and moisture. The FOS Reconstituting Fluid and reconstituted FOS Culture Supplement should be examined for evidence of contamination such as excessive cloudiness, bulging or depressed stopper, or leakage. DO NOT USE any vial showing evidence of contamination or deterioration.

Prior to use, the user should examine the vials for evidence of damage or deterioration. Vials displaying turbidity, contamination, or discoloration (darkening) should not be used. On rare occasions, the glass bottle neck may be cracked and the neck may break during removal of the flip-off cap or in handling. Also, on rare occasions a vial may not be sealed sufficiently. In both cases the contents of the vials may leak or spill.

To minimize the potential of leakage, use syringes with permanently attached needles or securely fastened BD Luer-Lok™ tips.

Reconstitute the supplement vial only with FOS Reconstituting Fluid.

DO NOT add the supplement to BD BACTEC culture vials prior to the inoculation of blood if direct draw technique is to be used.

Storage Instructions

The kit, containing unreconstituted supplement, may be stored at 2–25 °C. Reconstituted supplement vials *must* be stored under refrigeration at 2–8 °C and may be used for up to one month after reconstitution. **Do not use** after the expiration date.

SPECIMEN COLLECTION AND PREPARATION

The specimen must be collected using sterile techniques to reduce the chance of contamination (please refer to the culture medium package insert).

PROCEDURE

Materials Provided:

Lyophilized FOS Culture Supplement
FOS Reconstituting Fluid

Materials Required But Not Provided:

BD BACTEC Blood Culture Medium
Sterile Syringe with permanently attached needle
70% Isopropyl Alcohol
BD BACTEC Instrument

Preparation of FOS Culture Supplement

If the FOS Reconstituting Fluid has been stored at 2–8 °C, allow the fluid to come to room temperature for ease of transfer.

Precautions: Examine the vials for evidence of contamination or defects. Do not use any defective or contaminated vials.

Remove the plastic flip-off caps from the vials to expose the septa. Clean the septum of the lyophilized FOS Culture Supplement vial and the FOS Reconstituting Fluid vial with an alcohol swab. **Aseptically transfer 11.0 mL** of the Reconstituting Fluid into the Supplement vial. **DO NOT SHAKE**, but instead allow the vial to sit at room temperature for 20–30 min, and then gently swirl the vial to complete the mixing. Reconstituted FOS *must* be stored at 2–8 °C. It may be used up to one month from the date of reconstitution.

Addition of the Supplement

Aseptically add 2.0 mL of reconstituted FOS per vial of BD BACTEC medium. Media containing FOS will appear slightly turbid. Incubate the culture vials at 34–36 °C.

Test Procedure

Follow the recommended testing procedure for each blood culture medium.

QUALITY CONTROL

The sterility of the supplement may be checked by examining for growth in sterile media.

Quality Control Certificates are provided with each kit of Supplement.⁶

Do not use vials past their expiration date.

Quality control requirements must be performed in accordance with applicable local, state and/or federal regulations or accreditation requirements and your laboratory's standard Quality Control procedures. It is recommended that the user refer to pertinent CLSI guidance and CLIA regulations for appropriate Quality Control practices.

LIMITATIONS OF THE PROCEDURE

Contamination

Care must be taken to prevent contamination of the body fluid sample and/or the Supplement. A contaminated sample and/or Supplement will give a positive reading, but will not reflect a positive clinical sample.

Performance data were developed using BD BACTEC PLUS 26 and PLUS 27 media.

EXPECTED VALUES AND PERFORMANCE CHARACTERISTICS

In seeded studies with two strains each of *Neisseria gonorrhoeae*, *Neisseria meningitidis*, and *Haemophilus influenzae*, BD BACTEC blood culture medium without additions was compared to medium supplemented with BD BACTEC FOS Supplement and medium containing whole human blood. The results showed that BD BACTEC medium containing BD BACTEC FOS Culture Supplement supports the growth of fastidious and/or SPS sensitive bacteria without the presence of blood.

AVAILABILITY

Cat. No. Description

442153 BD BACTEC™ FOS™ Culture Supplement Kit

*EPO Patent No. 489392 and other international patents

REFERENCES

1. Fuller, D., et al. 1991. Comparison of culturing sterile body fluids using the BACTEC Plus 26/27 media and FOS (Fastidious Organism Supplement) to conventional methods. Abstracts of the Annual Meeting of the Amer. Soc. for Microbiol., Dallas, TX.
2. National Committee for Clinical Laboratory Standards. 2001. Approved Standard M29-A2. Protection of laboratory workers from occupationally acquired infections, 2nd ed. NCCLS, Wayne, Pa.
3. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53–80.
4. U.S. Dept. of Health and Human Services. 1999. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 4th ed. U.S. Government Printing Office, Washington, D.C.
5. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021–0045.
6. National Committee for Clinical Laboratory Standards. 1996. Approved Standard M22-A2. Quality Assurance for Commercially Prepared Microbiological Culture Media, 2nd ed. NCCLS, Wayne, Pa.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

Change History

Revision	Date	Change Summary
(04)	2019-09	Converted printed instructions for use to electronic format and added access information to obtain the document from BD.com/e-labeling .

US Customers only: For symbol glossary, refer to www.bd.com/symbols-glossary



Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbicante / Аткарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производител / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник / 生产厂商



Use by / Използвайте до / Spotføjbejdt do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izljetot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použite do / Upotrebiti do / Använd före / Son kulanma tarihi / Використати до / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = slutning af måned)
JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)
EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)
AAAA-MM-DD / AAAA-MM (MM = fin del mes)
AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)
AAAA-MM-JJ / AAAA-MM (MM = fin du mois)
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)
EEEE-HH-NN / EEEE-HH (HH = hónap utolsó napja)
AAAA-MM-GG / AAAA-MM (MM = fine mese)
ЖОЖОЖ-АА-КК / ЖОЖОЖ-АА / (АА = айдың соңы)
YYYY-MM-DD/YYYY-MM (MM = 월말)
MMMM-MM-DD / MMMM-MM (MM = mėnesio pabaiga)
GGGG-MM-DD/GGGG-MM (MM = mėneša beigas)
JJJJ-MM-DD / JJJJ-MM (MM = einde maand)
AAAA-MM-DD / AAAA-MM (MM = slutten av månaden)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = fim do mês)
AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
GGGG-MM-DD / GGGG-MM (MM = kraj meseca)
AAAA-MM-DD / AAAA-MM (MM = slutet av månaden)
YYYY-AA-GG / YYYY-AA (AA = ayın sonu)
PPPP-MM-DD / PPPP-MM (MM = кінець місяця)
YYYY-MM-DD / YYYY-MM (MM = 月末)



Catalog number / Каталоген номер / Katalógové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalógusszám / Numero di catalogo / Каталог номер / 카탈로그 번호 / Katalogo / numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalógové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目录号



Authorized Representative in the European Community / Оторизирани представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret representant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Evropskoj uniji / Meghatalmazott képviselő az Európai Közösségekben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті өкіл / 유럽 공동체의 위임 대표 / Įgaliotasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Avrupa Topluluğu Yetkili Temsilcisi / Уповноважений представник у країнах ЄС / 欧洲共同体授权代表



In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiiniparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnosztikai orvosi eszköz / Dispositivo medical per diagnostica in vitro / Жасанды жағдайда жүргізетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietais / Medicinas ierīces, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispozitiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomůcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diagnostik Tibbi Cihaz / Медицинский прибор для диагностики in vitro / 体外诊断医疗设备



Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrænsning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperaturi piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шектеу / 온도 제한 / Laikymo temperatūra / Temperatūras ierobežojumi / Temperaturlimit / Temperaturbegrensning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ohraničenje teploty / Ograničenje temperature / Temperaturgräns / Sıcaklık sınırlaması / Обмеження температури / 温度限制



Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топтама коды / 배치 코드(코트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot number / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партії / 批号 (亚批)



Contains sufficient for <n> tests / Съдържанието е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenido suficiente per <n> test / <n> тесттері үшін жеткінікті / <n> 테스트가 충분히 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor "n" testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Continut suficient pentru <n> teste / Достаточно для <n> тестов(a) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli malmzeme içerir / Вистачить для аналізів: <n> / 足够进行 <n> 次检测



Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / 사용 지침 참조 / Skaityti naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se bruksanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції з використання / 请参阅使用说明



Do not reuse / Не използвайте отново / Nepoužívejte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egyeszer használatos / Non riutilizzare / Пайдаланбаңыз / 재사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolositi / Не использовать повторно / Nepoužívejte opakovaně / Ne upotrebljavajte ponovo / Får ej återanvändas / Tekrar kulanmayin / Не використовувати повторно / 请勿重复使用



Serial number / Серийен номер / Sériové číslo / Seriennummer / Seriennummer / Σειριακός αριθμός / N° de serie / Seerianumber / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Топтамалық нөмірі / 일련 번호 / Serijos numeris / Sérjjas numurs / Serie nummer / Numer seryjny / Número de série / Număr de serie / Серийный номер / Seri numarası / Номер серії / 序列号

	For IVD Performance evaluation only / Само за оценка качество на работа на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD-Leistungsbewertungszwecke / Μόνο για αξιολόγηση απόδοσης IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réserve à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárlag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде» диагностикада тек жұмысты бағалау үшін / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisų veikimo charakteristikoms tikrinti / Vienīgi IVD darbības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering av IVD-ytelse / Tylko do oceny wydajności IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики in vitro / Určené iba na diagnostiku in vitro / Samo za procenu učinka u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Yalnızca IVD Performans değerlendirmesi için / Тільки для оцінювання якості діагностики in vitro / 仅限 IVD 性能评估
	For US: "For Investigational Use Only"
	Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūros žemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrense / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sıcaklık alt sınırı / Мінімальна температура / 温度下限
CONTROL	Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Contollo / Бақылау / 컨트롤 / Kontrolé / Kontrolle / Controle / Controlo / Контроль / kontroll / 对照
CONTROL +	Positive control / Положителен контрол / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Pozitív kontroll / Contollo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívá kontrolé / Positieve controle / Kontrola dodatnia / Controlo positivo / Положительный контроль / Pozitif kontrol / Позитивний контроль / 阳性对照试剂
CONTROL -	Negative control / Отрицателен контрол / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negativna kontrola / Negativ kontroll / Contollo negativo / Негативный бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívá kontrolé / Negatieve controle / Kontrola ujemna / Controlo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативний контроль / 阴性对照试剂
STERILE EO	Method of sterilization: ethylene oxide / Метод на стерилизация: етиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringensmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseringensmetode: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metoda di sterilizzazione: ossido di etilene / Стерилизация әдісі – этилен тотығы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksisds / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringensmetode: etylenoksid / Metoda sterylizacji: tienek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metodá sterilizácie: etylenoxid / Metoda sterilizacije: etilen oksid / Steriliseringensmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом / 灭菌方法: 环氧乙烷
STERILE R	Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringensmetode: bestrålning / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseringensmetode: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metoda di sterilizzazione: irradiazione / Стерилизация әдісі – сәуле тисы / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringensmetode: bestrålning / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизации: облучение / Metodá sterilizácie: ožiarenie / Metoda sterilizacije: ozračevanje / Steriliseringensmetod: strålning / Sterilizasyon yöntemi: ırdıyasyon / Метод стерилизації: опроміннення / 灭菌方法: 辐射
	Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologilised riskid / Risques biologiques / Biološki rizik / Biológiai veszélyes / Rischio biologico / Биологический тауекелдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisches risiko / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Biologische опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyolojik Riskler / Біологічна небезпека / 生物学风险
	Caution, consult accompanying documents / Внимание, направете справка в придружаващите документи / Pozor! Pristudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeka kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Urozorenje, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dămesio, žiūrėkite priedamus dokumentus / Piesardziba, skatit pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Vystraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацію / 小心, 请参阅附带文档。
	Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Ausgējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrense / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sıcaklık üst sınırı / Міксимальна температура / 温度上限
	Keep dry / Пазете сухо / Skladujte v suchém prostředi / Opbevarer tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laikykite sausiai / Uzglabāt sausu / Droog houden / Holdes tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávaťe v suchu / Držite na suvom mestu / Förvaras tørt / Kuru bir şekilde muhafaza edin / Бергетти від вологості / 请保持干燥
	Collection time / Време на събиране / Čas odběru / Opsamlingsstidspunkt / Entnahmeuhrzeit / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жынау уақыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid förövtagning / Godzina pobrania / Hora de colheita / Ora colectării / Время сбора / Doba odboru / Vreme prikupljanja / Uppsamlingsstid / Toplama zamanı / Час забору / 采集时间
	Peel / Обелете / Otevfete zde / Abn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Ўстиғи қабатын алып таста / 벗기 / Plęsti čia / Atfimt / Schillen / Trek av / Oderwać / Destacar / Se dezlipęte / Отклеить / Odrhните / Oljuštiti / Dra isär / Ayırma / Відкрити / 撕下
	Perforation / Перфорация / Perforace / Perforering / Διάτρηση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Тесик тесу / 찢취선 / Perforacija / Perforácia / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация / 穿孔
	Do not use if package damaged / Не използвайте, ако опаковката е повредена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packungnicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά / No usar si el paquete está dañado / Inne kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használna, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Ереп пакет бүзылган болса, пайдаланба / 패키지가 손상된 경우 사용 금지 / Jei pakuotė pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не использовать при повреждении упаковки / Nepoužívaťe, ak je obal poškodený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包裝破損, 請勿使用
	Keep away from heat / Пазете от топлина / Nevystavujte přílišnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valguusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Сапқын жерде сақта / 열을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / Не награвать / Uchovávaťe mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Isidan uzak tutun / Бергетти від дії тепла / 请远离热源
	Cut / Срежете / Odstřihněte / Klip / Schneiden / Κόψτε / Cortar / Lőigata / Découper / Reži / Vágja ki / Tagliare / Keciңiz / 잘라내기 / Kirpti / Nogriez / Knippen / Kutt / Odciąć / Cortar / Decupaji / Отрезать / Odstřihnite / Iseći / Klipp / Kesme / Pozpizati / 剪下



Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuurpäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаған тізбекүні / 수집 날짜 / Paëmimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colheita / Data colectării / Дата сбора / Datum odberu / Datum prikupljanja / Uppsamlingsdatum / Toplama tarihi / Дата забору / 采集日期



µL/test / µL/тест / µL/Test / µL/εξέταση / µL/prueba / µL/teszt / µL/테스트 / мкл/тест / µL/tyrimas / µL/pårbaude / µL/teste / мкл/анализ / µL/检测



Keep away from light / Παзете от светлина / Nevystavujte světlu / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svetla / Fény nem érheti / Tenere al riparo dalla luce / Қараңғыланған жерде ұста / 빛을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródła światła / Manter ao abrigo da luz / Feriți de lumină / Хранить в темноте / Uchovávajte mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Ізійтан узак тутун / Беретти від дії світла / 请远离光线



Hydrogen gas generated / Образуван е водород газ / Možnost úniku plynného vodíku / Frembringer hydrogengas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadrží hydrogen vodík / Hidrogén gázt fejleszt / Produzione di gas idrogeno / Газтекес сутері пайда болды / 수소 가스 생성됨 / Išskiria vandenilio dujas / Rodas ūdenradis / Waterstofgas gegenereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené použitím vodíka / Osloбаda se vodonik / Genererad vätgas / Açığa çıkan hidrojen gazı / Реакція з виділенням водню / 会产生氢气



Patient ID number / ИД номер на пациента / ID pacienta / Patientens ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsiendi ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттің идентификациялық нөмірі / 환자 ID 번호 / Paciento identifikavimo numeris / Pacienta ID numurs / Identificationnummer van de patiënt / Pasientens ID-nummer / Numer ID pacienta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası / Идентификатор пациента / 患者标识号



Fragile, Handle with Care / Чупливо, Работете с необходимото внимание. / Křehké, Při manipulaci postupujte opatrně. / Forsigtig, kan gå i stykker. / Zerbrechlich, vorsichtig handhaben. / Εύθραστο. Χειριστείτε το με προσοχή. / Frágil. Manipular con cuidado. / Őm, kásitsege ettevaatlíkuil. / Fragile. Manipuler avec précaution. / Lomljivo, rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. / Сынғыш, абайлап пайдаланыңыз. / 조심 깨지기 쉬운 처리 / Trapu, elkites atsargiai. / Trausls; rikoties uzmanīgi / Breekbaar, voorzichtig behandelen. / Ømtålig, håndter forsigtig. / Krucha zawartość, przenosić ostrożnie. / Frágil, Manuseie com Cuidado. / Frágil, manipulați cu atenție. / Хрупкое! Обращаться с осторожностью. / Křehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşıyin. / Тендітна, звертатися з обережністю / 易碎, 小心轻放

R_x Only

This only applies to US: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / S'applique uniquement aux États-Unis: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Vale solo per gli Stati Uniti: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Gilt nur für die USA: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Sólo se aplica a los EE.UU.: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner."



bd.com/e-labeling
KEY-CODE: PP083JAA

Europe, CH, GB, NO:		+800 135 79 135
International:		+31 20 794 7071
AR	+800 135 79 135	LT 8800 30728
AU	+800 135 79 135	MT +31 20 796 5693
BR	0800 591 1055	NZ +800 135 79 135
CA	+1 855 805 8539	RO 0800 895 084
CO	+800 135 79 135	RU +800 135 79 135
EE	0800 0100567	SG 800 101 3366
GR	0800 161 22015 7799	SK 0800 606 287
HR	0800 804 804	TR 0800 142 064 866
IL	+800 135 79 135	US +1 855 236 0910
IS	800 8996	UY +800 135 79 135
LI	+31 20 796 5692	VN 122 80297



Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA



Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:

Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

ATCC® is a trademark of the American Type Culture Collection.

BD, the BD Logo, BACTEC, FOS and Luer-Lok are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2019 BD. All rights reserved.