


Revisions

SO 0191-5


Rev from	Rev to	ECO #
02	2011/01	5677-11

Notes:

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
- BD Cat. Number 448903
- Blank (Sheet) Size : Length: 14" Width: 8.5"
 Number of Pages: 2 Number of Sheets: 1
 Page Size: Length 14" Width 8.5" Final Folded Size: No Fold
- Style (see illustrations below): # 1




#1

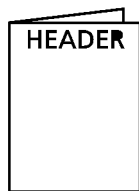


#2

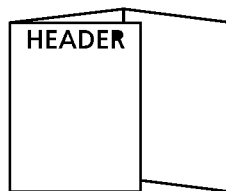


#3

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


#4

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

#5

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

#6

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

#7

- See Specification Control Number N/A for Material Information
- Ink Colors: Printed two sides ☒ Yes ☐ No
 No. of Colors: 1 PMS# Standard Black
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

PC-Generated Insert

Label Design	Date	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION	 Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA	
Proofer	Date			
Checked By	Date			
Part Number: L009438		Category and Description Phoenix Insert, PMIC/ID-51	Sheet: 1 of 3	A
			Scale: N/A	

Table 1

Antimicrobial / Antibiotiques / Antibiotikum / Antibiotico / Agente antimicrobiano / Anti-microbiano		(µg/mL)	<i>S. aureus</i> ATCC 29213	<i>E. faecalis</i> ATCC 29212	<i>E. faecalis</i> ATCC 51299	<i>S. aureus</i> ATCC 25923	
Amoxicillin/Clavulanate (f)	AXC	2/2 - 8/2	≤0.25/2 - 0.5/2	≤0.25/2 - 1/2			
Ampicillin	AM	2 - 8	0.5 - >1	0.5 - 2			
Beta-Lactamase (Nitrocefin-based)	NCF	<10	POS			NEG	
Cefazolin	CZ	2 - 8	≤0.5 - 1				
Cefoxitin	FOX	2 - 8	≤1 - 4				
Ciprofloxacin	CIP	0.5 - 2	≤0.125 - 0.5	0.5 - 4			
Clindamycin	CC	0.5 - 4	≤0.125 - 0.25	4 - >8			
Erythromycin	E	0.25 - 4	0.25 - 1				
Fosfomycin	FF	16 - 64	<8	32 - 128			
Fusidic Acid	FA	2 - 8	≤0.5 - 1	1 - 4			
Gentamicin	GM	1 - 8	≤0.5 - 1				
Gentamicin-Synergy	GMS	500		≤500	>500		
Levofloxacin	LVX	0.5 - 2	≤0.25 - 0.5	≤0.25 - 2			
Linezolid	LZD	1 - 4	1 - 4	1 - 4			
Meropenem	MEM	2 - 8	≤0.5	2 - 8			
Moxifloxacin	MXF	0.5 - 2	≤0.125	≤0.125 - 0.5			
Mupirocin -High Level	MUH	256	<256	<256			
Ofloxacin	OFX	0.5 - 2	≤0.25 - 1	1 - 4			
Oxacillin	OX	0.25 - 2	0.125 - 0.5	>4			
Penicillin	P	0.125 - 0.5	0.25 - >1	1 - 4			
Rifampin	RA	1 - 4	≤0.25				
Streptomycin-Synergy	STS	1000		≤1000	>1000		
Teicoplanin	TEC	1 - 16	≤0.5 - 1	≤0.5			
Tetracycline	TE	0.5 - 4	≤0.5 - 1	8 - >16			
Trimethoprim/Sulfamethoxazole (DIN)	STG	0.8/15.2 - 3.2/60.8	≤0.4/7.6	≤0.4/7.6			
Trimethoprim/Sulfamethoxazole	SXT	0.5/9.5 - 2/38	≤0.5/9.5	≤0.5/9.5			
Vancomycin	VA	1 - 16	≤0.5 - 2	1 - 4			

Table 2

Organism / Organisme / Organismus / Organismo / Organismo / Microrganismo	Not Reported / Nonrapporté / Nicht berichtet / Non riportato / No registrado / Nao Participado	Alternate Method / Utiliser autre méthode / Alternativmethode anwenden / Usare metodo alternativo / Método alternativo recomendado / Metodo Alternativo

Table 3

Organism / Organisme / Organismus / Organismo / Organismo / Microrganismo	Not Reported / Nonrapporté / Nicht berichtet / Non riportato / No registrado / Nao Participado	Alternate Method / Utiliser autre méthode / Alternativmethode anwenden / Usare metodo alternativo / Método alternativo recomendado / Metodo Alternativo
<i>Staphylococcus epidermidis</i>	Ampicillin	Trimethoprim/Sulfamethoxazole (DIN)
Coagulase-negative <i>Staphylococcus</i> other than <i>S. epidermidis</i>	Ampicillin, Penicillin	
<i>Enterococcus</i> other than <i>E. faecalis</i> and <i>E. faecium</i>	Ciprofloxacin, Levofloxacin	
<i>Enterococcus faecalis</i>		Erythromycin
All <i>Enterococcus</i> species	Rifampin	