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XIV REFERANSLAR

1. Federal Register. 1994. Prevention of group B streptococcal disease: a public health perspective. Fed. Regist. 59:64764-64773.
2. Centers for Disease Control and Prevention. 2002. Morbid. Mortal. Weekly Rep. 51 (No. RR-11):1.
3. Ruoff, K.L., R.A. Wiley, and D. Beighton. 2003. *Streptococcus*, p. 405-421. In P.R. Murray, E.J. Baron, J.H. Jorgensen, M.A. Tenover, and R.H. Tenover (ed.), Manual of clinical microbiology, 8th ed. American Society for Microbiology, Washington, D.C.
4. Forbes, B.A., D.F. Sahm, and A.S. Weissfeld. 2002. Bailey & Scott's diagnostic microbiology, 11th ed. Mosby, Inc., St. Louis.

5. Todd, E.W., and L.F. Hewitt. 1932. A new culture medium for the production of antigenic streptococcal haemolysin. J. Pathol. Bacteriol. 35:973-974.
6. Updyke, E.L., and M.I. Nickle. 1954. A dehydrated medium for the preparation of type specific extracts of group A streptococci. Appl. Microbiol. 2:117-118.
7. MacFaddin, J.F. 1985. Media for isolation-cultivation-identification-maintenance of medical bacteria, vol. 1. Williams & Wilkins, Baltimore.
8. National Committee for Clinical Laboratory Standards. 2001. Approved Guideline M29-A2. Protection of laboratory workers from occupationally acquired infections, 2nd ed. NCCLS, Wayne, Pa.
9. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53-80.
10. U.S. Department of Health and Human Services. 1999. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 4th ed. U.S. Government Printing Office, Washington, D.C.
11. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021-0045.
12. Murray, P.R., E.J. Baron, J.H. Jorgensen, M.A. Pfaller, and R.H. Tenover (ed.) 2003. Manual of clinical microbiology, 8th ed. American Society for Microbiology, Washington, D.C.
13. Isenberg, H. (ed.). 2004. Clinical microbiology procedures handbook, vol. 1, 2 and 3, 2nd ed. American Society for Microbiology, Washington, D.C.
14. Holt, J.G., N.R. Krieg, P.H.A. Sneath, J.T. Staley, and S.T. Williams (ed.). 1994. Bergey's Manual™ of determinative bacteriology, 9th ed. Williams & Wilkins, Baltimore.

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Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

ATCC is a trademark of the American Type Culture Collection.
BD, BD Logo, and all other trademarks are property of Becton, Dickinson and Company. © 2015 BD