(bočice za staničnu kulturu)

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442023 BD BACTEC Plus Aerobic/F Medium (podloga)

REFERENCE

1. Wallis, C. *et al.* 1980. Rapid isolation of bacteria from septicemic patients by use of an antimicrobial agent removal device. *J. Clin. Microbiol.* 11:462–464.
2. Applebaum, P.C. *et al.* 1983. Enhanced detection of bacteremia with a new BACTEC resin blood culture medium. *J. Clin. Microbiol.* 17:48–51.
3. Pohlman, J.K. *et al.* 1995. Controlled clinical comparison of Isolator and BACTEC 9240 Aerobic/F resin bottle for detection of bloodstream infections. *J. Clin. Microbiol.* 33:2525–2529.
4. Clinical and Laboratory Standards Institute. 2005. Approved Guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, Pa.
5. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. *Infect. Control Hospital Epidemiol.* 17: 53–80.
6. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C.
7. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021–0045.
8. Smith, J.A. *et al.* 1995. Comparison of BACTEC 9240 and BacT/Alert blood culture systems in an adult hospital. *J. Clin. Microbiol.* 33:1905–1908.
9. Flayhart, D. *et al.* 2007. Comparison of BACTEC Plus blood culture media to BacT/Alert FA blood culture media for detection of bacterial pathogens in samples containing therapeutic levels of antibiotics. *J. Clin. Microbiol.* 45:816–821.
10. Data available from BD Life Sciences.
11. Howden, R.J. 1976. Use of anaerobic culture for the improved isolation of *Streptococcus pneumoniae*. *J. Clin. Pathol.* 29:50–53.
12. Ilstrup, Duane M. and John A. Washington. 1983. The importance of volume of blood cultured in the detection of bacteremia and fungemia. *Diagn. Microbiol. Infect Dis.* 1:107–110.

Tehnički servis i podrška: obratite se lokalnom predstavniku tvrtke BD ili posjetite bd.com.

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Revizija	Datum	Sažetak izmjena
(05)	2019-09	Tiskano uputstvo za uporabu pretvoreno je u elektronički oblik i dodani su pristupni podaci za dohvaćanje dokumenta s web-mjesta BD.com/e-labeling. U odjeljku Upozorenja i mjere opreza, dodane su preporuke za provođenje molekularnih testova na pozitivnim kulturama krvi u skladu s praksama zdravstvenih standarda i uputstvima za uporabu proizvođača.

	Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbricante / Атқарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirkētājs / Producent / Producător / Produçor / Производитель / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник / 生产厂商
	Use by / Используйте до / Spotrebujte do / Brug før / Verwendbar bis / Хрънг ёвс / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrijebite do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izletot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza pánâ la / Использовать до / Použíte do / Upotrebiti do / Använd före / Son kullanma tarihi / Використати доділе / 使用截止日期 YYYY-MM-DD / YYYY-MM (MM = end of month) ГГГГ-ММ-ДД / ГГГГ-ММ (MM = края на месеца) RRRR-MM-DD / RRRR-MM (MM = konec měsíce) AAAA-MM-DD / AAAA-MM (MM = slutning af måneden) JJJJ-MM-TT / JJJJ-MM (MM = Monatsende) EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα) AAAA-MM-DD / AAAA-MM (MM = fin del mes) AAAA-KK-PP / AAAA-KK (KK = кuu lõpp) AAAA-MM-JJ / AAAA-MM (MM = fin du mois) GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca) ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónag utolsó napja) AAAA-MM-GG / AAAA-MM (MM = fine mese) ЖОЮЖА-АА-КК / ЖОЮЖА-АА (АА = айдын соны) YYYY-MM-DD/YYYY-MM(MM = 월말) ММММ-ММ-ДД / ММММ-ММ (MM = ménésio pabaiga) GGGG-MM-DD/GGGG-MM (MM = mēneša beigas) JJJJ-MM-DD / JJJJ-MM (MM = einde maand) AAAA-MM-DD / AAAA-MM (MM = slutten av måneden) RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca) AAAA-MM-DD / AAAA-MM (MM = fim do mês) AAAA-LI-ZZ / AAAA-LI (LL = sfârșitul lunii) ГГГГ-ММ-ДД / ГГГГ-ММ (MM = конец месяца) RRRR-MM-DD / RRRR-MM (MM = koniec mesiaca) GGGG-MM-DD / GGGG-MM (MM = kraj meseca) AAAA-MM-DD / AAAA-MM (MM = slutten av månaden) YYYY-AA-GG / YYYY-AA (AA = ayin sonu) PPP-P-MM-ДД / PPPP-MM (MM = кінець місяця) YYYY-MM-DD / YYYY-MM (MM =月末)
	Catalog number / Каталожен номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Kataloġu sszám / Numero di catalogo / Katalog nömrə / ကတ္လာဂုဏ် 번호 / Catalogo / numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер на каталог / Katalógové číslo / Kataloški broj / Katalog numerasi / Номер за каталогом / 目录号
	REF Catalog number / Каталожен номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Kataloġu sszám / Numero di catalogo / Katalog nömrə / ကတ္လာဂုဏ် 번호 / Catalogo / numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер на каталог / Katalógové číslo / Kataloški broj / Katalog numerasi / Номер за каталогом / 目录号
	EC REP Authorized Representative in the European Community / Оторизиран представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret repræsentant i De Europæiske Fællesskaber / Autorisierte Vertreter in der Europäischen Gemeinschaft / Εξουπούρημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Reprézentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Europskoj uniji / Meghatalmazott képviselő az Európai Közösségen / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастырындағы үекілдегі екін / 유럽 공동체의 위임 대표 / Igaliotasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autorisiert representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Représentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupce v Evropském společenství / Autorizovano predstaviňstvo v Evropskoy uniji / Auktoriserad representant i Europeiska gemenskapen / Avrupa Topluluğu Yetkilisi Temsilcisi / Упновоначеній представник в країнах ЄС / 欧洲共同体授权代表
	IVD In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин vitro / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro биохимияткі істрикі сүзгекі / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsinskaia aparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomaga za In Vitro Dijagnostiku / In vitro diagnostosztikai orvosi eszköz / Dispositivo medicaile per diagnostica in vitro / Жасанды жағдайда жүргізетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietaisais / Medicinas ierīces, ko lieto in vitro diagnostikā / Medische hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispositiv medical pentru diagnostic in vitro / Medicinskiy прибор для диагностики in vitro / Medicínska pomôcka na diagnostiku in vitro / Medicinsk uredaj za in vitro diagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diagnostik Tibbi Cihaz / Медицинский пристрой для диагностики in vitro / 体外診断医疗设备
	Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrenzung / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperatuuri piirang / Limites de température / Dozvoljena temperatura / Hörmérsékti határ / Limiti di temperatura / Температурны шектеу / 온도 제한 / Laikymo temperatūra / Temperatūras ierobežojumi / Temperaturlimit / Temperaturbegrenzung / Ограничение температуры / Limites de temperatura / Limite de temperatūr / Ограничение температуры / Ohranenie teploty / Ograniczenie temperature / Temperaturgräns / Sicaklık sınırlaması / Обмеження температури / 温度限制
	LOT Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Товарная карта / 배치 코드(로트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партии / 批号 (亚批)
	Contains sufficient for <n> tests / Съдържанието е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testimede jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> tesztzéhez elegendő / Contenuto sufficiente per <n> test / <n> тесттери чиңи жекелікті / <n> 테스트가 충분히 포함됨 / Pakankanak kieksitilki <n> test / Satur pietiekami <n> párbauděm / Inhoud voldoende voor "n" testen / Inholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Continut suficient pentru <n> teste / Достаточно для <n> тестов(а) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli malzemeler / Вистачить для аналізів: <n> / 足够进行 <n> 次检测
	Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήστης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasításf / Consultare le istruzione per l'uso / Пайдалану нұсқаудағы мен танысын алышыз / 사용 지침 참조 / Skaitykite naudojimo instrukcijas / Skafit lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcję użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používání / Pogledajte uputstvo za upotrebu / Se bruksanvisningen / Kullanımları'nı başvurun / Див. інструкції з використання / 请参阅使用说明
	Do not reuse / Не използвайте отново / Nepoužívajte opakovane / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte Kasutada korduvalla / Не pas réutiliser / Не користити поново / Egyszer használatós / Non riutilizzare / Пайдаланбаңыз / 제사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolositi / Не использовать повторно / Nepoužívajte opakovane / Ne upotrebljavajte ponovo / Får ej återanvändas / Tekrar kullanmayın / Не використовувати повторно / 请勿重复使用
	SN Serial number / Серийен номер / Sériové číslo / Serienummer / Serienummer / Σειριακός αριθμός / Nº de serie / Seerianumber / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Товарная номенклатура / 일련 번호 / Serijos numeris / Sériras numurs / Serie nummer / Numer seryjny / Número de serie / Număr de serie / Серийный номер / Seri numeralı / Homer cepit / 序列号

For IVD Performance evaluation only / Само за оценка качеството на работата на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD-Leistungsbewertungszwecke / Μόνο για αξιολόγηση σπέσιος IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réservez à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárólag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде», диагностикада тек жұмысты бағанап шын / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisys veikimo charakteristikoms tikrinti / Vientig IVD darbības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering av IVD-ytelse / Tylko do oceny wydajności IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики in vitro / Určené iba na diagnostiku in vitro / Samo za procenu učinku i u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Yalnızca IVD Performans değerlendirme için / Тільки для оцінювання якості діагностики in vitro / 仅限 IVD 性能评估

For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolni hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Като́теро óρio θερμοκρασίας / Límite inferior de temperatura / Alumne temperaturupirii / Limite inférieure de température / Najniža dovoljenja temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температуралыңқы төмөнгі руқсат шеги / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrense / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Минимальна температура / 温度下限

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Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

ATCC® is a trademark of American Type Culture Collection.

BD, the BD Logo, BACTEC, FOS, Luer-Lok, Safety-Lok, and Vacutainer® are trademarks of Becton, Dickinson and Company or its affiliates. © 2019 BD. All rights reserved.