Part Number:	500017021	BALTSO0191 Version 13.0 Template 4 Inserts
Category and Description: BD CytoRich™ Blue Preservative Package Insert	Rev from: 02	Rev to: 03
	Job Number:	692-18

Catalog Number: 491335, 491303, 491304, 491459

Blank (Sheet) Size: Length: 11" Width: 8.5"

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
Number of Pages: 12 Number of Sheets: 6

Page Size: Length: 11" Width: 8.5" Final Folded Size: 5.5"L X 8.5"W

Ink Colors: Number of Colors: 2 PMS #: 032 Red; PMS Standard Black

Printed Two Sides: Yes: ☒ No: ☐

Style (see illustrations below): # 1 (3-hole punched); Stapled top left corner

Vendor Printed: ☒ Online / In House Printed: ☐ Web Printed: ☐

See Specification control no. N/A for material information.

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Company confidential. This document is the property of Becton, Dickinson and Company and is not to be used outside the company without written permission. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Revised By: **REVISED BY**
By Tori Pagani at 9:35 am, Sep 11, 2018

Proofing Approved By: **PROOFING APPROVED BY**
By Terrence Means at 12:39 pm, Sep 11, 2018

Third Eye By: **THIRD EYE BY**
By Daria Kuznetsova at 12:57 pm, Sep 11, 2018

REF	491335	BD CytoRich™ Blue Preservative
REF	491303	BD PrepStain™ Non-GYN Test Kit
REF	491304	BD Totalys™ SlidePrep Non-GYN Test Kit
REF	491459	BD Non-GYN Stain Kit
English: pages 1 – 2 Deutsch: Seiten 4 – 5 Español: páginas 7 – 8 Français: pages 2 – 3 Italiano: pagine 5 – 6 Português: páginas 8 – 9		

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne miestni zástupce spoločnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyts lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőjétől. / Нұсқаулар үшін жергілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielem BD. / Contacte o reprezentante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указаний обратитесь к местному представителю компании BD. / Instrukcie ziskate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD

INTENDED USE

BD CytoRich™ Blue Preservative is an alcohol-based preservative liquid intended for preservation of cytology cells in suspension and their subsequent preparation for cytological evaluation.

SUMMARY AND EXPLANATION

Non-cervicovaginal cytologic screening involves the microscopic examination of cell samples that have been harvested from human body sources, including sputum, bronchial washings, breast tissue aspirations, spinal fluid, other fluids, exudates and fine needle aspirates of specific organs. These cell samples are usually fixed in an alcohol-based liquid (e.g. BD CytoRich Blue Preservative) and are concentrated with centrifugation. These samples are then either smeared or sedimented onto a glass microscope slide and stained by the Papanicolaou or other staining technique. BD CytoRich Blue Preservative is packaged in leak-resistant plastic containers. Its ingredients include alcohols and polyethylene glycol.

LIMITATIONS OF THE PROCEDURE

- Cytologic samples should be fixed in BD CytoRich Blue Preservative as soon as possible after collection.
- A sample which has become degraded prior to fixation may be unsatisfactory for cytologic examination.
- All supplies are intended for single use only and cannot be reused.

WARNINGS AND PRECAUTIONS

Cytologic specimens may contain infectious agents. Wear suitable protective clothing, gloves, and eye/face protection. Follow appropriate biohazard precautions when handling samples.

BD CytoRich Blue Preservative contains an aqueous solution of denatured ethanol. The mixture contains methanol and ethylene glycol. Do not ingest.

Danger

H225 Highly flammable liquid and vapor. **H371** May cause damage to organs. **P210** Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P241** Use explosion-proof electrical/lighting equipment. **P260** Do not breathe dust/fume/gas/mist/vapors/spray. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P240** Ground/bond container and receiving equipment. **P233** Keep container tightly closed. **P242** Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P264** Wash thoroughly after handling.

P270 Do not eat, drink or smoke when using this product. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P308+P311** IF exposed or concerned: Call a POISON CENTER or doctor/physician. **P405** Store locked up. **P403+P235** Store in a well-ventilated place. Keep cool. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

BD EA/OG Combo Stain contains ethanol, methanol and acetic acid. Do not ingest.

Danger

H225 Highly flammable liquid and vapor. **H312+H332** Harmful in contact with skin or if inhaled. **H370** Causes damage to organs.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P233** Keep container tightly closed. **P240** Ground/bond container and receiving equipment. **P241** Use explosion-proof electrical/lighting equipment. **P242** Use only non-sparking tools. **P243** Take precautionary measures against static discharge. **P260** Do not breathe dust/fume/gas/mist/vapors/spray. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P312** Call a POISON CENTER or doctor/physician if you feel unwell. **P321** Specific treatment (see on this label). **P362+P364** Take off contaminated clothing and wash it before reuse. **P370+P378** In case of fire: Use for extinction: CO₂, powder or water spray. **P403+P235** Store in a well-ventilated place. Keep cool. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

BD Hematoxylin Stain 0.50 contains ethylene glycol. Do not ingest.

Warning

H302 Harmful if swallowed. **H315** Causes skin irritation. **H319** Causes serious eye irritation. **H335** May cause respiratory irritation.

P261 Avoid breathing dust/fume/gas/mist/vapors/spray. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. **P330** Rinse mouth. **P302+P352** IF ON SKIN: Wash with plenty of soap and water. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P332+P313** If skin irritation occurs: Get medical advice/attention. **P337+P313** If eye irritation persists: Get medical advice/attention. **P403+P233** Store in a well-ventilated place. Keep container tightly closed. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

PRECAUTIONS

1. For *in vitro* diagnostic use.
2. For professional use only.
3. Good laboratory practices should be followed.
4. Avoid splashing or generating aerosols. Operators should use appropriate hand, eye/face and clothing protection.
5. Do not pipette by mouth.
6. Do not ingest (contains denatured alcohol).
7. Used supplies and product should be disposed of properly in accordance with institutional and governmental regulations.

MATERIALS REQUIRED

BD CytoRich Blue Preservative can be used with the BD PrepStain™ Slide Processor or BD Totalys™ SlidePrep or in a manual process. Refer to the Operator's Manual for the BD PrepStain Slide Processor or BD Totalys SlidePrep for complete information regarding staining reagents, components and accessories. Not all of the materials listed below are required for preparing slides manually.

Materials Provided

Reagents

- **BD CytoRich Blue Preservative**, 3600 mL

Materials Sold Separately by BD

Reagents

- **BD Non-GYN Stain Kit**
 - 1 – **BD Hematoxylin Stain 0.5**, 480 mL
 - 1 – **BD EA/OG Combo Stain**, 480 mL

Instruments, Equipment and Accessories

- **BD PrepStain Slide Processor**
- **BD Totalys SlidePrep**
- Easy Aspirator Kit
- Centrifuge and accessories
- **BD PrepMate™ Automated Accessory**
- **BD PrepStain Non-GYN Test Kit**
 - 192 – **BD Centrifuge Tubes**
 - 192 – **BD SurePath PreCoat Slides**
 - 192 – **BD Settling Chambers**
 - 192 – **BD PrepStain Transfer Tips**
- **BD Totalys SlidePrep Non-GYN Test Kit**
 - 192 – **BD Centrifuge Tubes**
 - 192 – **BD SurePath PreCoat Slides**
 - 192 – **BD Settling Chambers**
 - 192 – **BD Totalys Transfer Tips**

Materials Required But Not Provided By BD

- Specimen Collection Container
- Vortex Mixer
- Precision Pipettes with Disposable Tips
- Deionized Water (pH 7.5 to 8.5)
- Reagent Grade Alcohol
- Xylene or Xylene substitute
- Mounted Media
- Coverslips

STORAGE

1. The storage limit of the **BD CytoRich Blue Preservative** (without a patient specimen) is up to 24 months from date of manufacture at room temperature (15–30 °C).
2. Do not use fluid beyond the expiration date printed on the label.
3. The storage limit for **BD CytoRich Blue Preservative** with cytology sample should be determined in accordance with your laboratory's procedures.

PROCEDURE

- Label the specimen collection container before use.
- **BD CytoRich Blue Preservative** contains 50% w/v alcohol and should be used by mixing one (1) volume of sample with one (1) volume of **BD CytoRich Blue Preservative**.
- The minimum fixation time is 30 minutes.
- Cytologic preparations may be prepared using the **BD PrepStain Slide Processor**, **BD Totalys SlidePrep**, manual methods, cytocentrifugation methods, or smear techniques.

BIBLIOGRAPHY

1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881–906.
2. Barrett DL: Cytocentrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80–83.
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412–446.
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495.
5. Hajdu SI: A note on the history of Carbo wex. Acta Cytol 1983; 27:204–206.
6. Raistrick J, Shambayati B, Dunsmuir W. Collection fluid helps preservation in voided urine cytology. Cytopathology. 2008 Apr;19(2):111–7.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

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© 2018 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.
Benex Limited
Becton Dickinson Pty Ltd.