

BD Vacutainer® Sitratlı Tüpler, BD Vacutainer® CTAD Tüpler ve BD Vacutainer® ESR Tüpleri

BD Vacutainer® Sitratlı Tüpler, çift cidarlı tüp teknolojisi ile uçucu sitrat katkı maddesini tüp içerisinde tutarak doğru kan katkı maddesi oranının tüp ömrü boyunca teminini garanti eder. Minimum dolum çizgisi ile hassas koagülasyon testleriniz için daima doğru hacimde kan aldığınızdan emin olmanız için güvenilir bir rehberlik sağlar. Şimdi %3.2 ve %3.8 Sitrat konsantrasyon seçenekleri ile.

BD Vacutainer® CTAD Tüpleri ile heparin terapisi alan hastalarda bile hemostaz testlerinizden emin olun. İçerdiği sodium sitrat, teofilin, adenozin ve dipridamol karışımı ile *in-vitro* trombosit aktivasyonunun önüne geçer ve heparin terapisi alan hastalarda bile daha doğru koagülasyon test sonuçları almanızı sağlar.

BD Vacutainer® ESR Tüpleri, manuel sedimentasyon analizleri yada otomasyona uygun farklı tüp seçenekleri ile Westergren metoduna uygun sedimentasyon analiz sonuçları almanızı garanti eder.

LabNotes[†]

Volume 11 No.1, Summer 2002

A Newsletter
from BD
Vacutainer
Systems,
Preanalytical
Solutions

Blood Testing: Choosing the Right Specimen

By Jeffrey J. Chance, PhD

Blood testing is an essential component in the delivery of health care. Each day, healthcare workers collect millions of blood samples, and laboratories test for a wide range of analytes and biological parameters to aid in the diagnosis and monitoring of disease. Testing is generally performed on one of three different specimens — whole blood, plasma, or serum. There are numerous criteria that determine which specimen is most suitable for a particular test and setting.

Centralized Testing vs. Point-of-Care Testing

Today, commercially manufactured diagnostic tests are available for almost every known analyte and blood component. Certain types of tests require a whole-blood specimen, such as hematology, erythrocyte sedimentation rate, blood gases, tests for red cell constituents or surface antigens, and genetic testing. Some drugs, such as cyclosporine, exhibit variable distribution into erythrocytes in vitro and are also most reliably measured with a whole-blood specimen. Serum is generally the preferred specimen for chemistry testing, while plasma is the traditional specimen for coagulation testing. However, whole blood may also be used for these tests.

Generally speaking, whole blood can be considered a “universal specimen” since it contains every analyte, as well as the various types of blood cells. Moreover, advantages to testing whole blood include no waiting time for clotting and specimen centrifugation, and no additional cost associated with the purchase and operation of centrifuges.

But if whole blood can in theory


continued on page 3

IN THIS ISSUE

FEATURE:
Blood Testing:
Choosing the
Right Specimen

— 2 —
LabNotes
Best Practice:
Citrate
Concentrations

— 3 —
Introducing, New
BD Vacutainer
Plus Plastic
Citrate Tube

— 5 —
BD Awards 10
Clinical Laboratory
Scholarships

— 5 —
BD Education
Center

— 6 —
Order of Draw
for Glass and
Plastic Multiple
Tube Collections

[†] This publication is a service to the customers and friends of BD, and is designed only to provide general information. It is not intended to be comprehensive or provide any legal or medical advice.

Best Practice

Question: We just found out that BD makes two different citrate tube concentrations a 3.2% and 3.8%. We've been using both interchangeably. Can we do this and what are the differences in APTT and PT results?

The assumption is that you have collected a quality sample following proper handling conditions¹. Sodium citrate concentrations can have significant effects on APTT and PT assay results, especially when results are outside of the normal range and responsive reagents are used*. Laboratories should determine their normal range of APTT and PT based on one citrate concentration, and must consistently use this concentration for all patient samples until a new normal range is developed. NCCLS recommends the use of 3.2% citrate concentration².

Potential issues between 3.2% and 3.8% sodium citrate concentrations are as follows:

- When responsive reagents (ex. Dade®Actin® FS, Dade®Innovin®) are used, statistical differences in APTT and PT test results between the two citrate concentrations will occur.
- The PT test is consistently higher when responsive reagents and 3.8% sodium citrate are used.
- Normal ranges for APTT and PT may shift higher when 3.8% citrate is compared to 3.2% citrate with responsive reagents. Less variation in the normal ranges occurs between the citrate concentrations when nonresponsive reagents (ex. Dade®Actin®, Thromboplastin C•Plus) are used.
- When nonresponsive reagents are used, varying the citrate concentration has little clinical significance except with patients receiving IV heparin therapy*.

Normal PT and APTT Range – Citrate Effects				
Citrate Concentration	Dade®Actin® FS APTT	Dade® Innovin® PT	Dade®Actin® APTT	Dade® Thromboplastin C•Plus PT
3.2%	22-31	8.6-10.7	23-33	12-14
3.8%	24-33	9.2-11.4	22-31	11-14
p	<.001	<.001	Non-significant	Non-significant

*The reference article used for this bulletin conducted studies on the variability and interchangeability of 3.2% and 3.8% citrate concentrations on five populations: healthy volunteers, hospitalized patients not receiving anticoagulant therapy, patients receiving intravenous (IV) heparin therapy, or receiving both IV heparin and oral anticoagulant therapy, and outpatients receiving oral anticoagulant therapy. Please refer to this paper for information on these study groups³.

Reference:

1. Adcock, Dorothy M; Kressin; Marlar, Richard A PhD; *Preanalytical Variables in the Routine Coagulation Laboratory*. ASCP Teleconference Series Sep. 12, 2000; Program No. 6064
2. NCCLS – *Guidelines, Collection, Transport and Processing of Blood Specimens for Coagulation Testing and General Performance of Coagulation Assays*; Approved Guideline – Third Edition, H21-A3; Vol 18 No. 20; 5.2.1 December 1998.
3. Adcock, Dorothy M; Kressin; David C; Marlar, Richard A PhD; *Effect of 3.2% vs 3.8% Sodium Citrate Concentration on Routine Coagulation Testing*. Am J Clin Pathol. 1997; 107:105-110

Introducing BD Vacutainer™ Plus Plastic Citrate Tube



- **Innovative tube geometry** that *minimizes tube headspace and associated platelet activation* to optimize APTT monitoring of unfractionated heparin patients. You always get a full draw with standard outside tube dimensions, for 1.8mL and 2.7mL draw volumes.
- **Clinically equivalent performance** to the recognized global “Gold Standard”, the 4.5mL BD Vacutainer™ Glass Citrate Tube.
- **Clinically Proven** with multicenter clinical trials evaluating all routine coagulation tests on major patient populations, and utilizing widely used coagulation analytical systems.

continued from front cover

be used for all types of blood testing, then the question arises, why isn't it? Certainly, historical limitations and conventions have played a role in the current widespread use of serum and plasma. Assay development has long focused on serum-based methodologies due to the ease of handling, manipulation, and storage associated with the relatively clean matrix of serum. The chemical reactions that take place between the reagent and the target analyte are also easier to develop and optimize in a serum-based format. In recent years, many chemistry tests traditionally performed on serum have also become available for plasma.

While the turnaround time (TAT) is longer for tests that require a clotted and centrifuged specimen, this wait has been tolerable for routine analysis done in central laboratories. Furthermore, improvements in TAT have been realized with sample transport initiatives and the shortened time it

takes to perform analysis on modern analyzers. Consequently, serum and plasma have become de facto standards for many tests in the central lab, and the long-standing acceptance of these conventions has given diagnostic manufacturers little incentive to develop whole-blood assays for routine testing.

Although the technology now exists to perform most serum or plasma tests on whole blood, there are barriers other than instrumentation that make it difficult to accept this specimen type for many tests in the central lab. Over time, cell lysis in whole-blood specimens alters certain analyte concentrations such as potassium, lactate dehydrogenase, and phosphate, and cellular metabolism alters analytes like glucose and lactate. Consequently, whole-blood specimens require analysis within a limited time window to ensure accurate results for several common tests. Numerous other analytes exhibit some degree of reduced stability in whole blood, and clot formation and loss of specimen

integrity over time severely limit the storage of whole blood for delayed or repeat testing. Taken together, these constraints and conventions indicate that current practices in specimen choice for central lab testing will likely continue in the foreseeable future.

When testing is performed outside the central lab, the criteria used for specimen selection change considerably. In settings such as the patient bedside, critical care and emergency room (ER) satellite labs, operating rooms, and physician office labs (POLs), the results need to be available as quickly as possible in order to make decisions on patient management. As a result, convenience and short TAT are the most important factors for point-of-care testing (POCT). Because whole blood requires no processing and is available for immediate analysis, it is often the specimen of choice in such settings. Consequently, the aforementioned limitations regarding short testing windows and stability over time are less relevant.

continued on page 4

continued from page 3

The extent to which decentralized testing using whole blood might continue to expand remains to be seen. While the technology exists to perform most tests on whole-blood samples, the technology and test volumes associated with POCT tend to increase the cost per test—in some cases significantly—compared to centralized testing. In some institutions, a reduction in satellite laboratory personnel requirements and a shift to testing performed by nursing staff could possibly offset the cost burden. However, there may be limits to which nursing can accommodate increased testing duties, and the financing of new POCT supervisory positions may be necessary to ensure that quality assurance procedures are enforced. In institutions where centralized testing is available, it is clear that strong arguments based on medical, financial, and outcome studies will be needed to justify expansion of POCT. Perhaps the biggest hurdle to expansion of decentralized whole-blood testing is the consolidation and automation of core laboratories—powerful trends that tend to pull testing back to serum- or plasma-based high-volume analyzers.

Whole-Blood Tests at the Point of Care

As discussed above, most POCT uses whole blood, while tests conducted in the central laboratory utilize serum or plasma. The differences in sample matrix are not trivial as it turns out, because in some instances the results for certain tests are not equivalent.

One particularly good example is prothrombin time (PT) measurement, which is used to monitor patients on warfarin therapy in order to maintain an optimal therapeutic dose. While whole blood is arguably the more suitable specimen for coagulation assay because it contains plasma and platelets, plasma-based PT measurement remains the standard method. With the development of small whole-blood analyzers as a near-patient alternative to the conventional plasma

While whole blood is arguably the more suitable specimen for coagulation assay because it contains plasma and platelets, plasma-based PT measurement remains the standard method.

method, laboratories have encountered the challenge of demonstrating equivalence of international normalized ratio (INR) values between the two methods.

POCT for glucose has also received much attention. The question is not whether whole blood gives better results, but rather if more frequent testing can reduce the incidence of adverse outcomes. Whole blood is regarded as the most convenient specimen for diabetics who need to test themselves daily.

For both of these tests, passing the equivalency hurdle is essential for success. In the hospital

setting, strict equivalence with central lab methods—as defined by each institution—may be required to avoid having different reference ranges for the same test, which might confuse medical staff. Given the potential for negative sequelae due to incorrect INR results, self-testing using whole blood PT devices is being examined perhaps even more rigorously than it has been with home glucose testing.

Serum vs. Plasma in the Central Lab

The choice between serum and plasma for centralized chemistry testing is an area of great interest in terms of specimen selection. As noted earlier, serum has been the conventional standard for most chemistry tests, including routine, special chemistry, therapeutic drug monitoring (TDM), and tumor marker assays. One of the reasons for this convention—the historical development and availability of tests for serum only—is to an extent no longer applicable since plasma assays are now available for most common tests.




Today, the choice between serum and plasma is in large part a decision based on the unique requirements and priorities of an individual lab. When the priority is TAT, plasma has a clear advantage since it can be centrifuged immediately upon collection. However, in settings where the mean time between phlebotomy and specimen arrival at the laboratory exceeds recommended serum specimen clot times, or where the laboratory does not otherwise hold specimens, the advantages in TAT with plasma are not realized. Consequently, some institutions may limit plasma chemistry testing to settings where the specimen is immediately available for processing, such as an ER lab.

Labs may also find that serum samples lead to more instrument downtime than plasma samples as a result of fibrin formation. This claim needs to be verified by each laboratory, since specimen quality is highly dependent on adherence to recommended collection and handling conditions for both serum and plasma tubes. Aside from logistic considerations, another rationale for the use of plasma has been that it is free from several coagulation-induced changes in analyte concentrations, and more accurately reflects the composition of circulating blood.

A number of different anticoagulants are available for obtaining plasma samples. For chemistry testing, lithium heparin has been the preferred anticoagulant since both EDTA and

continued on page 7

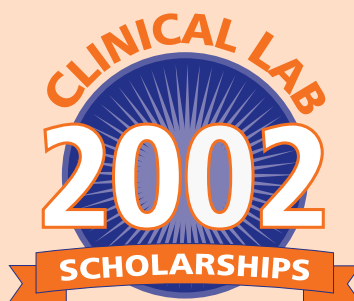

Ten individuals from laboratories and hospitals throughout the United States were recently selected as recipients of a Clinical Lab Scholarship, compliments of BD, inventor of the BD Vacutainer™ Blood Collection System.

The scholarship granted each recipient an all-expense paid trip to the 2002 CLMA/ASCP Convention in

New Orleans, Louisiana. Each recipient also received an exclusive invitation to attend a specially arranged dinner meeting with a keynote address by one of the industry's prominent clinicians and noted speakers, Mr. Thomas Dilts, MT (ASCP), MBPA, Director of Laboratory Operations at MCV Hospitals, Virginia Commonwealth University.

The ten scholarship recipients were selected at random from over 800 entries received via a special BD website for contest entry.

This educational opportunity is part of an on-going program from BD Vacutainer Systems, Preanalytical Systems to endorse the continuing education of medical professionals throughout the field. The BD Vacutainer™ family of products includes a complete system of resources and educational materials — from clinical data, to training programs, to in-servicing — all designed to help medical professionals get the best results possible.

Additional information on the Clinical Lab Scholarships program will be in your next issue of LabNotes, and on the web at www.bd.com/vacutainer.

The BD Education Center

BD has long recognized the necessity for the ongoing education of healthcare providers and their patients. This is especially apparent in today's environment, where technological advancements and governmental regulations continue to change the face of healthcare. Healthcare professionals require current information regarding products, techniques, and industry standards, and consumers increasingly search for information to help make educated decisions regarding their own healthcare or the care of other people in their lives.

The **BD Education Center** is a comprehensive resource for up-to-date educational materials focusing on a variety of human health topics. The **BD Education Center** offers carefully selected titles, made available through strategic alliances with leading professional associations and publishers, as well as titles and programs created and offered by BD.

For more information, go to www.bd.com/education or call **1.800.ALL.MEDIA** (1.800.255.6334)



NCCLS Order of Draw GUIDELINES

Under Procedures for the Collection of Diagnostic Blood Specimens by Venipuncture, the current approved NCCLS standard for Multiple Specimen Collection order of draw (7.13.3.2) recommends the following:

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







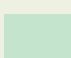

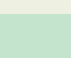

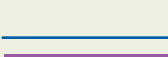
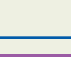



The following order-of-draw, which is recommended when drawing several specimens during a single venipuncture, is based on pragmatism. Its purpose is to avoid possible test result error due to cross contamination from tube additives. This procedure should be followed for both evacuated tubes and syringe transfer of blood to multiple tubes.

1. Blood culture tube
2. Plain tube, nonadditive (e.g., red stopper)
3. Coagulation tube (e.g., blue stopper)
4. Additive tubes:
 - Gel separator
 - Heparin (e.g., green stopper)
 - EDTA (e.g., lavender stopper)
 - Oxalate/fluoride (e.g., gray stopper)

Reproduced with NCCLS permission from their document H3-A4, "Procedures for the Collection of Diagnostic Blood Specimens by Venipuncture; Approved Standard – Fourth Edition" Vol.18, No.7, June 1998. Copies of the current edition may be obtained from NCCLS, 940 West Valley Road, Suite 1400, Wayne, PA 19087, USA. (610)688-0100. www.nccls.org


BD Vacutainer™ Order of Draw for Multiple Tube Collections

Designed for Your Safety

Closure Color		Collection Tube	Mix by Inverting
BD Vacutainer™ Glass Tubes	BD Vacutainer™ Plus Plastic Tubes		
		Blood Cultures – SPS	8 to 10 times
		Serum Tube (glass)	–
		Citrate Tube	3 to 4 times
 or 	 or 	BD SST™ Gel Separator Tube	5 times
		Serum Tube (plastic)	5 times
		Heparin Tube	8 to 10 times
 or 	 or 	BD PST™ Gel Separator Tube With Heparin	8 to 10 times
	 or 	EDTA Tube	8 to 10 times
		Fluoride (glucose) Tube	8 to 10 times

**Note: Always follow
your facility's protocol
for order of draw**

Handle all biologic samples and blood collection "sharps" (lancets, needles, luer adapters and blood collection sets) according to the policies and procedures of your facility. Obtain appropriate medical attention in the event of any exposure to biologic samples (for example, through a puncture injury) since they may transmit viral hepatitis, HIV (AIDS), or other infectious diseases. Utilize any built-in used needle protector if the blood collection device provides one. BD does not recommend resheathing used needles, but the policies and procedures of your facility may differ and must always be followed. Discard any blood collection "sharps" in biohazard containers approved for their disposal.

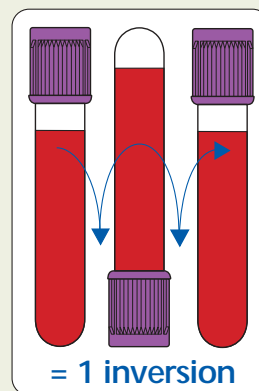

For copies of this document, contact your local BD Vacutainer representative or call

BD Technical Services
1.800.631.0174

BD Customer Service
1.888.237.2762

(ask for document VS5729-3)

continued from page 5

citrate bind calcium and have commonly-measured counterions (potassium, sodium). EDTA and citrate also cannot be used for enzymatic assays that require calcium as a cofactor, such as alkaline phosphatase. For accurate lactate determination, a fluoridated oxalate plasma specimen is often recommended to minimize glycolysis.

However, a number of analyte biases between serum and heparin plasma exist and must be assessed. Due to large positive biases, ammonia is typically not determined using serum. The clotting process also increases concentrations of potassium and inorganic phosphate, as well as activity of lactate dehydrogenase and alanine aminotransferase. However, the biases may be clinically acceptable or may be handled via an adjustment to the reference range. Total protein results are higher in plasma due to the presence of fibrinogen, and heparin itself may interfere with certain assays (e.g. amylase).

Many analyte biases are also highly analyzer-dependant. Published evaluations in the literature are few, and the assessment of equivalence and acceptability tends to be a very individual and variable process. In addition, storage stability of plasma for each analyte must be documented, since acceptable testing windows are smaller for many tests compared with serum. Thus, it is essential that each lab perform an independent evaluation when considering a switch to plasma testing.



It is interesting to note that the use of heparinized plasma is presently only a small fraction of that for serum. The percentage growth in plasma in recent years has, however, been slightly higher than with serum, and certain countries, particularly in Europe, tend to do considerably more plasma testing than others. Making the switch from serum to plasma is certainly feasible, but not without overcoming some hurdles. Given the workload involved in validation, laboratories need to make a very careful assessment of the benefits in order to ensure that any conversion to plasma testing makes sense for a given institution.

Future Trends in Blood-Based Testing

Choosing the right specimen is not merely a clinical decision, but a combination of clinical, financial,

technological, and logistical considerations. Where a choice exists, the choice of specimen will be closely aligned with other trends in diagnostic testing. POCT outcome studies, pneumatic tube systems, core laboratory consolidation and automation, and continued biosensor development all have the potential to influence decisions on the choice of specimen. Certainly, it will be important for laboratorians to monitor these trends. ■

Jeffrey J. Chance, PhD, is a senior clinical scientist with BD Vacutainer Systems, Preanalytical Solutions, in Franklin Lakes, NJ

SUGGESTED READINGS

- Doumas BT, Hause LL, Simuncak DM, Breitenfeld D. *Differences between values for plasma and*

continued on back cover

continued from page 7

- serum in tests performed in the Ektachem 700 XR Analyzer; and evaluation of plasma separator tubes (PST). Clin Chem 1989; 35:151-3.
- Guder WG, Narayanan S, Wisser H, et al. *Samples: from the patient to the laboratory: the impact of preanalytic variables on the quality of laboratory results.* Darmstadt, Germany: GIT Verlag, 1996: 1-149.
 - Harr R, Bond L, Trumbull D. *A comparison of results for serum versus heparinized plasma for 30 common analytes.* Lab Med 1987; 18:449-55.

- Narayanan S. *The preanalytic phase. An important component of laboratory medicine.* Am J Clin Pathol 2000; 113:429-52.
- NCCLS. Document H1-A4: *Evacuated Tubes and Additives for Blood Specimen Collection, Fourth Edition; Approved Standard.* Wayne, Pa.: NCCLS, 1996.
- Sainato D. *POCT moves to coagulation.* Clin Lab News 2001; 27(1):1,8.

Working Group on Preanalytical Variables of the German Society for Clinical Chemistry and the German Society for Laboratory Medicine.
Serum, plasma or whole blood?
Which anticoagulants to use?

- *Recommendations of the Working Group on Preanalytical Variables of the German Society for Clinical Chemistry and the German Society for Laboratory Medicine.* J Lab Med 1998; 22:297-312.

Reprinted with permission from Clinical Laboratory News, June 2001, published by AACCC, Washington, DC.

LAB NOTES is published by BD Vacutainer Systems, *Preanalytical Solutions*, Franklin Lakes, NJ, providing the laboratory community with laboratory products, including the BD Vacutainer™ Brand System, the BD Unopette Brand System, BD Microtainer™ Brand Tubes, BD Microtainer™ Safety Flow Lancets, BD Vacutainer™ Safety-Lok Needle Holder, BD Vacutainer™ Safety-Lok Blood Collection Set, BD Vacutainer™ Needle Disposal Container and BD Eclipse™ Blood Collection Needle.

Address all correspondence to: Leslie Magee, Associate Editor, LAB NOTES, BD Vacutainer Systems, *Preanalytical Solutions*, 1 Becton Drive MC325, Franklin Lakes, NJ 07417-1885.

LabNotes

Volume 11 No.1, Summer 2002

A Newsletter
from BD
Vacutainer
Systems,
*Preanalytical
Solutions*

IN THIS ISSUE

Blood Testing: Choosing the Right Specimen

LabNotes Best Practice:
Citrate Concentrations

Introducing, New BD Vacutainer
Plus Plastic Citrate Tube

BD Awards 10 Clinical
Laboratory Scholarships

BD Education Center

Order of Draw for Glass and
Plastic Multiple Tube Collections


BD Vacutainer Systems, *Preanalytical Solutions*
1 Becton Drive, Franklin Lakes, NJ 07417

BD Technical Services: 800.631.0174

Dade, Actin and Innovin are trademarks of Dade Behring Inc.
BD, BD Logo and all other trademarks are the property of
Becton, Dickinson and Company. © 2002 BD.
Printed in USA www.bd.com/vacutainer 08/02 VS7105



TRUST BD.
FOR THINGS YOU
CAN RELY ON.

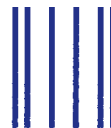

BUSINESS REPLY MAIL

FIRST CLASS MAIL PERMIT NO. 319, FRANKLIN LAKES, NJ

BD Vacutainer Systems
Preanalytical Solutions
LabNotes

1 Becton Drive, MC304
Franklin Lakes, NJ 07417-9935

FPO

Your feedback is appreciated!

(Check all that apply)

☐ **YES, I would like a complimentary subscription to LabNotes.
Please add me to the LabNotes mailing list.**

Last Name First Name MI

Job Title Facility # of Beds

Street Address

City State Zip

E-mail

Phone Fax

☐ Please have a BD sales representative call me.

☐ Please remove my name from the LabNotes mailing list.

We welcome your comments. Please describe specific subject matter that you would like to see covered in future issues of LabNotes:

Would you be interested in receiving LabNotes electronically?

☐ YES, please send me LabNotes via e-mail when it becomes available.

☐ YES, Please send me e-mail updates on news and information related to blood collection, BD products, services and promotions.



To continue receiving LabNotes by mail, please send us this postage-paid reply card

You can place your trust in BD Vacutainer Systems, Preanalytical Solutions to provide meaningful opportunities for educational learning experiences. Another valuable service provided to you from BD.