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IRODALOMJEGYZÉK

1. Clinical and Laboratory Standards Institute. 2014. Approved guideline M29-A4. Protection of laboratory workers from occupationally acquired infections, 4th ed., CLSI, Wayne, Pa.
2. U.S. Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030. 1991. Occupational exposure to bloodborne pathogens. Federal Register 56:64175–64182.
3. Clinical and Laboratory Standards Institute. 2015. Approved standard M07-A10. Methods for dilution antimicrobial susceptibility tests for bacteria that grow aerobically, 10th ed., CLSI, Wayne, Pa.

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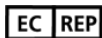
For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolni hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiiri / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураны т мөңгі ру сат шері / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrænse / Dolna granica temperatury / Limite mínimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Мінімальна температура / 温度下限

Control / Контроль / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Controlo / Ба ылау / 컨트롤 / Kontrolé / Kontrolė / Controlo / Controlo / Контроль / kontrol / Контроль / 对照

Positive control / Положителен контрол / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Pozitiv kontroll / Controllo positivo / О ба ылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívna kontrolė / Positieve controle / Kontrola dodatnia / Controllo positivo / Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивный контроль / 阳性对照试剂

Negative control / Отрицателен контрол / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negativna kontrola / Negativ kontroll / Controllo negativo / Негативтік ба ылау / 음성 컨트롤 / Neigiama kontrolė / Negatívna kontrolė / Negative controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативный контроль / 阴性对照试剂

Method of sterilization: ethylene oxide / Метод на стерилизация: етиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseringmetode: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizácie: etilen oksid / Sterilizálás módszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Стерилизация дісі – этилен тоты ы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksīds / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringmetode: etylenoksid / Metoda sterilizacji: tienek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metóda sterilizácie: etylénoxid / Metoda sterilizacije: etilen oksid / Steriliseringmetode: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом / 灭菌方法: 环氧乙烷

Method of sterilization: irradiation / Метод на стерилизация: ирадиация / Způsob sterilizace: záření / Steriliseringmetode: bestråling / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseringmetode: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metodo di sterilizzazione: irradiazione / Стерилизация дісі – с улет сирү / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringmetode: bestråling / Metoda sterilizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизации: облучение / Metóda sterilizácie: ožiarovanie / Metoda sterilizacije: ozračevanje / Steriliseringmetode: strålning / Sterilizasyon yöntemi: irradyasyon / Метод стерилизації: опроміненням / 灭菌方法: 辐射

Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Bioförfärdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologilised riskid / Risques biologiques / Biološki rizik / Biológiallag veszélyes / Rischio biologico / Биологичны т уекепдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biyojik Riskler / Билогічна небезпека / 生物学风险

Caution, consult accompanying documents / Внимание, направете справка в придружаващите документи / Pozor! Prostudujte si příloženou dokumentaci! / Forsiktig, se ledsagende dokumenter / Achtung, Begleiddokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatus! Lugeda kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Upozorenje, koristiti prateću dokumentaciju / Figyelem! Olvassa el a mellékeltájékoztatót / Attenzione: consultare la documentazione allegata / Абайна ыз, тиісті жаттармен танысы ыз / 주의, 동봉된 설명서 참조 / Dėmesio, žiūrėkite priedamą dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Diikkat, birlikte verilen belgelerle başvurun / Увага: див. супутню документацію / 小心，请参阅附带文档。

Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiiri / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураны ру сат етілген жо ар ы шері / 상한 온도 / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrænse / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sicaklık üst sınırı / Максимальна температура / 温度上限

Keep dry / Пазете сухо / Skladujte v suchém prostředí / Opbevarer tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Ра к йінде ста / 건조 상태 유지 / Laikykite sausai / Uzglabāt sausu / Droog houden / Hoides tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať v suchu / Držite na suvom mestu / Förvaras torrt / Kuru bir şekilde muhafaza edin / Берегти від вологи / 请保持干燥

Collection time / Време на събиране / Čas odběru / Opsamlingsstidspunkt / Entnahmehzeit / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жынау ы ыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colecta / Ora colectării / Время сбора / Doba odboru / Vreme prikupljanja / Uppsamlingstid / Toplama zaman / Час забору / 采集时间

Peel / Обенете / Otevíte zde / Abn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / сти ги абатын алып таста / 벗 기기 / Plešti čia / Aftimēt / Schillen / Trekk av / Oderwać / Destacar / Se dezlipeste / Отклеить / Odrhňte / Oljušiti / Dra isår / Айрма / Відклеїти / 撕下

Perforation / Перфорация / Perforace / Perforering / Διείληξη / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Тесик тесу / 절취선 / Perforacija / Perforácia / Perforatie / Perforacia / Perfuração / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация / 穿孔

Do not use if package damaged / Не използвайте, ако опаковката е повредена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packung nicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιό. / No usar si el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használja, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Erep paket b zyl an bolca, pайдaлaнбa / 패키지가 손상된 경우 사용 금지 / Jei pakuotė pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Nào usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не използвайте при повреждени упаковки / Nepoužívejte, ak je obal poškozený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包装破损, 请勿使用

Keep away from heat / Пазете от топлина / Nevystavujte přilišnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Сап ын жерде са та/ 열을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / Не награвать / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Isidan uzak tutun / Берегти від дії тепла / 请远离热源

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Made in USA

Becton, Dickinson and Company
7 Loveton Circle
Sparks, Maryland 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:
Becton Dickinson Pty Ltd.
66 Waterloo Road
Macquarie Park NSW 2113
Australia

New Zealand Sponsor:
Becton Dickinson Limited
14B George Bourke Drive
Mt. Wellington Auckland 1060
New Zealand

ATCC® is a trademark of American Type Culture Collection.

BD, the BD Logo, and Phoenix are trademarks of Becton, Dickinson and Company or its affiliates. © 2020 BD. All rights reserved.