NAMENA

Panel BD Phoenix™ PMIC/ID koristi se za brzu identifikaciju i testiranje osetljivosti većine aerobnih i ponekih anaerobnih gram-pozitivnih bakterija ljudskog porekla pomoću odabranih antimikrobnih agenasa (celu listu vrsta možete pronaći u Priručniku za korisnike sistema BD Phoenix). Ovaj panel treba koristiti samo sa automatizovanim mikrobiološkim sistemom BD Phoenix.

PRINCIPI PROCEDURE

Pogledajte Priručnik za korisnike sistema BD Phoenix.

SADRŽAJ PANELE

Identifikacija: Pogledajte Priručnik za korisnike sistema BD Phoenix.

Osetljivost: Panel PMIC/ID sadrži antimikrobne agense i koncentracije u razblaženim rastvorima za dupliranje koji se nalaze u tabeli 1.

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25 panela

50 zatvarača panela

ČUVANJE

Čuvajte na temperaturi od 15 °C do 25 °C. Ne koristite panel ako je kesica probušena ili otvarana. Ne koristite panel ako nema desikanta ili ako je kesica desikanta pocepana.

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Pogledajte Priručnik za korisnike sistema BD Phoenix.

REFERENCE

1. Clinical and Laboratory Standards Institute. 2014. Approved guideline M29-A4. Protection of laboratory workers from occupationally acquired infections, 4th ed., CLSI, Wayne, Pa.
2. U.S. Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030. 1991. Occupational exposure to bloodborne pathogens. Federal Register 56:64175–64182.
3. Clinical and Laboratory Standards Institute. 2015. Approved standard M07-A10. Methods for dilution antimicrobial susceptibility tests for bacteria that grow aerobically, 10th ed., CLSI, Wayne, Pa.

Tehnička služba: obratite se lokalnom predstavniku kompanije BD ili bd.com.

Samo za EU: korisnici treba da prijave proizvođaču i državnom nadležnom organu sve ozbiljne incidente u vezi sa uređajem.

Sadržaj panela		Organizmi za kontrolu kvaliteta i očekivani rezultati						
Antimikrobik	Kôd	Konc. raspon (µg/ml)	<i>S. aureus</i> ATCC 29213	<i>E. faecalis</i> ATCC 29212	<i>E. faecalis</i> ATCC 51299	<i>S. aureus</i> ATCC 25923	<i>S. aureus</i> ATCC BAA-976	<i>S. aureus</i> ATCC BAA-977
Amikacin	AN	4–16	1–8	–	–	–	–	–
Ampicilin	AM	2–8	0,5 – >1	0,5–2	–	–	–	–
Cefoksitin	FOX	2–8	≤1–4	–	–	–	–	–
Ceftarolin	CPT	0,125–1	0,125–0,5	0,25–2	–	–	–	–
Ciprofloksacin	CIP	0,25–4	≤0,125–0,5	0,5–4	–	–	–	–
Klindamicin	CC	0,25–1	≤0,125–0,25	4 – >8	–	–	–	–
Eritromicin	E	0,25–2	0,25–1	–	–	–	–	–
Fosfomicin	FF	16–64	≤8	32–128	–	–	–	–
Fusidinska kiselina	FA	1–8	≤0,5–1	1–4	–	–	–	–
Gentamicin	GM	1–4	≤0,5–1	–	–	–	–	–
Sinergija sa gentamicinom	GMS	500	–	≤500	>500	–	–	–
Test otpornosti na makrolide koja se može indukovati (IMLSb)*	ECC	DA	NEG	–	–	–	NEG	POZ
Linezolid	LZD	0,5–4	1–4	1–4	–	–	–	–
Beta-laktamaza (zasnovana na nitrocefenu)	NCF	DA	POZ	–	–	NEG	–	–
Moksifloksacin	MXF	0,25–1	≤0,125	≤0,125–0,5	–	–	–	–
Mupirocin	MUP	1–4	≤0,5	>8	–	–	–	–
Visok nivo mupirocina	MUH	256	≤256	≤256	–	–	–	–
Nitrofurantoin	FM	16–64	≤16–32	≤16	–	–	–	–
Oksacilin	OX	0,25–2	0,125–0,5	>4	–	–	–	–
Penicilin	P	0,0625–0,25	0,25 – >1	1–4	–	–	–	–
Rifampin	RA	0,25–1	≤0,25	–	–	–	–	–
Teikoplanin	TEC	1–4	≤0,5–1	≤0,5	–	–	–	–
Tetraciklin	TE	0,5–2	≤0,5–1	8 – >16	–	–	–	–
Tobramicin	NN	1–4	≤1	–	–	–	–	–
Trimetoprim	TMP	1–4	1–4	≤0,5–1	–	–	–	–
Trimetoprim/sulfametoksazol (SXT)	SXT	1/19–4/76	≤0,5/9,5	≤0,5/9,5	–	–	–	–
Vankomicin	VA	0,5–4	≤0,5–2	1–4	–	–	–	–

* Kao negativna kontrola za kontrolu kvaliteta CLSI mogu se koristiti ili *S. aureus* ATCC 29213 ili *S. aureus* ATCC BAA-976.

Tabela 2

Organizam	Nije prijavljeno	Druga metoda
–	–	–

Tabela 3

Organizam	Nije prijavljeno	Druga metoda
Vrste <i>stafilokoka</i> negativne na koagulazu osim vrste <i>S. epidermidis</i>	Ampicilin, penicilin	–
<i>Staphylococcus epidermidis</i>	Ampicilin	–
Rod <i>Enterococcus</i> osim vrsta <i>E. faecalis</i> ili <i>E. faecium</i>	Ciprofloksacin	–
Sve vrste iz roda <i>Enterococcus</i>	Rifampin	–
<i>Enterococcus faecalis</i>	–	Eritromicin

Istorija promena

Revizija	Datum	Rezime promena
03	2020-04	Odštampano uputstvo za upotrebu je pretvoreno u elektronski oblik i dodati su podaci za pristup za pribavljanje dokumenta sa veb lokacije bd.com/e-labeling. Dodate adrese sponzora za Australiju i Novi Zeland.

Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbricante / Atkaryшы / 제조업체 / Gamintojas / Ražotājs / Tilvirkir / Producent / Producător / Производителъ / Výrobca / Proizvodač / Tillverkare / Üretici / Виробник / 生产厂商

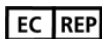
Use by / Използвайте до / Spotřebujte do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейин пайдаланура / Naudokite iki / Izljetot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použít do / Upotrebiti do / Använd före / Son kulanma tarihi / Використати до / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = slutning af måned)
JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)
EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)
AAAA-MM-DD / AAAA-MM (MM = fin del mes)
AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)
AAAA-MM-JJ / AAAA-MM (MM = fin du mois)
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)
ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja)
AAAA-MM-GG / AAAA-MM (MM = fine mese)
ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА / (АА = айдың соңы)
YYYY-MM-DD/YYYY-MM (MM = 월말)
MMMM-MM-DD / MMMM-MM (MM = mēnesio pabaiga)
GGGG-MM-DD/GGGG-MM (MM = mēneša beigas)
JJJJ-MM-DD / JJJJ-MM (MM = einde maand)
AAAA-MM-DD / AAAA-MM (MM = slutten av måneden)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = fim do mês)
AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)
ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
GGGG-MM-DD / GGGG-MM (MM = kraj meseca)
AAAA-MM-DD / AAAA-MM (MM = slutet av månaden)
YYYY-AA-GG / YYYY-AA (AA = ayın sonu)
PPPP-MM-DD / PPPP-MM (MM = кінець місяця)
YYYY-MM-DD / YYYY-MM (MM = 月末)

Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalogszám / Numero di catalogo / Каталог номер / 카탈로그 번호 / Katalog / numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目录号

Authorized Representative in the European Community / Оторизирани представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret representant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Représentante autorisé en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizirani predstavnik u Europskoj uniji / Meghatalmazott képviselő az Európai Községségben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті өкіл / 유럽 공동체의 위임 대표 / Įgalotasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Reprezentante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Avrupa Topluluğu Yetkili Temsilcisi / Уповноважений представник у країнах ЄС / 欧洲共同体授权代表

In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiiniparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnosztikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасаңды жағдайда жүргізетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietaisai / Medicinas ierices, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispozitiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomôcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Dijagnostik Tibbi Cihaz / Медицинский прибор для диагностики in vitro / 体外诊断医疗设备

Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegränsning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperatuuri piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температуры шектеу / 온도 제한 / Laikymo temperatūra / Temperatūras ierobežojumi / Temperatuurilimiet / Temperaturbegränsning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ochraničenie teploty / Ograničenje temperature / Temperaturgräns / Sıcaklık sınırlaması / Обмеження температури / 温度限制

Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (lot) / Codice batch (lotto) / Топтама коды / 배치 코드(로트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (lot) / Parti Kodu (lot) / Код партии / 批号 (亚批)

Contains sufficient for <n> tests / Съдържа много е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenuto sufficiente per <n> test / <n> тесттері үшін жеткілікті / <n> 테스트가 충분히 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor "n" testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Conținut suficient pentru <n> teste / Достаточно для <n> тестов(а) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli miktarda içerir / Вистачить для аналізів: <n> / 足够进行 <n> 次检测

Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нускаулығымен танысып алыңыз / 사용 지침 참조 / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se bruksanvisningen / Kullanim Talimatları'na başvurun / Див. інструкції з використання / 请参阅使用说明

Do not reuse / Не използвайте отново / Nepoužívejte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egyszer használatos / Non riutilizzare / Пайдаланбаңыз / 재사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolosiți / Не использовать повторно / Nepoužívať opakovane / Ne upotrebľavajte ponovo / Får ej återanvändas / Tekrar kullannayin / Не використовувати повторно / 请勿重复使用

Serial number / Серийн номер / Sériové číslo / Seriennummer / Seriennummer / Σειριακός αριθμός / N° de serie / Seerianumber / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Топтамалық нөмірі / 일련 번호 / Serijos numeris / Sérijas numurs / Serie nummer / Numer seryjny / Número de série / Număr de serie / Серийный номер / Seri numarası / Номер серії / 序列号

For IVD Performance evaluation only / Само за оценка качеството на работа на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD- Leistungsbewertungszwecke / Μόνο για αξιολόγηση απόδοσης IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réserve à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárólag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде» диагностикада тек жұмысты бағалау үшін / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisų veikimo charakteristikoms tikrinti / Vienigi IVD darības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering av IVD-ytelse / Tylko do oceny wydajności IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики in vitro / Určené iba na diagnostiku in vitro / Samo za procenu učinka u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Yalnızca IVD Performans değerlendirmesi için / Тільки для оцінювання якості діагностики in vitro / 仅限 IVD 性能评估

For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolni hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiiri / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrense / Dolna granica temperatury / Limite mínimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Мінімальна температура / 温度下限

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Made in USA

Becton, Dickinson and Company
7 Loveton Circle
Sparks, Maryland 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:
Becton Dickinson Pty Ltd.
66 Waterloo Road
Macquarie Park NSW 2113
Australia

New Zealand Sponsor:
Becton Dickinson Limited
14B George Bourke Drive
Mt. Wellington Auckland 1060
New Zealand

ATCC® is a trademark of American T type Culture Collection.

BD, the BD Logo, and Phoenix are trademarks of Becton, Dickinson and Company or its affiliates. © 2020 BD. All rights reserved.