

BD Phoenix NMIC-502

Srpski

NAMENA

Panel **BD Phoenix** NMIC koristi se za testiranje osetljivosti većine aerobnih i ponekih anaerobnih gram-negativnih bakterija ljudskog porekla pomoću odabranih antimikrobnih agenasa (celu listu vrsta možete pronaći u Priručniku za korisnike **BD Phoenix** sistema). Ovaj panel treba koristiti samo sa automatizovanim sistemom za mikrobiološka ispitivanja **BD Phoenix**.

PRINCIPI PROCEDURE

Pogledajte Priručnik za korisnike sistema **BD Phoenix**.

SADRŽAJ PANELE

Osetljivost: NMIC panel sadrži antimikrobne agense i koncentracije u razblaženim rastvorima za dupliranje koji se nalaze u tabeli 1.

Potvrđujući test za detekciju CPO: koristi meropenem, doripenem, temocilin i kloksacilin, samostalno i u kombinaciji sa različitim helatorima i inhibitorima beta-laktamaze u količinama potrebnim za detekciju i klasifikaciju organizama koji proizvode karbapenemaze.

ČUVANJE

Čuvajte na temperaturi od 15 °C do 25 °C. Ne koristite panel ako je kesica probušena ili otvarana. Ne koristite panel ako nema desikanta ili ako je kesica desikanta pocepana.

Paneli se moraju upotrebiti u roku od 2 sata po vađenju iz kesice.

Mere opreza:

Za *in vitro* dijagnostičku upotrebu.

UPOZORENJA

Držite se ustanovljenih mera opreza protiv mikrobioloških opasnosti u toku svih procedura. Pri rukovanju svim predmetima koji su kontaminirani uzorcima i mikroorganizmima treba se pridržavati „Standardnih mera predostrožnosti“^{1,2} i ustanovljenih uputstava. Pre bacanja u otpad, sterilizite kontejnere za uzorke i drugi inokulisani materijal obradom u autoklavu.

KONTROLA KVALITETA

Osetljivost: Pogledajte očekivane rezultate u tabeli 1.

NAPOMENA: Rezultat kontrole kvaliteta može biti prijavljen kao manji ili jednak najnižoj koncentraciji, ili kao veći od najviše koncentracije antimikrobika.

OGRANIČENJA PROCEDURE

Klinički značaj specifične kombinacije antimikrobika i organizma određuje se prema delovanju tog antimikrobika na taj organizam i prema tome da li je taj antimikrobik indikovao za lečenje bolesti koja se dovodi u vezu sa tim organizmom. Sistem **BD Phoenix** daje rezultate za kombinacije bile one klinički značajne ili ne. U globalu, rezultati koji se dobijaju pomoću sistema **BD Phoenix** bolji su od CLSI referentne metodologije mikrodilucije bujona;³ međutim, tabela 2 sadrži klinički značajne kombinacije koje nisu zadovoljile stroge standarde kompanije BD tokom kliničkih evaluacija, pa se ili ne prijavljuju ili se za potvrdu rezultata preporučuje druga metoda. Slično, tabela 3 sadrži kombinacije koje nisu klinički značajne, pa se ili ne prijavljuju ili se za potvrdu rezultata preporučuje druga metoda.

FUNKCIONALNE KARAKTERISTIKE

Pogledajte Priručnik za korisnike sistema **BD Phoenix**.

REFERENCE

1. Clinical and Laboratory Standards Institute. 2014. Approved guideline M29-A4. Protection of laboratory workers from occupationally acquired infections, 4th ed., CLSI, Wayne, Pa.
2. U.S. Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030. 1991. Occupational exposure to bloodborne pathogens. Federal Register 56:64175-64182.
3. Clinical and Laboratory Standards Institute. 2015. Approved standard M07-A10. Methods for dilution antimicrobial susceptibility tests for bacteria that grow aerobically, 10th ed., CLSI, Wayne, Pa.

Tehnička služba: obratite se lokalnom predstavniku kompanije BD ili www.bd.com.

ATCC is a trademark of the American Type Culture Collection.

© 2018 BD. BD, the BD Logo and BD Phoenix are trademarks of Becton, Dickinson and Company.

 Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

 Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:
Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia



MADE IN USA
L011105(02)
2018-01

Antimicrobic / Антимикробен агент / Zaštita od mikroba / Antimikrobiální činidlo / Antimikrobe / Antimikroob / Antimicrobien / Antibiotikum / Αντιμικροβιακό / Antibiotico / Микробқа қарсы зат / Antimikrobinis vaistas / Antimikrobiell agens / Antybiotyki / Antimicrobiano / Antibiotic / Антибиотик / Antimikrobik / Antimikrobiálna látka / Antimikrobikum / Противомикробный агент	(µg/mL)	<i>E. coli</i> ATCC® 25922	<i>P. aeruginosa</i> ATCC 27853	<i>E. coli</i> ATCC 35218	<i>K. pneumoniae</i> ATCC 700603	<i>K. pneumoniae</i> ATCC BAA-1705™
Amikacin	AN	4 - 16	≤0.5 - 4	1 - 4		
Amoxicillin-Clavulanate (f)	AXC	2/2 - 32/2	2/2 - 8/2	4/2 - 16/2		
Ampicillin	AM	2 - 8	2 - 8			
Aztreonam	ATM	1 - 16	≤0.25	2 - 8		
Cefepime	FEP	1 - 16	≤0.125	0.5 - 4		
Cefixime	CFM	0.5 - 2	0.25 - 1	≥8		
Ceftazidime	CAZ	0.5 - 8	≤ 0.5	1 - 4		
Ceftazidime-Avibactam	CZA	0.25/4 - 8/4	≤0.25/4 - 0.5/4	0.5/4 - 4/4	≤ 0.25/4	≤0.25/4 - 2/4
Ceftriaxone	CRO	0.5 - 4	≤0.5	8 - 64		
Cefuroxime	CXM	2 - 8	2 - 8			
Cephalexin	CN	4 - 16	4 - 16			
Ciprofloxacin- Other than <i>Salmonella</i> species	CIP	0.125 - 1	≤0.125			
Ciprofloxacin- <i>Salmonella</i> species		0.0625 - 1		0.25 - 1		
Colistin	CL	0.5 - 2	≤0.5 - 2	≤0.5 - 4		
Ertapenem	ETP	0.25 - 1	≤0.0625	2 - 8		
Fosfomycin	FF	16 - 128	≤16	≤16		
Gentamicin	GM	1 - 4	≤0.5 - 1	≤0.5 - 2		
Imipenem	IPM	0.25 - 8	≤0.0625 - 0.25	1 - 4		
Levofloxacin	LVX	0.5 - 2	≤0.25	0.5 - 4		
Mecillinam	MEC	2 - 8	≤0.5			
Meropenem	MEM	0.125 - 8	≤0.125	0.25 - 1		
Nitrofurantoin	FM	16 - 64	≤ 8 - 16			
Piperacillin	PIP	4 - 64	1 - 4	1 - 8		
Piperacillin-Tazobactam	TZP	4/4 - 64/4	1/4 - 4/4	1/4 - 8/4	≤0.5/4 - 2/4	
Temocillin	TEM	4 - 32	≥ 8	>32		
Tigecycline	TGC	0.5 - 2	≤0.25			
Tobramycin	NN	1 - 4	0.25 - 1	0.25 - 1		
Trimethoprim-Sulfamethoxazole	SXT	1/19 - 4/76	≤0.5/9.5	8/152 >16/304		
CPO detect	CPO3	-	NEG			POS
ESBL	ESBL	-	NEG		POS	
Cefotaxime/Clavulanate (ESBL)	CCX	<9	N/A		N/A	
Ceftazidime/Clavulanate (ESBL)	CCZ	<9	N/A		N/A	
Cefpodoxime-proxetil (ESBL)	CPD	<9	N/A		N/A	
Ceftazidime (ESBL)	CAZ	<9	N/A		N/A	
Ceftriaxone/Clavulanate (ESBL)	CCR	<9	N/A		N/A	

Table 2

Organism / Микроорганизъм / Organizam / Organismus / Organisme / Οργανισμός / Mikroorganizmus / Microorganismo / Организм / Mikroorganizmas / Drobnoustroj / Microorganism / Микроорганизм / Organizmus / Microorganismo / Mikroorganism / Organizma / Микроорганизм	Not Reported / Неотчетени / Nije zabilježeno / Nenahlášeno / Ikke rapporteret / Esitamata / Non rendu / Nicht berichtet / Δεν αναφέρεται / Nincs eredmény / Non riportato / Берілмеген / Nepranešta / Ikke rapportert / Nie zgłoszono / Não reportado / Nu sunt raportate / Не указано / Nisu prijavljeni / Neuvádza sa / No informado / Ej rapporterad / Raportlanmadı / Не включаться до звіту	Alternate Method / Альтернативен метод / Drugačija metoda / Alternativní metoda / Alternativ metode / Alternativne metod / Autre méthode / Alternative Methode / Εναλλακτική μέθοδος / Alternatív módszer / Metodo alternativo / Балама әдіс / Alternatyvus metodos / Metoda zamienna / Método alternativo / Metodă alternativă / Альтернативный метод / Druga metoda / Alternativa metoda / Alternativ metod / Alternativ Yöntem / Альтернативный метод
<i>Hafnia alvei</i>	Temocillin	
<i>Morganella morganii</i>	Piperacillin, Temocillin	
<i>Providencia</i> species except <i>P. stuartii</i>	Ceftazidime-Avibactam	
<i>Providencia</i> species	Temocillin	
<i>Pseudomonas aeruginosa</i>		Piperacillin

Table 3

Organism / Микроорганизъм / Organizam / Organismus / Organisme / Οργανισμός / Mikroorganizmus / Microorganismo / Организм / Mikroorganizmas / Drobnoustroj / Microorganism / Микроорганизм / Organizmus / Microorganismo / Mikroorganism / Organizma / Микроорганизм	Not Reported / Неотчетени / Nije zabilježeno / Nenahlášeno / Ikke rapporteret / Esitamata / Non rendu / Nicht berichtet / Δεν αναφέρεται / Nincs eredmény / Non riportato / Берілмеген / Nepranešta / Ikke rapportert / Nie zgłoszono / Não reportado / Nu sunt raportate / Не указано / Nisu prijavljeni / Neuvádza sa / No informado / Ej rapporterad / Raportlanmadı / Не включаться до звіту	Alternate Method / Альтернативен метод / Drugačija metoda / Alternativní metoda / Alternativ metode / Alternativne metod / Autre méthode / Alternative Methode / Εναλλακτική μέθοδος / Alternatív módszer / Metodo alternativo / Балама әдіс / Alternatyvus metodos / Metoda zamienna / Método alternativo / Metodă alternativă / Альтернативный метод / Druga metoda / Alternativa metoda / Alternativ metod / Alternativ Yöntem / Альтернативный метод
<i>Burkholderia</i> species	Cefepime	
<i>Proteus mirabilis</i>	Tigecycline	
<i>Providencia</i> species	Tigecycline	
<i>Ralstonia</i> species	Cefepime	
<i>Serratia marcescens</i>		Colistin
<i>Serratia</i> species		Fosfomycin
<i>Stenotrophomonas maltophilia</i>	Ceftazidime-Avibactam, Colistin	
<i>Yersinia frederiksenii</i>	Aztreonam, Cefepime, Ceftazidime, Ceftriaxone	