

BD Onclarity™ HPV Self Collection Diluent Tubes



REF 444869

L0112242(01)

2020-10

English

INTENDED USE

BD Onclarity™ HPV Self Collection Diluent Tube is intended for the transfer and resuspension of self-collected vaginal specimens in the laboratory for testing with the BD Onclarity HPV Assay.

REAGENTS

Materials Provided: Each BD Onclarity HPV Self Collection Diluent Tube contains approximately 3 mL of a Tris/Sodium Chloride solution with surfactant and preservative.

Materials Required but Not Provided: Gloves and specimen transfer tool (if applicable)

Storage and Handling Requirements: Reagents may be stored at 2–33 °C. Do not use past the expiration date.

WARNINGS AND PRECAUTIONS

1. For *in vitro* Diagnostic Use. For trained laboratory personnel.
2. For self-collected specimens, use only the BD Onclarity HPV Self Collection Diluent Tubes.
3. Only use the self-collection devices validated with the BD Onclarity Self Collection Diluent Tubes. Check the registration status of the self-collection devices in your region to be used with BD Onclarity HPV Assay.
4. Pathogenic microorganisms including hepatitis viruses and Human Immunodeficiency Virus may be present in clinical specimens. "Standard Precautions"¹⁻⁴ and institutional guidelines should be followed in handling all items contaminated with blood and other body fluids.
5. Optimal performance of the BD Onclarity HPV Assay requires proper specimen collection, handling and transport.
6. When breaking the shaft of a self-collection device, take care to avoid splashing, spilling, or creating aerosols.
7. Take care to avoid cross-contamination during the specimen handling steps. If gloves come in contact with specimen, change gloves to avoid cross-contamination.
8. Breaking or cutting the shaft of a self-collection device at the wrong location, or adding the brush in the incorrect orientation could result in downstream processing errors, delaying test results.
9. Use only the designated transfer tool (if applicable) for the self-collection device transfer.
10. Handle one self-collection device at a time for processing.
11. Do not transport the self-collected specimens or store specimens in BD Onclarity HPV Self Collection Diluent Tubes beyond the product expiration date.
12. Refer to the self-collection device manufacturers' instructions for use for additional warnings and precautions.

EU Only: users should report any serious incident related to the device to the Manufacturer and National Competent Authority.

Outside EU: Contact your local BD representative for any incident or inquiry related to this device.

Warning



H317 May cause an allergic skin reaction. **H411** Toxic to aquatic life with long lasting effects.

P261 Avoid breathing dust/fume/gas/mist/vapors/spray. **P272** Contaminated work clothing should not be allowed out of the workplace.

P280 Wear protective gloves/ protective clothing/eye protection/face protection. **P273** Avoid release to the environment. **P302+P352** IF

ON SKIN: Wash with plenty of soap and water. **P333+P313** If skin irritation or rash occurs: Get medical advice/attention. **P321** Specific treatment (see on this label). **P363** Wash contaminated clothing before reuse. **P391** Collect spillage. **P501** Dispose of contents/container to an appropriate treatment and disposal facility in accordance with applicable laws and regulations, and product characteristics at time of disposal.

Dispose of all used reagents and any other contaminated disposable materials following procedures for infectious or potentially infectious waste. It is the responsibility of each laboratory to handle solid and liquid waste according to their nature and degree of hazardousness and to treat and dispose of them (or have them treated and disposed of) in accordance with any applicable regulations.

SPECIMEN COLLECTION, TRANSFER, STORAGE AND TRANSPORT

Specimens must be collected using self-collection devices as described in the respective manufacturer's product insert. Self-collection devices validated for use with the BD Onclarity HPV Self Collection Diluent Tube include the Copan FLOQSwabs[®] (Catalog No. 5E089N), Rovers[®] Evalyn[®] Brush (Catalog No. 380500131), and Rovers[®] Viba-Brush[®] (Catalog No. 380200331). Rovers[®] ReMover[®] (Catalog No. 360106045) can be used as the specimen transfer tool for the Rovers devices.

After collection, specimens shall be stored and transported dry according to the manufacturer's instructions. Post collected specimens can be stored up to 30 days at -20 °C or 2–30 °C. Self-collected samples in the BD Onclarity HPV Self Collection Diluent Tube can be stored up to 15 days at -20 °C or 2–30 °C prior to testing with BD Onclarity HPV Assay. After pre-warming, samples can be stored up to 7 days at -20 °C or 2–30 °C.


For domestic and international shipments, specimens should be packaged and labeled in compliance with applicable state, federal, and international regulations covering the transport of clinical specimens and etiologic agents/infectious substances. Time and temperature conditions for storage must be maintained during transport.

Wear gloves when handling the BD Onclarity HPV Self Collection Diluent Tube and the self-collected specimens. If gloves come in contact with the specimen, immediately change them to prevent contamination of other specimens.

SELF COLLECTION DEVICES


Copan FLOQSwabs (Catalog No. 5E089N)

1. Soft nylon flocked tip
2. Breakpoint with red mark
3. Peelable barcode label

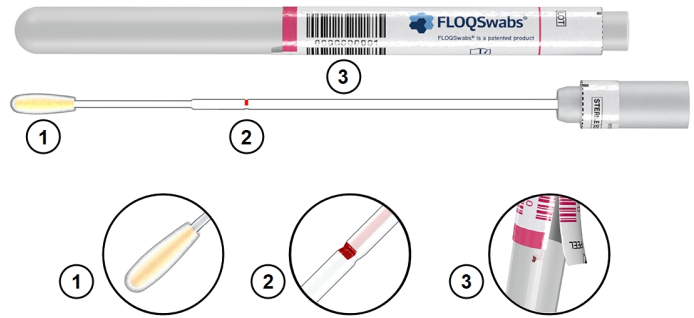


Rovers Evalyn Brush (Catalog No. 380500131)

1. White brush
2. Pink cap
3. Wings
4. Transparent casing
5. Pink plunger

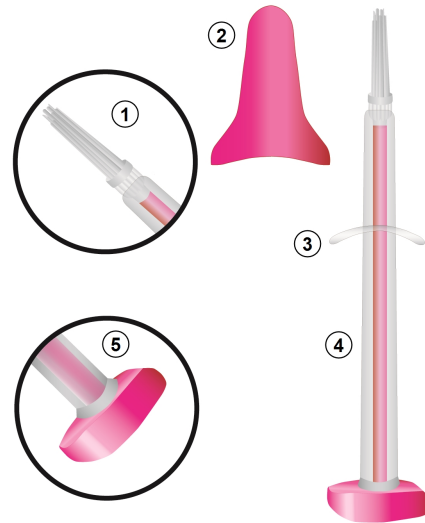

Rovers Viba Brush (Catalog No. 380200331)


1. White brush
2. Pink handle

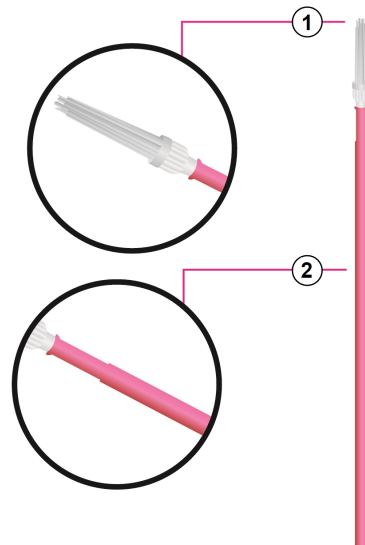

PROCEDURES


Transfer of Copan FLOQSwabs to the BD Onclarity HPV Self Collection Diluent Tube

1. Uncap a BD Onclarity HPV Self Collection Diluent Tube.
2. Hold the tube in one hand with the tube opening facing away from your face.
3. Grip the end of the shaft with the thumb and forefinger of the other hand.
4. Align the breakpoint (red area with indent) with the edge of the tube.

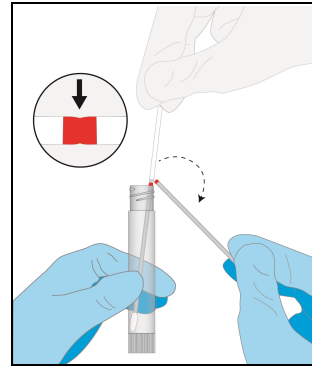


5. Bend the swab shaft to an angle of 180° so that it breaks at the breakpoint. If necessary, gently turn the swab shaft until it is completely broken off. Discard the broken off part of the swab shaft.

WARNING: Breaking or cutting the shaft at the wrong location could result in downstream processing errors, delaying test results.





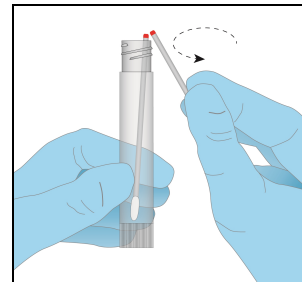


6. Tightly recap the tube.
7. Use a vortex mixer to vortex the tube at high speed for 5 seconds to ensure that the specimen and diluent are well mixed.
8. Refer to BD Onclarity HPV Assay instructions for use to process sample for HPV testing.

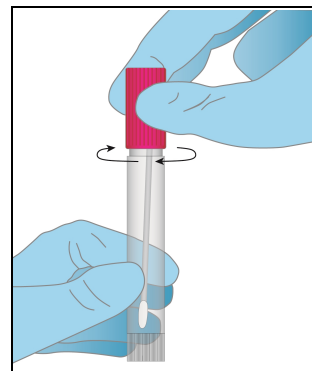


Transfer of Rovers Evalyn Brush (fully assembled device) to the BD Onclarity HPV Self Collection Diluent Tube

1. Uncap a BD Onclarity HPV Self Collection Diluent Tube.
2. Squeeze the sides of the pink cap with your thumb and index finger to remove pink cap from the Evalyn brush.
3. Depress the plunger into the casing to expose the brush head.
4. Ensure that you do not touch the white fibers of the Evalyn brush with your hands.
5. Guide the Evalyn brush into the ReMover until the casing hits the ReMover.
NOTE: A new Rovers ReMover must be used for each brush.
6. Squeeze the ReMover firmly together and pull off the brush head. Firmly keep the ReMover compressed.
7. Hold the inverted ReMover over the mouth of the tube and drop the brush into the tube by releasing your grip.
WARNING: The brush head must be oriented with the bristles facing up in the tube to avoid downstream processing errors.
8. Tightly recap the tube.
9. Use a vortex mixer to vortex the tube at high speed for 5 seconds to ensure that the specimen and diluent are well mixed.
10. Refer to BD Onclarity Assay instructions for use to process sample for HPV testing.

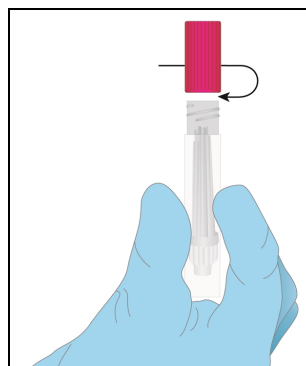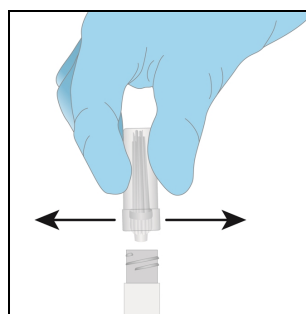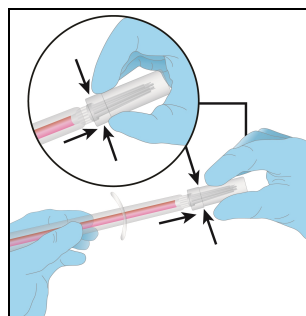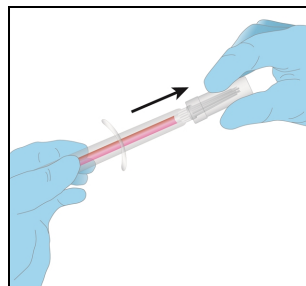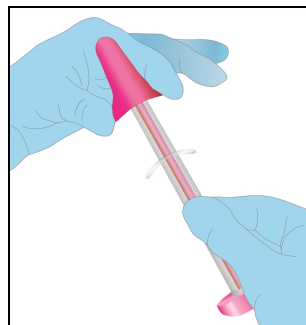


Transfer of Rovers Evalyn and Viba in the ReMover to the BD Onclarity HPV Self Collection Diluent Tube

1. Uncap a BD Onclarity HPV Self Collection Diluent Tube.
2. Grip the ReMover and pull off the cap. Maintain grip on the ReMover.

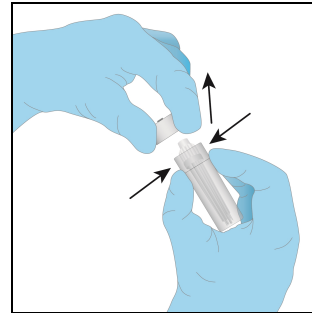


3. Hold the inverted ReMover over the mouth of the tube and drop the brush into the tube by releasing your grip.

WARNING: The brush head must be oriented with the bristles facing up in the tube to avoid downstream processing errors.

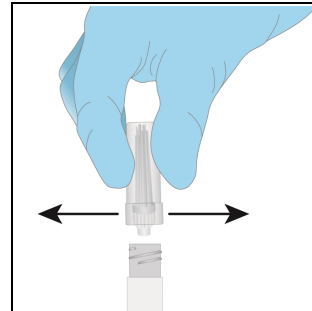

4. Tightly recap the tube.
5. Use a vortex mixer to vortex the tube at high speed for 5 seconds to ensure that the specimen and diluent are well mixed.
6. Refer to BD Onclarity HPV Assay instructions for use to process sample for HPV testing.

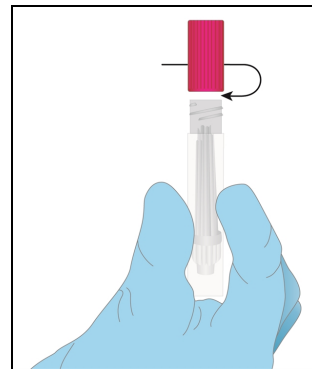

REFERENCES

1. Clinical and Laboratory Standards Institute. Protection of laboratory workers from occupationally acquired infections. Approved guideline M29 (Refer to the latest edition).
2. Garner, J.S. 1996. Hospital Infection control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guide for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53–80.
3. U. S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C. USA.
4. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/31/EEC). Official Journal L262, 17/10/2000, p. 0021–0045.

Technical Service and Support: contact your local BD representative or bd.com.

Change History

Revision	Date	Change Summary
01	2020-10	Initial release.

Some symbols listed below may not apply to this product.



Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbricante / Atқарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirkker / Producent / Producător / Производител / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник / 生产厂商



Use by / Използвайте до / Spotřebujte do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / 사용 기한 / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izlietot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Использовать до / Použite do / Upotrebiti do / Använd före / Son kullanma tarihi / Використати до / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)
TTTT-MM-DD / TTTT-MM (MM = края на месеца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = slutning af måned)
JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)
EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)
AAAA-MM-DD / AAAA-MM (MM = fin del mes)
AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)
AAAA-MM-JJ / AAAA-MM (MM = fin du mois)
GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)
ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja)
AAAA-MM-GG / AAAA-MM (MM = fine mese)
ЖОЖОЖ-АА-КК / ЖОЖОЖ-АА / (АА = айдың соңы)
YYYY-MM-DD/YYYY-MM (MM = 월말)
MMMM-MM-DD / MMMM-MM (MM = menses pabaiga)
GGGG-MM-DD/GGGG-MM (MM = mēneša beigas)
JJJJ-MM-DD / JJJJ-MM (MM = einde maand)
AAAA-MM-DD / AAAA-MM (MM = sluten av månaden)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
AAAA-MM-DD / AAAA-MM (MM = fim do mês)
AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)
TTTT-MM-DD / TTTT-MM (MM = конец месяца)
RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)
GGGG-MM-DD / GGGG-MM (MM = kraj meseca)
AAAA-MM-DD / AAAA-MM (MM = slutet av månaden)
YYYY-AA-GG / YYYY-AA (AA = ayın sonu)
PPPP-MM-DD / PPPP-MM (MM = кінець місяця)
YYYY-MM-DD / YYYY-MM (MM = 月末)



Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalógusszám / Numero di catalogo / Каталог номери / 카탈로그 번호 / Katalogo / numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目录号



Authorized Representative in the European Community / Оторизирани представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret repræsentant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitadud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Europskoj uniji / Meghatalmazott képviselő az Európai Közösségben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті өкіл / 유럽 공동체의 위임 대표 / Įgaliojasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Avrupa Topluluğu Yetkili Temsilcisi / Уповноважений представник у країнах ЄС / 欧洲共同体授权代表



In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiinaparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnostikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жүргізетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietaisai / Medicinas ierīces, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispositiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomôcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diyagnostik Tibbi Cihaz / Медицинский прибор для диагностики in vitro / 体外诊断医疗设备



Temperature limitation / Температурни ограничения / Temperaturlimite / Temperaturbegrænsning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperaturi piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шектеу / 온도 제한 / Laikymo temperatūra / Temperatūras ierobežojumi / Temperaturilimiet / Temperaturbegrænsning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ograničenje toploty / Ograničenje temperature / Temperaturgräns / Sıcaklık sınırlaması / Обмеження температури / 温度限制



Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-code (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топтама коды / 배치 코드 (로트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-code (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partnummer (Lot) / Parti Kodu (Lot) / Код партії / 批号 (亚批)



Contains sufficient for <n> tests / Съдържанието е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenuto sufficiente per <n> test / <n> тесттері үшін жеткілікті / <n> 테스트가 충분히 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor "n" testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Conținut suficient pentru <n> teste / Достаточное для <n> тестов(а) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli miktarda içerir / Вистачить для аналізів: <n> / 足够进行 <n> 次检测



Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / 사용 지침 참조 / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Poglédajte uputstvo za upotrebu / Se bruksanvisningen / Kullanim Talimatları'na başvurun / Див. інструкції з використання / 请参阅使用说明



Do not reuse / Не използвайте отново / Nepoužívejte opakovaně / Ikke til genbrug / Nicht wiederverwenden / Μην επαναχρησιμοποιείτε / No reutilizar / Mitte kasutada korduvalt / Ne pas réutiliser / Ne koristiti ponovo / Egyszer használatos / Non riutilizzare / Пайдаланбаңыз / 재사용 금지 / Tik vienkartiniam naudojimui / Nelietot atkārtoti / Niet opnieuw gebruiken / Kun til engangsbruk / Nie stosować powtórnie / Não reutilize / Nu refolosiți / Не использовать повторно / Nepoužívejte opakovane / Ne upotrebljavajte ponovo / Får återanvändas / Tekrar kullanmayın / Не використовувати повторно / 请勿重复使用

	Serial number / Серийный номер / Sériové číslo / Seriennummer / Seriennummer / Σειριακός αριθμός / N° de serie / Seerianumber / Numéro de série / Serijski broj / Sorozatszám / Numero di serie / Топтамалық нөмірі / 일련 번호 / Serijos numeris / Sérijas numurs / Serie nummer / Numer seryjny / Número de série / Număr de serie / Серийный номер / Seri numarası / Номер серії / 序列号
	For IVD Performance evaluation only / Само за оценка качеството на работа на IVD / Pouze pro vyhodnocení výkonu IVD / Kun til evaluering af IVD ydelse / Nur für IVD-Leistungsbewertungszwecke / Μόνο για αξιολόγηση απόδοσης IVD / Sólo para la evaluación del rendimiento en diagnóstico in vitro / Ainult IVD seadme hindamiseks / Réserve à l'évaluation des performances IVD / Samo u znanstvene svrhe za In Vitro Dijagnostiku / Kizárolag in vitro diagnosztikához / Solo per valutazione delle prestazioni IVD / Жасанды жағдайда «пробирка ішінде» диагностикада тек жұмысты бағалау үшін / IVD 성능 평가에 대해서만 사용 / Tik IVD prietaisų veikimo charakteristikoms tikrinti / Vienigi IVD darības novērtēšanai / Uitsluitend voor doeltreffendheidsonderzoek / Kun for evaluering av IVD-ytelse / Tylko do oceny wydajności IVD / Uso exclusivo para avaliação de IVD / Numai pentru evaluarea performanței IVD / Только для оценки качества диагностики in vitro / Určené iba na diagnostiku in vitro / Samo za procenu učinka u in vitro dijagnostici / Endast för utvärdering av diagnostisk användning in vitro / Yalnızca IVD Performans değerlendirmesi için / Тільки для оцінювання якості діагностики in vitro / 仅限 IVD 性能评估 For US: "For Investigational Use Only"
	Lower limit of temperature / Долен лимит на температурата / Dolni hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuripiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шегі / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurlimiet / Nedre temperaturgrænse / Dolna granica temperatury / Limite minimo de temperatura / Limită minimă de temperatură / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Мінімальна температура / 温度下限
CONTROL	Control / Контроль / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Controllo / Бақылау / 컨트롤 / Kontrolé / Kontrole / Controle / Controllo / Контроль / kontrol / Контроль / 对照
CONTROL +	Positive control / Положительный контроль / Pozitivní kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Pozitív kontroll / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitívá kontrolle / Positieve controle / Kontrola dodatnia / Controllo positivo/ Control pozitiv / Положительный контроль / Pozitif kontrol / Позитивный контроль / 阳性对照试剂
CONTROL -	Negative control / Отрицательный контроль / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negatívna kontrola / Negatív kontroll / Controllo negativo / Негативтік бақылау / 음성 컨트롤 / Neigiama kontrolė / Negatívá kontrolle / Negatieve controle / Kontrola ujemna / Controllo negativo / Control negativ / Отрицательный контроль / Negatif kontrol / Негативный контроль / 阴性对照试剂
STERILE EO	Method of sterilization: ethylene oxide / Метод на стерилизация: этиленов оксид / Způsob sterilizace: ethylenoxid / Steriliseringmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseerimismetod: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etlén-oxid / Metodo di sterilizzazione: ossido di etilene / Стерилизация адісі – этилен тотығы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksīds / Gesteriliseerd met behulp van ethylenoxide / Steriliseringmetode: etylenoksid/ Metoda sterylizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Метод стерилизации: этиленоксид / Metóda sterilizácie: etylénoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: этиленоксидом / 灭菌方法: 环氧乙烷
STERILE R	Method of sterilization: irradiation / Метод на стерилизация: иррадиация / Způsob sterilizace: záření / Steriliseringmetode: bestråling / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseerimismetod: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metodo di sterilizzazione: irradiazione / Стерилизация адісі – сәуле түсіру / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringmetode: bestråling / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metodă de sterilizare: iradiere / Метод стерилизации: облучение / oziarenie / Metóda sterilizácie: ozračovanie / Steriliseringmetod: strålning / Sterilizasyon yöntemi: irradyasyon / Метод стерилизації: опроміненням / 灭菌方法: 辐射
	Biological Risks / Биологични рискове / Biologická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biologilised riskid / Risques biologiques / Bioiški rizik / Biológiallag veszélyes / Rischio biologico / Биологиялык тәуекелдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisk risk / Biológico riesgo / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Bioiški rizici / Biologisk risk / Biyolojik Riskler / Біологічна небезпека / 生物学风险
	Caution, consult accompanying documents / Внимание, направьте справку в придружающие документы / Pozor! Prostudujte si příloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleiddokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatus! Lugeka kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Upozorenie, koristi prateću dokumentaciju / Figyelem! Olvassa el a mellékeltájékoztatót / Attenzione: consultare la documentazione allegata / Абайланба, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dėmesio, žiūrėkite priedamus dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увага: див. супутню документацію / 小心, 请参阅附带文档。
	Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrænse / Gorna granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sicaklık üst sınırı / Максимальна температура / 温度上限
	Keep dry / Пазете сухо / Skladujte v suchém prostredí / Opbevares tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec / Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрғақ күйінде ұста / 건조 상태 유지 / Laikykite sausai / Uzglabāt sausu / Droog houden / Hoides tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať v suchu / Držite na suvom mestu / Förvaras torrt / Kuru bir şekilde muhafaza edin / Берегти від вологи / 请保持干燥
	Collection time / Време на събиране / Čas odběru / Opsamlingsstidspunkt / Entnahmezeit / Όρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жынау уақыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colheita / Ora colectării / Время сбора / Doba odboru / Vreme prikupljanja / Uppsamlingstid / Toplama zamani / Час забору / 采集时间
	Peel / Обелете / Olevfete zde / Abn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Ўстиғи қабатын алып таста / 벗기기 / Plešti čia / Alfmitt / Schillen / Trekk av / Oderwać / Destacar / Se dezlipeste / Отклеить / Odtrhnite / Oljuštiti / Dra isår / Айurma / Відклеїти / 撕下
	Perforation / Перфорация / Perforace / Perforering / Διάτρηση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Tecik tescy / 찢질선 / Perforacija / Perforacija / Perforatie / Perforacja / Perfuração / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация / 穿孔
	Do not use if package damaged / Не използвайте, ако опаковката е повредена / Npoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packungnicht verwenden / Mn χρησιμοποίητε εάν η συσκευασία έχει υποστεί ζημιό. / No usar si el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használja, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Егер пакет бүзылган болса, пайдаланба / 패키지가 손상된 경우 사용 금지 / Jei pakuočių pažeista, nenaudoti / Nelietot, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Не использовать при повреждении упаковки / Npoužívať, ak je obal poškozený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包装破损, 请勿使用



Keep away from heat / Πазете от топлина / Nevystavujte přilišnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protéger de la chaleur / Držati dalje od izvora topline / Övja a melegtől / Tenere lontano dal calore / Сапқын жерде сақта / 열을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródła ciepła / Manter ao abrigo do calor / A se feri de căldură / Не награвать / Uchovávať mimo zdroja tepla / Držite dalje od toplote / Får ej utsättas för värme / Isidan uzak tutun / Берегти від дії тепла / 请远离热源



Cut / Срежете / Odstřihněte / Klip / Schneiden / Κόψτε / Cortar / Lögata / Découper / Reži / Vágja ki / Tagliare / Keciňiz / 잘라내기 / Kirpti / Nogriez / Knippen / Kutt / Odciąć / Cortar / Decupați / Отрезать / Odstrihnite / Iseći / Klipp / Kesme / Pozpizati / 剪下



Collection date / Дата на събиране / Datum odběru / Opsamlingsdato / Entnahmedatum / Ημερομηνία συλλογής / Fecha de recogida / Kogumiskuupäev / Date de prélèvement / Dani prikupljanja / Mintavétel dátuma / Data di raccolta / Жинаган тизбекүні / 수집 날짜 / Paėmimo data / Savākšanas datums / Verzameldatum / Dato prøvetaking / Data pobrania / Data de colheita / Data colectării / Дата сбора / Datum odberu / Datum prikupljanja / Uppsamlingsdatum / Toplama tarihi / Дата забору / 采集日期



μL/test / μL/тест / μL/Test / μL/εξέταση / μL/prueba / μL/teszt / μL/테스트 / мкл/тест / μL/tyrimas / μL/pārbaude / μL/teste / мкл/анализ / μL/检测



Keep away from light / Πазете от светлина / Nevystavujte světlu / Må ikke udsættes for lys / Vor Licht schützen / Κρατήστε το μακριά από το φως / Mantener alejado de la luz / Hoida eemal valgusest / Conserver à l'abri de la lumière / Držati dalje od svetla / Fény nem érheti / Tenere al riparo dalla luce / Қаранғыланған жерде ұста / 빛을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no gaismas / Niet blootstellen aan zonlicht / Må ikke utsettes for lys / Przechowywać z dala od źródła światła / Manter ao abrigo da luz / Feriți de lumină / Хранить в темноте / Uchovávať mimo dosahu svetla / Držite dalje od svetlosti / Får ej utsättas för ljus / Isıktan uzak tutun / Берегти від дії світла / 请远离光线



Hydrogen gas generated / Образуван е водород газ / Možnost úniku plynného vodíku / Frembringer hydrogen gas / Wasserstoffgas erzeugt / Δημιουργία αερίου υδρογόνου / Producción de gas de hidrógeno / Vesinikgaasi tekitatud / Produit de l'hydrogène gazeux / Sadržaji hydrogen vodik / Hidrogén gázt fejleszt / Produzione di gas idrogeno / Газтөктес сутегі пайда болды / 수소 가스 생성됨 / Išskiria vandenilio dujas / Rodas ūdeņradis / Waterstofgas genereerd / Hydrogengass generert / Powoduje powstawanie wodoru / Produção de gás de hidrogénio / Generare gaz de hidrogen / Выделение водорода / Vyrobené použitím vodíka / Oslobađa se vodonik / Genererad välgas / Açığa çıkan hidrojen gazı / Реакция з виділенням водню / 会产生氢气



Patient ID number / ИД номер на пациента / ID pacienta / Patientens ID-nummer / Patienten-ID / Αριθμός αναγνώρισης ασθενούς / Número de ID del paciente / Patsiendi ID / No d'identification du patient / Identifikacijski broj pacijenta / Beteg azonosító száma / Numero ID paziente / Пациенттің идентификациялық нөмірі / 환자 ID 번호 / Paciento identifikavimo numeris / Pacienta ID numurs / Identificatienummer van de patiënt / Pasientens ID-nummer / Numer ID pacjenta / Número da ID do doente / Număr ID pacient / Идентификационный номер пациента / Identifikačné číslo pacienta / ID broj pacijenta / Patientnummer / Hasta kimlik numarası / Идентифікатор пацієнта / 患者标识号



Fragile, Handle with Care / Чупливо, Роботете с необходимото внимание. / Křehké. Při manipulaci postupujte opatrně. / Forsigtig, kan gå i stykker. / Zerbrechlich, vorsichtighandhaben. / Εύθραστο. Χειριστείτε το με προσοχή. / Frágil. Manipular con cuidado. / Örn, käsitsege ettevaatlikult. / Fragile. Manipuler avec précaution. / Lomljivo, rukujte pažljivo. / Törékeny! Óvatosan kezelendő. / Fragile, maneggiare con cura. / Сыңғыш, абайлап пайдаланыңыз. / 조심 깨지기 쉬운 처리 / Trapu, elkités atsargiai. / Trausis; rūkotiis uzmanigi / Breekbaar, voorzichtig behandelen. / Ømtålig, håndter forsigtig. / Krucha zawartość, przenosić ostrożnie. / Frágil, Manuseie com Cuidado. / Frágil, manipulați cu atenție. / Хрупкое! Обращаться с осторожностью. / Křehké, vyžaduje sa opatrná manipulácia. / Lomljivo - rukujte pažljivo. / Bräckligt. Hantera försiktigt. / Kolay Kırılır, Dikkatli Taşıyın. / Тендітна, звертатися з обережністю / 易碎，小心轻放



bd.com/e-labeling



Becton, Dickinson and Company
7 Loveton Circle
Sparks, Maryland 21152 USA



Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:
Becton Dickinson Pty Ltd.
66 Waterloo Road
Macquarie Park NSW 2113
Australia

New Zealand Sponsor:
Becton Dickinson Limited
14B George Bourke Drive
Mt. Wellington Auckland 1060
New Zealand

BD, the BD Logo, and Onclarity are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2020 BD. All rights reserved.