


BD PosiFlush™ Pre-Filled Syringes

Usage Guidelines




Do not place BD PosiFlush™ Pre-Filled Syringes on a sterile field.

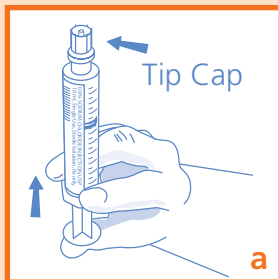


BEFORE USE:


1. Tear open package at either end and remove syringe.
2. Depress plunger with **tip cap on** to relieve the resistance between the stopper and the barrel (Fig. a).

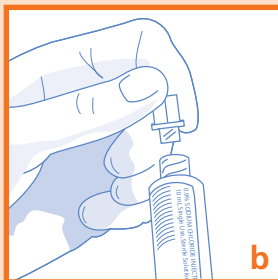

3. Using aseptic technique, remove the syringe tip cap from the FLUSH syringe by twisting it off (Fig. b).

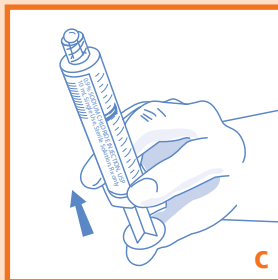

4. Hold the syringe upright and expel the air and any excess solution in the syringe by positioning the front rib of the gray plunger tip at desired solution volume (Fig. c).
5. Attach the syringe to the injection site and flush following institution's policy.

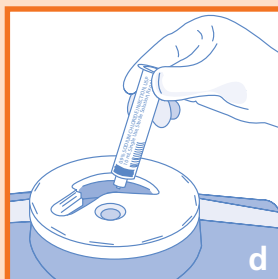

AFTER USE:

6. Discard used syringe including any unused solution following institution policy (Fig. d). **DO NOT REUSE.**

Note: 3, 5 and 10 mL BD PosiFlush™ syringes all have the same diameter as a standard 10 mL syringe and may be used for flushing catheters when protocols stipulate use of a 10 mL syringe to minimize flush pressure (PSI).