

# Safety Comes With Advantages

## BD Integra™ 3ml Syringe with Retracting BD PrecisionGlide™ Needle



### IMPORTANT USAGE GUIDELINES

#### 1. ATTACHMENT



Attach the BD Integra™ needle onto the BD Integra™ syringe with a *push and a clockwise twist*. Pull the shield directly off of the needle

**\*Important:**

Always tighten the needle on to the syringe prior to use.

#### 2. ASPIRATION\*



Draw up medication in accordance with established protocol.

#### 3. INJECTION



Administer medication in accordance with established protocols, making sure all medication has been dispensed.

#### 4. ACTIVATION\*



Push plunger rod to activate the safety mechanism. Depress until 2 "clicks" are sensed.

**\*Important:**

Activation of device can occur before or after† needle is withdrawn from patient.

#### 5. NEEDLE RETRACTION



The needle will disappear into the syringe and the plunger will be recessed into the barrel when activation is complete. There is no exposure to the used needle.

**Discard immediately into nearest sharps container.**

#### ISO NEEDLE HUB<sup>1</sup>/ PACKAGING CHART

25G

23G

22G

21G

#### BD INTEGRA BLUNT FILL AND FILTER NEEDLE<sup>2</sup>

18G Fill

18G Filter

#### THINGS TO REMEMBER:



- BDIntegra Needle and Syringe **DO NOT** HAVE A STANDARD LUER CONNECTION. Conventional Luer Lock needles cannot be used with the BD Integra Syringe.
- TheBD Integra Syringe is not compatible with valves or other needleless systems.
- When pressurizing a medication vial, do not continue to press the plunger rod past the point of resistance, as this may activate the device prematurely.
- If Activation is performed before needle is withdrawn from patient, do not advance needle while depressing plunger rod.
- † If Activation is performed after needle is withdrawn from patient, minimal splatter may occur. When activating the device outside of the patient, hold the device downward at arm's length, and activate while pointing away from self and others. Keep device within sight at all times to avoid accidental needlestick injury.
- **DO NOT** use this product to draw blood or for venipuncture.
- When delivering viscous medications (Ativan, Penicillin G) use a 22G BD Integra Retracting Needle or larger, and inject at a controlled rate.

<sup>1</sup> Colors reflect worldwide ISO color standards to enhance worldwide sourcing and product supply.

<sup>2</sup> Not for injection