





Indispensable to human health

BD QuikHeel™ Lancet


QUICK REFERENCE CARD

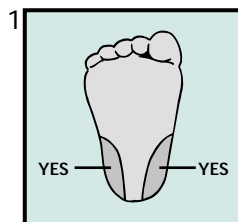

- Select the site and cleanse thoroughly

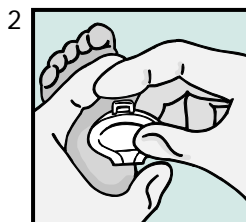


- With a gloved hand, place the QuikHeel Lancet against the site with the QuikHeel logo facing you, your thumb on the logo and your middle finger on the back contour. Place the blade slot area securely against the heel at a 90 degree angle to the length of the foot

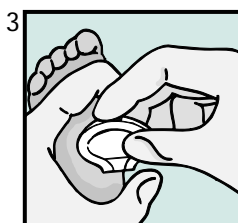


- Using your index finger, firmly and completely depress the trigger

BD QuikHeel™ Lancet

QUICK REFERENCE CARD

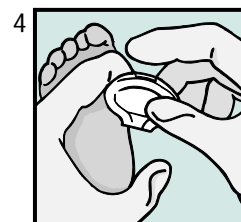

- After triggering, remove the lancet and discard lancet into a biohazard sharps

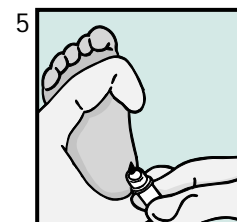

- Collect the specimen

Things to Remember:

- Handle all biologic samples and blood collection sharps (lancets, needles, luer adapters and blood collection sets) according to the policies and procedures of your facility
- Obtain appropriate medical attention in the event of any exposure to biologic samples (for example, through a puncture injury) since they may transmit viral hepatitis, HIV (AIDS) or other infectious diseases
- Utilize any built-in needle protector if the blood collection device provides one
- BD does not recommend resheilding used needles, but the policies and procedures of your facility may differ and must always be followed
- Discard BD QuikHeel Lancet in the nearest sharps collector. Follow your institution's policy for safe disposal of all medical waste

All noted trademarks are property of Becton, Dickinson and Company
1 Becton Drive, Franklin Lakes, NJ 07417 1-888-237-2762 www.bd.com
©1999 Becton, Dickinson and Company Printed in USA BD10/99-0373