

PRODUCT NOTIFICATION BULLETIN

Type: Product Addition

Bulletin Number: 3

Date: July 1, 2009

The BD™ RCRA collector product line, available since September 2008, has expanded. There are now two categories of collectors available:

1. Non-DOT collectors – 3 and 5 gallon one piece, 9 and 17 gallon hinge top, and 9 and 19 gallon slide top
2. DOT PGII collectors - 13 and 16 gallon with ports

Reference #	Product Size	Qty/Case
305066	RCRA Collector 3 Gallon One Piece Plug Top	12
305067	RCRA Collector 5 Gallon One Piece Plug Top	8
305068	RCRA Collector 9 Gallon Hinge Top	8
305069	RCRA Collector 9 Gallon Slide Top	8
305071	RCRA Collector 17 Gallon Hinge Top	5
305072	RCRA Collector 19 Gallon Slide Top	5
305073	RCRA Collector 13 Gallon, DOT, Port Top	6
305074	RCRA Collector 16 Gallon, DOT, Port Top	6



These collectors must be incinerated for final disposal.

What is PGII?

PG stands for Packing Group. This is a Department of Transportation term that groups hazardous materials according to the danger presented by the materials. There are three packing group classifications.

Packing Group Number	Danger
I	Great Danger
II	Medium Danger
III	Minor Danger



Product Comparison – DOT versus Non DOT

	DOT PG II	Non DOT
Color	Black base with black top	Black base with clear top
Visibility of contents	No	Yes, through lid only
Recycled content	None	70% (base 100% recycled plastic, top 0%)
Lid Type	Port with 7" opening	Varies per product (hinge, slide, 3 lock cap)
Trolley compatibility	Not available yet	Yes for 9, 17 and 19 gallon collectors
Liquid-absorbent pad in collector	No	Yes
Gasketed for liquid leak resistant seal	Yes, glue-type gasket	Yes, foam-type gasket with 9, 17 and 19 gallon collectors

How To Handle Hazardous Waste that include Sharps

RCRA collectors are not sharps collectors (see product label). These products are designed to contain solid hazardous waste and trace liquids associated with these wastes.

Non-DOT RCRA collectors should be secured in a DOT compliant container/cardboard box for transportation. The hazardous waste hauler may recommend the best packaging option based on the hazardous waste generated by the customer and the location for final disposal.

Trolley Compatibility

The non-DOT RCRA collectors can be used with trolleys for added stability and mobility of the collector.

Reference #	Product Description
305611	Large Basic Trolley
305612	Large Foot-operated Trolley
305092	Large BD Recykleen™ Basic Trolley
305093	Large BD Recykleen™ Foot-operated Trolley

