# CYTORICH® BLUE PRESERVATIVE FLUID

REF 05NG000019 3600 ml

Preservative fluid for general cytology, packaged with leak-resistant caps. (Contains alcohol and polyethylene glycol.)

## INTENDED USE

CYTORICH® Blue Preservative Fluid is intended for preservation of cells in suspension and their subsequent preparation for cytological evaluation.

## SUMMARY AND EXPLANATION

Non-cervicovaginal cytologic screening involves the microscopic examination of cell samples that have been harvested from human body sources, including sputum, bronchial washings, breast tissue aspirations, spinal fluid, other fluids, exudates and fine needle aspirates of specific organs. These cell samples are usually fixed in an alcohol-based liquid (e.g. CYTORICH® Blue Preservative Fluid) and are concentrated with centrifugation. These samples are then either smeared or sedimented onto a glass microscope slide and stained by the Papanicolaou or other staining system. CYTORICH® Blue Preservative Fluid is an alcohol-based preservative fluid which may be used for the preservation of cells in suspension.

## STORAGE REQUIREMENTS

Store at room temperature (15° - 30° C).

## GENERAL PRECAUTIONS

1. Do not pipette by mouth.
2. Do not allow any reagents to come in contact with open wounds.
3. Wear powder-free latex gloves, lab coat and eye protection during the procedure.
4. Do not ingest (contains denatured alcohol).

**WARNING:** Cytologic specimens may contain infectious agents. Wear gloves and follow appropriate biohazard precautions when handling samples.

## SPECIMEN COLLECTION AND STABILITY

- Label the specimen collection container before use.
- CYTORICH® Blue Preservative Fluid contains 50% w/v alcohol and should be used by mixing one (1) volume of sample with one (1) volume of CYTORICH® Blue Preservative Fluid.
- The minimum fixation time is 30 minutes.
- Cytologic specimens are stable in CYTORICH® Blue Preservative Fluid for up to fourteen days at room temperature.
- Cytologic preparations may be prepared using the PREPSTAIN® slide processor, standard cytocentrifugation methods, or smear techniques.

## LIMITATIONS OF THE PROCEDURE

Cytologic samples should be fixed in CYTORICH® Blue Preservative Fluid as soon as possible after collection. A sample which has become degraded prior to fixation may be unsatisfactory for cytologic examination.

## BIBLIOGRAPHY

1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881-906
2. Barrett DL: Cytocentrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80-83
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495
5. Hajdu SI: A note on the history of Carbo wax. Acta Cytol 1983; 27:204-206

REF Catalog number

IVD For *in vitro* diagnostic use

i Consult instructions for use

Σ Contains / Sufficient for <n> tests

⚠ Caution, consult accompanying document

🌡 Storage Temperature Limitations

LOT Batch code (lot)

🕒 Use by

🏭 Manufacturer

TECHNICAL SUPPORT

USA

Telephone: 1-877-822-7771

Fax: 1-336-290-8333

Email: [tripathtechsupport@bd.com](mailto:tripathtechsupport@bd.com)

Europe

Telephone: +32 (0)53 720 673

Fax: +32 (0)53 720 678

TriPath Imaging, Inc.  
780 Plantation Drive  
Burlington, NC 27215 USA

EC REP  
Benex Limited  
Rineanna House  
Shannon Free Zone  
Shannon, County Clare  
Ireland

SurePath, SurePath PreCoat slides, CytoRich and PrepStain are products and registered trademarks of TriPath Imaging, Inc.

©2011 TriPath Imaging, Inc. All Rights Reserved.

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## BIBLIOGRAPHIE

1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881-906
2. Barrett DL: Cytozentrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80-83
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495
5. Hajdu SI: A note on the history of Carbo wax. Acta Cytol 1983; 27:204-206

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### BIBLIOGRAPHIE

1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881-906
2. Barrett DL: Cyto centrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80-83
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495
5. Hajdu SI: A note on the history of Carbo wax. Acta Cytol 1983; 27:204-206

REF

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IVD

Dispositif médical de diagnostic in vitro

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Attention, consulter les documents joints

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EC REP

Benex Limited  
Rineanna House  
Shannon Free Zone  
Shannon, County Clare  
Ireland

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### BIBLIOGRAFI

1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, ss. 881-906
2. Barrett DL: Cytocentrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, ss. 80-83
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495
5. Hajdu SI: A note on the history of Carbo wax. Acta Cytol 1983; 27:204-206

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### BIBLIOGRAFIA

1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881-906
2. Barrett DL: Cyto-centrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80-83
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495
5. Hajdu SI: A note on the history of Carbo wex. Acta Cytol 1983; 27:204-206

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- Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881-906
- Barrett DL: Cytocentrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80-83
- Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446
- Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495

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### BİBLİYOGRAFYA

1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881-906
2. Barrett DL: Cytoentrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80-83
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495
5. Hajdu SI: A note on the history of Carbo wex. Acta Cytol 1983; 27:204-206

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1. Keebler CM: Cytopreparatory Techniques. In Bibbo M (ed): Comprehensive Cytopathology, 1st ed. Philadelphia: WB Saunders, 1991, pp 881-906
2. Barrett DL: Cytocentrifuge Technique. In Keebler CM, Reagan JW, Wied GL (eds): Compendium on Cytopreparatory Techniques, 4th ed. Chicago: Tutorials of Cytopathology, 1976, pp 80-83
3. Keebler CM, Somrak RM: The Manual of Cytotechnology, 7th ed. Chicago: ASCP Press, 1993, 35:412-446
4. Saccomanno G: Sputum fixatives. Acta Cytol 1977; 21:495
5. Hajdu SI: A note on the history of Carbo wex. Acta Cytol 1983; 27:204-206

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©2011 TriPath Imaging, Inc. Wszelkie prawa zastrzeżone.