

# #1 National Patient Safety Goal: "Improve the Accuracy of Patient Identifiers" –The Joint Commission

## BD Microtainer® MAP Microtube for Automated Process When Every Drop Counts



Examples of **misaligned** secondary labels



Steps to **properly align** secondary labels




**Step 1**  
Using the colored sidebar as your guide, position the V-Notch™ secondary label on the tube, leaving the colored sidebar exposed.

**Step 2**  
Smoothly wrap the V-Notch™ secondary label around the tube, leaving a clear window to view the sample.



**Benefits** of Proper Secondary Label Alignment

- Positive patient ID
- Bar code scannable
- Full-size patient labels
- Operational efficiencies
- Easy to see fill volumes
- Tube type ID if closure is removed

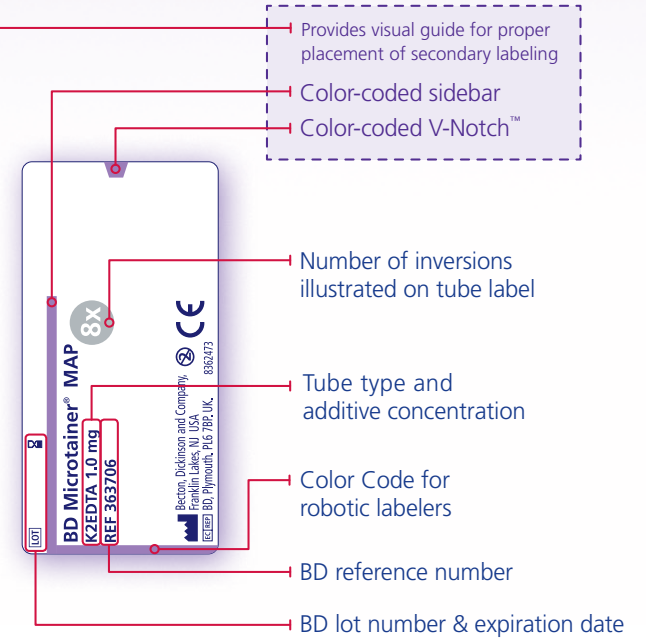

**BD Diagnostics**  
 Preanalytical Systems  
 1 Becton Drive  
 Franklin Lakes, NJ 07417  
[www.bd.com/vacutainer](http://www.bd.com/vacutainer)

### Positive ID. Positive Outcomes.™

For more information, please contact:  
 BD Global Technical Services: **1.800.631.0174**  
 BD Customer Service: **1.888.237.2762**  
[www.bd.com/vacutainer](http://www.bd.com/vacutainer)

For more information on V-Notch™ secondary labels please contact: Precision Dynamics Corporation  
 144 Tower Drive • Burr Ridge, IL 60527  
 Phone: **1.800.323.4840** • Fax: **1.800.548.5359**  
[www.pdcorp.com/v-notch](http://www.pdcorp.com/v-notch)

The PDC logo is a registered trademark of Precision Dynamics Corporation.  
 BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. ©2010 BD. 03/10 VS8814



POSITIVE ID. POSITIVE OUTCOMES.™