

BD Product Name: BD Pyxis Products Carefusion Coordination Engine for Department of Defence Customers

September 2019

Abstract: DoD CCE Management Console broken by multiple Microsoft patches

Microsoft® & Third-Party Patches

BD has identified patches from Microsoft® that have been identified as critical or security related.

Patches include updates that may adversely affect BD Pyxis Carefusion Coordination Engine 1.4.1 2008R2 for Department of Defense customers. These patches will not be and should not be installed at this time.

Customers that maintain patches independent of BD automated delivery should ensure these actions are not performed as the acting responsible entity in order to maintain the correct security posture of the system(s).

Note: These patches only apply to the latest supported version(s) of the BD offering

KB File	Patch Description	Patch ID	Notes
4499180	2019-05 Security Only Quality Update for Windows Server 2008 for x64-based Systems (KB4499180)	4499180	1
4507414	2019-07 Security Only Quality Update for .NET Framework 2.0, 3.0, 4.5.2, 4.6 for Windows Server 2008 SP2 for x64-based Systems (KB4507414)	4507414	1
890830	Windows Malicious Software Removal Tool - August 2019 for x64-based Systems (KB890830)	890830	1
4507452	2019-07 Security Monthly Quality Rollup for Windows Server 2008 for x64-based Systems (KB4507452)	4507452	1
4471325	2018-12 Security Monthly Quality Rollup for Windows Server 2008 for x64-based Systems (KB4471325)	4471325	1
4467706	2018-11 Security Monthly Quality Rollup for Windows Server 2008 for for x64-based Systems (KB4467706)	4467706	1
4489873	2018-11 Security Monthly Quality Rollup for Windows Server 2008 for for x64-based Systems (KB4489873)	4489873	1
4489880	2019-03 Security Monthly Quality Rollup for Windows Server 2008 for x64-based Systems (KB4489880)	4489880	1
4489876	2019-03 Security Only Quality Update for Windows Server 2008 for x64-based Systems (KB4489876)	4489876	1
4503273	2019-06 Security Monthly Quality Rollup for Windows Server 2008 for x64-based Systems (KB4503273)	4503273	1
4474419	2019-09 Security Update for Windows Server 2008 for x64-based Systems (KB4474419)	4474419	1
4480957	2019-01 Security Only Quality Update for Windows Server 2008 for x64-based Systems (KB4480957)	4480957	1
4487019	2019-02 Security Only Quality Update for Windows Server 2008 for x64-based Systems (KB4487019)	4487019	1
4516026	2019-09 Security Monthly Quality Rollup for Windows Server 2008 for x86-based Systems (KB4516026)	4516026	1



4499149	2019-05 Security Monthly Quality Rollup for Windows Server 2008 for x86-based Systems (KB4499149)	4499149	1
4487023	2019-02 Security Monthly Quality Rollup for Windows Server 2008 for x86-based Systems (KB4487023)	4487023	1

Notes

1. Listed Microsoft Updates are causing Carefusion Coordination Engine Management console to fail, patches are declined for install.