

BD Product Security - Security Patch Recommendations

BD Biosciences Product Name: BD FACSAria™ III and Fusion Cell Sorters

January / February / March 2022

Critical and/or Security Patches: The January 2022 security rollup/cumulative update was released on January 11, 2022. The February 2022 security rollup/cumulative update was released on February 8, 2022. The March 2022 security rollup/cumulative update was released on March 8, 2022.

Microsoft™ and Third-Party Patches

BD has identified patches from Microsoft™ that have been indicated as security related for the months of January, February and March. The BD products listed above have been tested with security patches from the January, February and March rollup/cumulative releases installed. The patches in these updates were not found to adversely affect operation of the listed BD products.

Customers that maintain patches independent of BD automated delivery should ensure the validated patches are installed on their BD systems as the acting responsible entity in order to maintain the correct security posture of the system(s). For products that are supported remotely by BD Service, patches will be applied according to customers' service agreements.

Windows™ 10 Enterprise:

Patch Name	Patch Description	Patch ID	Notes
January 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB5009557	1,2
February 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB5010351	1,3
March 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB5011503	1,4



Windows™ 10 Professional:

Patch Name	Patch Description	Patch ID	Notes
January 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H2	KB5009543	1,2
February 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H2	KB5010342	1,3
March 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H2	KB5011487	1,4

Notes

1. Patches may require the system to reboot.
2. For further information from Microsoft regarding CVEs covered by the January Security updates, please refer to the following link –
<https://msrc.microsoft.com/update-guide/releaseNote/2022-Jan>
3. For further information from Microsoft regarding CVEs covered by the February Security updates, please refer to the following link –
<https://msrc.microsoft.com/update-guide/releaseNote/2022-Feb>
4. For further information from Microsoft regarding CVEs covered by the March Security updates, please refer to the following link –
<https://msrc.microsoft.com/update-guide/releaseNote/2022-Mar>

For Research Use Only. Not for use in diagnostic or therapeutic procedures.
Class 1 Laser Product.

BD, the BD Logo and FACSARIA are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2022 BD. All rights reserved. BD-67215 (v1.0) 0722

