

BD Product Security - Security Patch Recommendations

BD Biosciences Product Name: BD FACSCanto™ 10-Color Cell Analyzers (IVD and RUO)

July / August / September 2020

Critical and/or Security Patches: The July 2020 security rollup/cumulative update was released on July 13, 2020. The August 2020 security rollup/cumulative update was released on August 10, 2020. The September 2020 security rollup/cumulative update was released on September 7, 2020.

Microsoft™ and Third-Party Patches

BD has identified patches from Microsoft that have been indicated as security related for the months of May and June. The BD products listed above have been tested with security patches from the May and June rollup/cumulative releases installed. The patches in these updates were not found to adversely affect operation of the listed BD products.

Customers that maintain patches independent of BD automated delivery should ensure the validated patches are installed on their BD systems as the acting responsible entity in order to maintain the correct security posture of the system(s). For products that are supported remotely by BD Service, patches will be applied according to customers' service agreements.

Windows™ 10 Enterprise:

Patch Name	Patch Description	Patch ID	Notes
July 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB4558998	1,2
August 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB4565349	1,3
September 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB4570333	1,4



Windows™ 10 Professional:

Patch Name	Patch Description	Patch ID	Notes
July 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v1909	KB4565483	1,2
August 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v1909	KB4565351	1,3
September 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v1909	KB4574727	1,4

Notes

1. Patches may require the system to reboot.
2. For further information from Microsoft regarding CVEs covered by the July Security updates, please refer to the following link –
<https://msrc.microsoft.com/update-guide/releaseNote/2020-Jul>
3. For further information from Microsoft regarding CVEs covered by the August Security updates, please refer to the following link –
<https://msrc.microsoft.com/update-guide/releaseNote/2020-Aug>
4. For further information from Microsoft regarding CVEs covered by the September Security updates, please refer to the following link –
<https://msrc.microsoft.com/update-guide/releaseNote/2020-Sep>

BD FACSCanto™ 10 Color Flow Cytometer is a Class 1 Laser Product.

The BD FACSCanto™ 10 Color Flow Cytometer is for In Vitro Diagnostic Use for up to six colors. Seven to ten colors are for Research Use Only.

BD, the BD Logo and FACSCanto are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2021 BD. All rights reserved. BD-29309 (v1.0) 0721

