

BD Product Security - Patch Whitelist / Blacklist

BD Biosciences Product Name: BD FACSCanto™ II Cell Analyzers

May / June 2020

Critical and/or Security Patches: The May 2020 security rollup/cumulative update was released in May 2020. The June 2020 security rollup/cumulative update was released in June 2020.

Microsoft™ and Third-Party Patches

BD has identified patches from Microsoft that have been indicated as security related for the months of May and June. The BD products listed above have been tested with security patches from the May and June rollup/cumulative releases installed. The patches in these updates were not found to adversely affect operation of the listed BD products.

Note: There were no updates released in May or June for Windows™ 7. Windows 7 support officially ended as of January 14, 2020.

Customers that maintain patches independent of BD automated delivery should ensure the validated patches are installed on their BD systems as the acting responsible entity in order to maintain the correct security posture of the system(s). For products that are supported remotely by BD Service, patches will be applied according to customers' service agreements.

Windows 10 Enterprise:

Patch Name	Patch Description	Patch ID	Notes
May 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB4551853 (Enterprise)	1,2
June 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB4561608 (Enterprise)	1,3

Windows 10 Professional:

Patch Name	Patch Description	Patch ID	Notes
May 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Pro (SAC) v1909	KB4556799 (Professional)	1,2
June 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Pro (SAC) v1909	KB4560960 (Professional)	1,3



Notes

1. Patches may require the system to reboot.
2. For further information from Microsoft regarding CVEs covered by the May Security updates, please refer to the following link –

<https://portal.msrc.microsoft.com/en-us/security-guidance/releasenotedetail/2020-May>

3. For further information from Microsoft regarding CVEs covered by the June Security updates, please refer to the following link –

<https://portal.msrc.microsoft.com/en-us/security-guidance/releasenotedetail/2020-Jun>

BD FACSCanto™ II Flow Cytometer is a Class 1 Laser Product.

For US, the BD FACSCanto™ II Flow Cytometer is for In Vitro Diagnostic Use for up to six colors. Seven and eight colors are for Research Use Only.

For EU, the BD FACSCanto™ II Flow Cytometer is CE marked in compliance with the European In Vitro Diagnostic Medical Device Directive 98/79/EC.

BD, the BD Logo and FACSCanto are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2021 BD. All rights reserved. BD-25607 (v1.0) 0721

