

## BD Product Security - Security Patch Recommendations

BD Biosciences Product Name: BD FACSCanto™ II Cell Analyzers

### January / February / March 2021

Critical and/or Security Patches: The January 2021 security rollup/cumulative update was released on January 12, 2021. The February 2021 security rollup/cumulative update was released on February 9, 2021. The March 2021 security rollup/cumulative update was released on March 9, 2021.

#### Microsoft™ and Third-Party Patches

BD has identified patches from Microsoft that have been indicated as security related for the months of January, February, and March. The BD products listed above have been tested with security patches from the January, February, and March rollup/cumulative releases installed. The patches in these updates were not found to adversely affect operation of the listed BD products.

Customers that maintain patches independent of BD automated delivery should ensure the validated patches are installed on their BD systems as the acting responsible entity in order to maintain the correct security posture of the system(s). For products that are supported remotely by BD Service, patches will be applied according to customers' service agreements.

#### Windows™ 10 Enterprise:

Patch Name	Patch Description	Patch ID	Notes
January 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB4598230</a>	1,2
February 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB4601345</a>	1,3
March 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5000822</a>	1,4



## Windows™ 10 Professional:

Patch Name	Patch Description	Patch ID	Notes
January 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v20H2	<a href="#">KB4598242</a>	1,2
February 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v20H2	<a href="#">KB4601319</a>	1,3
March 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v20H2	<a href="#">KB5000802</a>	1,4

### Notes

1. Patches may require the system to reboot.
2. For further information from Microsoft regarding CVEs covered by the January Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2021-Jan>
3. For further information from Microsoft regarding CVEs covered by the February Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2021-Feb>
4. For further information from Microsoft regarding CVEs covered by the March Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2021-Mar>

BD FACSCanto™ II Flow Cytometers are Class 1 Laser Products.

For US, the BD FACSCanto™ II Flow Cytometer is for In Vitro Diagnostic Use for up to six colors. Seven and eight colors are for Research Use Only.

For EU, the BD FACSCanto™ II Flow Cytometer is CE marked in compliance with the European In Vitro Diagnostic Medical Device Directive 98/79/EC.

BD, the BD Logo and FACSCanto are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2021 BD. All rights reserved. BD-41295 (v1.0) 0821

