

## BD Product Security - Security Patch Recommendations

BD Biosciences Product Name: BD FACSCanto™ II Cell Analyzers

### April / May / June 2021

Critical and/or Security Patches: The April 2021 security rollup/cumulative update was released on April 13, 2021. The May 2021 security rollup/cumulative update was released on May 11, 2021. The June 2021 security rollup/cumulative update was released on June 8, 2021.

#### Microsoft™ and Third-Party Patches

BD has identified patches from Microsoft™ that have been indicated as security related for the months of April, May and June. The BD products listed above have been tested with security patches from the April, May and June rollup/cumulative releases installed. The patches in these updates were not found to adversely affect operation of the listed BD products.

Customers that maintain patches independent of BD automated delivery should ensure the validated patches are installed on their BD systems as the acting responsible entity in order to maintain the correct security posture of the system(s). For products that are supported remotely by BD Service, patches will be applied according to customers' service agreements.

#### Windows™ 10 Enterprise:

Patch Name	Patch Description	Patch ID	Notes
April 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5001342</a>	1,2
May 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5003171</a>	1,3
June 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5003646</a>	1,4



## Windows™ 10 Professional:

Patch Name	Patch Description	Patch ID	Notes
April 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v20H2*	<a href="#">KB5001330</a>	1,2
May 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v20H2*	<a href="#">KB5003173</a>	1,3
June 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H1**	<a href="#">KB5003637</a>	1,4

\*Windows 10 version 20H2 released on October 20, 2020

\*\*Windows 10 version 21H1 released on May 18, 2021

### Notes

1. Patches may require the system to reboot.
2. For further information from Microsoft regarding CVEs covered by the April Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2021-Apr>
3. For further information from Microsoft regarding CVEs covered by the May Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2021-May>
4. For further information from Microsoft regarding CVEs covered by the June Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2021-Jun>

BD FACSCanto™ II Flow Cytometers are Class 1 Laser Products.

In the U.S., the BD FACSCanto™ II Flow Cytometer is for In Vitro Diagnostic Use for up to six colors. Seven and eight colors are for Research Use Only.

In the E.U., the BD FACSCanto™ II Flow Cytometer is CE marked.

BD, the BD Logo and FACSCanto are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2022 BD. All rights reserved. BD-49958 (v1.0) 0422

