

## BD Product Security - Security Patch Recommendations

BD Biosciences Product Name: BD FACSCanto™ II Cell Analyzers

### October / November / December 2020

Critical and/or Security Patches: The October 2020 security rollup/cumulative update was released on October 13, 2020. The November 2020 security rollup/cumulative update was released on November 10, 2020. The December 2020 security rollup/cumulative update was released on December 8, 2020.

#### Microsoft™ and Third-Party Patches

BD has identified patches from Microsoft that have been indicated as security related for the months of October, November, and December. The BD products listed above have been tested with security patches from the October, November, and December rollup/cumulative releases installed. The patches in these updates were not found to adversely affect operation of the listed BD products.

Customers that maintain patches independent of BD automated delivery should ensure the validated patches are installed on their BD systems as the acting responsible entity in order to maintain the correct security posture of the system(s). For products that are supported remotely by BD Service, patches will be applied according to customers' service agreements.

#### Windows™ 10 Enterprise:

Patch Name	Patch Description	Patch ID	Notes
October 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB4577668</a>	1,2
November 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB4586793</a>	1,3
December 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB4592440</a>	1,4



## Windows™ 10 Professional:

Patch Name	Patch Description	Patch ID	Notes
October 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v2004*	<a href="#">KB4579311</a>	1,2
November 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v2004*	<a href="#">KB4586781</a>	1,3
December 2020 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v20H2**	<a href="#">KB4592438</a>	1,4

\*Windows 10 version 2004 released on May 27, 2020

\*\*Windows 10 version 20H2 released on October 20, 2020

### Notes

1. Patches may require the system to reboot.
2. For further information from Microsoft regarding CVEs covered by the October Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2020-Oct>
3. For further information from Microsoft regarding CVEs covered by the November Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2020-Nov>
4. For further information from Microsoft regarding CVEs covered by the December Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2020-Dec>

BD FACSCanto™ II Flow Cytometer is a Class 1 Laser Product.

For US, the BD FACSCanto™ II Flow Cytometer is for In Vitro Diagnostic Use for up to six colors. Seven and eight colors are for Research Use Only.

For EU, the BD FACSCanto™ II Flow Cytometer is CE marked in compliance with the European In Vitro Diagnostic Medical Device Directive 98/79/EC.

BD, the BD Logo and FACSCanto are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2021 BD. All rights reserved. BD-31973 (v1.0) 0721

