

## BD Product Security - Security Patch Recommendations

BD Biosciences Product Name: BD FACSLytic™ Cell Analyzers (IVD and RUO)

### October / November / December 2021

Critical and/or Security Patches: The October 2021 security rollup/cumulative update was released on October 12, 2021. The November 2021 security rollup/cumulative update was released on November 9, 2021. The December 2021 security rollup/cumulative update was released on December 14, 2021.

#### Microsoft™ and Third-Party Patches

BD has identified patches from Microsoft™ that have been indicated as security related for the months of October, November, and December. The BD products listed above have been tested with security patches from the October, November, and December rollup/cumulative releases installed. The patches in these updates were not found to adversely affect operation of the listed BD products.

Customers that maintain patches independent of BD automated delivery should ensure the validated patches are installed on their BD systems as the acting responsible entity in order to maintain the correct security posture of the system(s). For products that are supported remotely by BD Service, patches will be applied according to customers' service agreements.

#### Windows™ 10 Enterprise:

Patch Name	Patch Description	Patch ID	Notes
October 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5006672</a>	1,2
November 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5007206</a>	1,3
December 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5008218</a>	1,4



## Windows™ 10 Professional:

Patch Name	Patch Description	Patch ID	Notes
October 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H1	<a href="#">KB5006670</a>	1,2
November 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H1	<a href="#">KB5007186</a>	1,3
December 2021 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H1	<a href="#">KB5008212</a>	1,4

## Notes

1. Patches may require the system to reboot.
2. For further information from Microsoft regarding CVEs covered by the October Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2021-Oct>
3. For further information from Microsoft regarding CVEs covered by the November Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2021-Nov>
4. For further information from Microsoft regarding CVEs covered by the December Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2021-Dec>

The BD FACSLytic™ Flow Cytometer is a Class 1 Laser Product.

In the U.S., the BD FACSLytic™ Flow Cytometer is for In Vitro Diagnostic Use with BD FACSuite™ Clinical Application for up to six colors.

In the U.S., the BD FACSLytic™ Flow Cytometer is for Research Use Only with BD FACSuite™ Application for up to 12 colors. Not for use in diagnostic or therapeutic procedures.

In the E.U., the BD FACSLytic™ Flow Cytometer with the BD FACSuite™ Clinical and BD FACSuite™ Applications is CE marked.

BD, the BD Logo, FACSLytic, and FACSuite are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2022 BD. All rights reserved. BD-61421 (v1.0) 0422

