

BD Product Security - Security Patch Recommendations

BD Biosciences Product Name: BD FACSLyric™ Cell Analyzers (IVD and RUO)

April / May / June 2022

Critical and/or Security Patches: The April 2022 security rollup/cumulative update was released on April 12, 2022. The May 2022 security rollup/cumulative update was released on May 12, 2022. The June 2022 security rollup/cumulative update was released on June 14, 2022.

Microsoft™ and Third-Party Patches

BD has identified patches from Microsoft™ that have been indicated as security related for the months of April, May and June. The BD products listed above have been tested with security patches from the April, May and June rollup/cumulative releases installed. The patches in these updates were not found to adversely affect operation of the listed BD products.

Customers that maintain patches independent of BD automated delivery should ensure the validated patches are installed on their BD systems as the acting responsible entity in order to maintain the correct security posture of the system(s). For products that are supported remotely by BD Service, patches will be applied according to customers' service agreements.

Windows™ 10 Enterprise:

Patch Name	Patch Description	Patch ID	Notes
April 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB5012647	1,2
May 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB5013941	1,3
June 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	KB5014692	1,4



Windows™ 10 Professional:

Patch Name	Patch Description	Patch ID	Notes
April 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H2	KB5012599	1,2
May 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H2	KB5013942	1,3
June 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H2	KB5014699	1,4

Notes

1. Patches may require the system to reboot.
2. For further information from Microsoft regarding CVEs covered by the April Security updates, please refer to the following link –
<https://msrc.microsoft.com/update-guide/releaseNote/2022-Apr>
3. For further information from Microsoft regarding CVEs covered by the May Security updates, please refer to the following link –
<https://msrc.microsoft.com/update-guide/releaseNote/2022-May>
4. For further information from Microsoft regarding CVEs covered by the June Security updates, please refer to the following link –
<https://msrc.microsoft.com/update-guide/releaseNote/2022-Jun>

The BD FACSLytic™ Flow Cytometer is a Class 1 Laser Product.

In the U.S., the BD FACSLytic™ Flow Cytometer is for In Vitro Diagnostic Use with BD FACSuite™ Clinical Application for up to six colors.

In the U.S., the BD FACSLytic™ Flow Cytometer is for Research Use Only with BD FACSuite™ Application for up to 12 colors. Not for use in diagnostic or therapeutic procedures.

In the E.U., the BD FACSLytic™ Flow Cytometer with the BD FACSuite™ Clinical and BD FACSuite™ Applications is an in vitro diagnostic medical device bearing a CE mark.

BD, the BD Logo, BD FACSLytic, and BD FACSuite are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2022 BD. All rights reserved. BD-69776 (v1.0) 0822

