

## BD Product Security - Security Patch Recommendations

BD Biosciences Product Name: BD FACSLyric™ Cell Analyzers (IVD and RUO)

### July / August / September 2022

Critical and/or Security Patches: The July 2022 security rollup/cumulative update was released on July 12, 2022. The August 2022 security rollup/cumulative update was released on August 9, 2022. The September 2022 security rollup/cumulative update was released on September 13, 2022.

#### Microsoft™ and Third-Party Patches

BD has identified patches from Microsoft™ that have been indicated as security related for the months of July, August and September. The BD products listed above have been tested with security patches from the July, August and September rollup/cumulative releases installed. The patches in these updates were not found to adversely affect operation of the listed BD products.

Customers that maintain patches independent of BD automated delivery should ensure the validated patches are installed on their BD systems as the acting responsible entity in order to maintain the correct security posture of the system(s). For products that are supported remotely by BD Service, patches will be applied according to customers' service agreements.

#### Windows™ 10 Enterprise:

Patch Name	Patch Description	Patch ID	Notes
July 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5015811</a>	1,2
August 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5016623</a>	1,3
September 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Enterprise (LTSC) v1809	<a href="#">KB5017315</a>	1,4



## Windows™ 10 Professional:

Patch Name	Patch Description	Patch ID	Notes
July 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H2	<a href="#">KB5015807</a>	1,2
August 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H2	<a href="#">KB5016616</a>	1,3
September 2022 Monthly Cumulative Updates	Security and quality updates for Microsoft Windows 10 Professional (SAC) v21H2	<a href="#">KB5017308</a>	1,4

## Notes

1. Patches may require the system to reboot.
2. For further information from Microsoft regarding CVEs covered by the July Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2022-Jul>
3. For further information from Microsoft regarding CVEs covered by the August Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2022-Aug>
4. For further information from Microsoft regarding CVEs covered by the September Security updates, please refer to the following link –  
<https://msrc.microsoft.com/update-guide/releaseNote/2022-Sep>

The BD FACSLyric™ Flow Cytometer is a Class 1 Laser Product.

In the U.S., the BD FACSLyric™ Flow Cytometer is for In Vitro Diagnostic Use with BD FACSuite™ Clinical Application for up to six colors.

In the U.S., the BD FACSLyric™ Flow Cytometer is for Research Use Only with BD FACSuite™ Application for up to 12 colors. Not for use in diagnostic or therapeutic procedures.

In the E.U., the BD FACSLyric™ Flow Cytometer with the BD FACSuite™ Clinical and BD FACSuite™ Applications is an in vitro diagnostic medical device bearing a CE mark.

BD, the BD Logo, BD FACSLyric and BD FACSuite are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2022 BD. All rights reserved. BD-75615 (v1.0) 1022

