

# Security Patches: BD Pyxis™ MedES Server

February 2023

BD has identified patches from Microsoft® that have been identified as critical or security related for February 2023 will not install on BD Pyxis™ MedES Server.

The patch includes updates that may adversely affect Pyxis™ MedES Server. This patch will not be and should not be installed on BD Pyxis™ MedES Server at this time.

Customers that maintain patches independent of BD automated delivery should ensure these actions are not performed as the acting responsible entity in order to maintain the correct security posture of the system(s).

Note: This patch only apply to the latest supported version(s) of the BD offering.

## Microsoft® patches

Patch name	Description	Patch ID	Notes
2023-02 Cumulative Security Update for Internet Explorer 11 for Windows Server 2008 R2 for x64-based systems	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022835</a>	1
2023-02 Security and Quality Rollup for .NET Framework 2.0, 3.0, 4.6.2 for Windows Server 2008 SP2	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022734</a>	1

2023-02 Security and Quality Rollup for .NET Framework 2.0, 3.0, 4.6.2 for Windows Server 2008 SP2 for x64	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system	<a href="#">KB5022734</a>	1
2023-02 Security and Quality Rollup for .NET Framework 3.5.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2008 R2 for x64	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022731</a>	1
2023-02 Security Monthly Quality Rollup for Windows Server 2008 for x64-based Systems	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022890</a>	1
2023-02 Security Monthly Quality Rollup for Windows Server 2008 for x86-based Systems	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022890</a>	1
2023-02 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022872</a>	1

2023-02 Security Only Quality Update for Windows Server 2008 for x64-based Systems	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022893</a>	1
2023-02 Security Only Quality Update for Windows Server 2008 for x86-based Systems	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022893</a>	1
2023-02 Security Only Quality Update for Windows Server 2008 R2 for x64-based Systems	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022874</a>	1
2023-02 Security Only Update for .NET Framework 2.0, 3.0, 4.6.2 for Windows Server 2008 SP2	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022786</a>	1
2023-02 Security Only Update for .NET Framework 2.0, 3.0, 4.6.2 for Windows Server 2008 SP2 for x64	A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.	<a href="#">KB5022786</a>	1

<p>2023-02 Security Only Update for .NET Framework 3.5.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2008 R2 for x64</p>	<p>A security issue has been identified in a Microsoft software product that could affect your system. You can help protect your system by installing this update from Microsoft. For a complete listing of the issues that are included in this update, see the associated Microsoft Knowledge Base article. After you install this update, you may have to restart your system.</p>	<p><a href="#">KB5022783</a></p>	<p>1</p>
<p>Windows Malicious Software Removal Tool x64 - v5.110</p>	<p>After the download, this tool runs one time to check your computer for infection by specific, prevalent malicious software (including Blaster, Sasser, and Mydoom) and helps remove any infection that is found. If an infection is found, the tool will display a status report the next time that you start your computer. A new version of the tool will be offered every month. If you want to manually run the tool on your computer, you can download a copy from the Microsoft Download Center, or you can run an online version from microsoft.com. This tool is not a replacement for an antivirus product. To help protect your computer, you should use an antivirus product.</p>	<p><a href="#">KB890830</a></p>	<p>1</p>

**Note:**

1. Due to license issue on Medes Sever 2008 KBs were not installed.

BD, Franklin Lakes, NJ, 07417, U.S.  
201.847.6800

**bd.com**

BD and the BD Logo are trademarks of Becton, Dickinson and Company or its affiliates. © 2022 BD. All rights reserved.



# Security Patches:

## BD Pyxis™ MedStation™ 3500

February 2023

BD has identified patches from Microsoft® that have been identified as critical or security related for February 2023 will not install on BD Pyxis™ MedStation™ 3500.

The patches include updates that may adversely affect BD Pyxis™ MedStation™ 3500. These patches will not be and should not be installed on BD Pyxis™ MedStation™ 3500 at this time.

Customers that maintain patches independent of BD automated delivery should ensure these actions are not performed as the acting responsible entity in order to maintain the correct security posture of the system(s).

Note: These patches only apply to the latest supported version(s) of the BD offering.

### Microsoft® patches

Patch name	Description	Patch ID	Notes
2023-02 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems	<p>This cumulative security update contains improvements that are part of update KB5022872 (released January 10, 2023).</p> <ul style="list-style-type: none"><li>This update contains miscellaneous security improvements to internal Windows OS functionality. No specific issues are documented for this release.</li></ul>	<a href="#">KB5022872</a>	1
2023-02 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems	<ul style="list-style-type: none"><li>This update contains miscellaneous security improvements to internal Windows OS functionality. No additional issues were documented for this release.</li></ul>	<a href="#">KB5022874</a>	1

Note:-

1. MS3500, Workstation (3K136W1) the above patches are getting reverted and failing after installing and restarting the device.

BD, Franklin Lakes, NJ, 07417, U.S.  
201.847.6800

[bd.com](#)

BD and the BD Logo are trademarks of Becton, Dickinson and Company or its affiliates. © 2022 BD. All rights reserved.



# Security Patches: BD Pyxis™ MedStation™ 4000

February 2023

BD has identified patches from Microsoft® that have been identified as critical or security related for February 2023 will not install on BD Pyxis™ MedStation™ 4000.

The patches include updates that may adversely affect BD Pyxis™ MedStation™ 4000. These patches will not be and should not be installed on BD Pyxis™ MedStation™ 4000 at this time.

Customers that maintain patches independent of BD automated delivery should ensure these actions are not performed as the acting responsible entity in order to maintain the correct security posture of the system(s).

Note: These patches only apply to the latest supported version(s) of the BD offering.

## Microsoft® patches

Patch name	Description	Patch ID	Notes
Windows Malicious Software Removal Tool - v5.110	The Windows Malicious Software Removal Tool (MSRT) helps remove malicious software from computers.	<a href="#">KB890830</a>	1

Note:-

1. MS4000, 2008 Console (4k5544S1) RSS remote agent is unavailable, due to which the above KB was not installed.

BD, Franklin Lakes, NJ, 07417, U.S.  
201.847.6800

[bd.com](#)

BD and the BD Logo are trademarks of Becton, Dickinson and Company or its affiliates. © 2022 BD. All rights reserved.



# Security Patches: BD Pyxis™ Supply

February 2023

BD has identified patches from Microsoft® that have been identified as critical or security related for February 2023 will not install on BD Pyxis™ Supply.

The patch includes updates that may adversely affect Pyxis™ Supply. This patch will not be and should not be installed on BD Pyxis™ Supply at this time.

Customers that maintain patches independent of BD automated delivery should ensure these actions are not performed as the acting responsible entity in order to maintain the correct security posture of the system(s).

Note: This patch only apply to the latest supported version(s) of the BD offering.

## Microsoft® patches

Patch name	Description	Patch ID	Notes
2023-02 Cumulative Security Update for Internet Explorer 9 for Windows Server 2008 for x86-based systems (KB5022835)	This security update resolves vulnerabilities in Internet Explorer. To learn more about these vulnerabilities, see Deployments   Security Update Guide.	<a href="#">KB5022835</a>	1
2023-02 Security Monthly Quality Rollup for Windows Server 2008 for x86-based Systems (KB5022890)	This cumulative security update contains improvements that are part of update KB5022890 (released January 10, 2023). <ul style="list-style-type: none"><li>This update contains miscellaneous security improvements to internal Windows OS functionality. No specific issues are documented for this release.</li></ul>	<a href="#">KB5022890</a>	1
2023-02 Security Only Quality Update for Windows Server 2008 for x86-based Systems (KB5022893)	This update contains miscellaneous security improvements to internal Windows OS functionality. No additional issues were documented for this release.	<a href="#">KB5022893</a>	1

Windows Malicious Software  
Removal Tool x64 - v5.110

The Windows Malicious Software Removal Tool (MSRT) helps remove malicious software from computers. Microsoft releases the MSRT on a monthly cadence as part of Windows Update or as a standalone tool. Use this tool to find and remove specific prevalent threats and reverse the changes they have made.

[KB890830](#)

1

Note:

1. Due to environment issue patch was unable to be validated this month.

BD, Franklin Lakes, NJ, 07417, U.S.  
201.847.6800

[bd.com](#)

BD and the BD Logo are trademarks of Becton, Dickinson and Company or its affiliates. © 2022 BD. All rights reserved.

