

BD Product Name: **BD Pyxis™ Supply**

Date of Critical or Security Patches: Dec 2021

Abstract: Critical or Security Patches – Dec 2021

Microsoft® & Third-Party Patches

BD has identified patches from Microsoft® that have been identified as critical or security related for Dec 2021 will not install on BD Pyxis™ Supply.

The patch includes updates that may adversely affect Pyxis™ Supply. This patch will not be and should not be installed on BD Pyxis™ Supply at this time.

Customers that maintain patches independent of BD automated delivery should ensure these actions are not performed as the acting responsible entity in order to maintain the correct security posture of the system(s).

Note: This patch only apply to the latest supported version(s) of the BD offering.

Patch Name	Description	Patch ID	Notes
2021-12 Security Monthly Quality Rollup for Windows Server 2008 for x86-based Systems	This security update includes improvements and fixes that were a part of update KB5007263 (released November 9, 2021) and addresses the following issues: Update to support the cancellation of daylight savings time (DST) for 2021 for the Republic of Fiji. Addresses a known issue that causes error codes 0x000006e4, 0x0000007c, or 0x00000709 when connecting to a remote printer that is shared on a Windows print server. Addresses a known issue that might prevent apps, such as Kaspersky apps, from opening after you attempt to repair or update the apps using the Microsoft Installer (MSI).	KB5008274	1
2021-12 Security Only Quality Update for Windows Server 2008 for x86-based Systems	This security update includes quality improvements. Key changes include: Update to support the cancellation of daylight savings time (DST) for 2021 for the Republic of Fiji.	KB5008271	1



	Addresses a known issue that causes error codes 0x000006e4, 0x0000007c, or 0x00000709 when connecting to a remote printer that is shared on a Windows print server. Addresses a known issue that might prevent apps, such as Kaspersky apps, from opening after you attempt to repair or update the apps using the Microsoft Installer (MSI).		
--	--	--	--

Note:

1. Due to environment issue patch was unable to be validated this month.