


# Swan-Ganz Jr Pulmonary Artery Catheters

## Specifications and Computation Constants

Over 50 years of innovation and experience applied to Swan-Ganz catheters, now indicated for adolescent patients ages 12 and up.

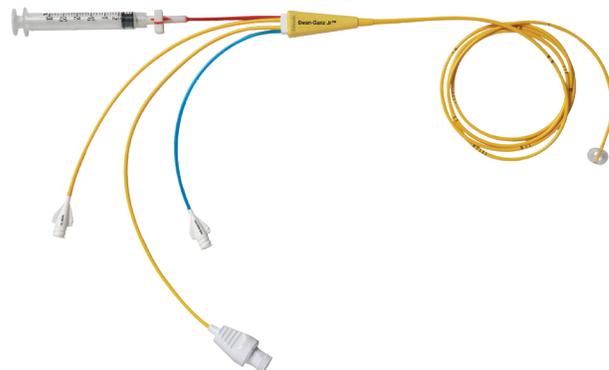

### Models / catheter sizes

- SGPT54 (5 Fr)
- SGPT64P (6 Fr)
- SGPT755P (7.5 Fr)

### Parameters

- Continuous right heart pressures (PAP, RAP)
- Intermittent wedge pressure (PAOP)
- Intermittent thermodilution cardiac output (ICO)
- Mixed venous blood sampling (SvO<sub>2</sub>)
- Continuous core temperature

### Intermittent Derived Parameters

- Systemic vascular resistance (SVR)
- Pulmonary vascular resistance (PVR)
- Stroke volume (SV)
- Indexed values (CI, SVI, SVRI)

### Specifications

Model Number	SGPT755P	SGPT64P	SGPT54
<b>French</b>	<b>7.5 F</b>	<b>6 F</b>	<b>5 F</b>
<b>Usable Length (cm)</b>	110	110	75
<b>Balloon Inflation Capacity (ml)</b>	1.5	1.0	0.7
<b>Distance from Tip</b>			
Injectate Port	30	30	15
Infusion (VIP) Port	31	–	–
<b>Lumen Volume (ml)</b>			
Distal Lumen	0.86	0.92	0.64
Injectate Lumen	0.86	0.74	0.57
VIP/RA Lumen	0.87	–	–
<b>Infusion Rate* (ml/min)</b>			
Infusion (VIP) Lumen	13	5 (distal), 7 (injectate)	–
<b>Maximum Guidewire Diameter</b>			
Distal Lumen	0.025 in (0.64 mm)	0.025 in (0.64 mm)	0.020 in (0.51 mm)



## Computation Constants

Bath Probe Injectate Temperature		SGPT755P	SGPT64P	SGPT54
Injectate Temp (°C)	Injectate Volume (ml)	Computation Constants (CC)*		
0-5	10	0.564	0.547	–
	5	0.262	0.259	0.270
	3	0.139	0.144	0.154
	1	–	–	0.037
19-22	10	0.588	0.582	–
	5	0.283	0.280	0.292
	3	0.158	0.161	0.170
	1	–	–	0.048
23-25	10	0.612	0.608	–
	5	0.301	0.305	0.307
	3	0.177	0.180	0.181
	1	–	–	0.055
In-Line Injectate Temperature (CO-Set+)	Injectate Volume (ml)	Computation Constants (CC)*		
<b>Cold Injectate</b>				
6°C -12°C	10	0.578	0.558	–
8°C -12°C	5	0.272	0.277	0.285
<b>Room Temp</b>				
18°C -25°C	10	0.592	0.607	–
18°C -25°C	5	0.290	0.301	0.307

\*CC = (1.08) C<sub>T</sub> (60)(V<sub>I</sub>)

CAUTION: Federal (United States) law restricts this device to sale by or on the order of a physician. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

BD, Irvine, CA 92618 USA

[bd.com](http://bd.com)



BD, the BD logo, Swan, Swan-Ganz and Swan-Ganz Jr are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2025 BD. All rights reserved. BD-149994 v1.0 US